SIDAPHARM Surgical Drapes

SIDAPHARM Surgical Gowns

SIDAPHARM Surgical Pack

INSTRUCTIONS OF USE

SIDAPHARM Surgical Drapes are made of non-woven material and are used for surface covering and they can serve surgical procedures of different scopes.

SIDAPHARM Surgical Gowns are made of non-woven material and are used to protect the healthcare personnel during surgical procedure.

SIDAPHARM Surgical Pack is a combination of certain pieces of surgical apparel, provided in a single packaging for reasons of time-saving, improved efficiency, reduced procurement cost & packaging waste.

INTENDED USE

Surgical Drapes are used as a barrier to liquid and microbial penetration between patient and operating room, the other surgical devices and the OR personnel.

- The plain *Surgical Drape* can be placed either below or above the patient that lies on the surgical bed. It can serve surgical procedures of different scopes.
- The Mayo Surgical Cover has a bag-style shape and is used to cover (at least) the horizontal surface of the mayo stand and later on to carry the waste (such as cotton, sponges, disposable instruments etc.) used in the surgical procedure. The area in the middle has PE reinforcement. It can serve surgical procedures of different scopes.
- The *Ophthalmic Drape* is placed over the head of the patient. It has a small aperture that leaves uncovered the area around one eye of the patient. It is usually used for minimal ophthalmic procedures, such as drug injection into the eye.
- The *Ophthalmic Drapes with one or two pouches* are placed over the head of the patient. They have a slightly bigger fenestration area in order to let surgeons perform on patient's eye procedures such as cataract extraction. The existence of pouch(es) facilitates the collection of the fluids generated by the irrigation of solutions to the ophthalmic area during the surgical procedure.

Surgical Gowns are used as a barrier to liquid and microbial penetration between patient and operating room, the other surgical devices and the OR personnel. The *standard* ones are intended for standard and short duration surgical operations (e.g. ophthalmic procedures), while the *reinforced* ones (with strengthened areas on chest and sleeves) are intended for heavy and long duration surgical operations (angiography; cardiovascular etc.).

The aforementioned apparel is also available with one or two towels made of cellulose, for the purpose of hand and/or face sweeping. Also, some articles are provided covered all over with a non-woven wrap inside the primary packaging. This way, when wrapped content is removed from its packaging and is placed on a surface, the wrap provides an extra sanitary protection to the content.

Surgical Pack is a combination of certain pieces of the aforementioned surgical apparel, provided in a single packaging. Plastic (PE) Bags can be additionally found as accessories that can be used for various reasons; e.g. sorting of surgical tools, disposal of used devices or biological material.

CONTRAINDICATIONS

None reported. However, surgeons should be aware whether patient is allergic to the material of any of the components.

STERILITY

Sterilized by ethylene oxide.

PRECAUTIONS

- If there is any damage of the sterile package, sterility may be broken. In this case, do not use the contents and discard them appropriately.
- Apply aseptic technique while peel opening the container pouch.
- If expiration date has past, the components should not be used.
- Do not resterilize.
- Do not reuse the contents of the pack.
- Store at ambient temperature (below 37 °C) and protect from moisture.

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