
Warning

If the reagent accidentally gets into your mouth, seek medical assistance immediately; if the reagent spills on your skin, wash it off with plenty of water immediately; if the reagent spills into your eyes, wash it off with plenty of water and seek medical assistance immediately.

Note:

This product must only be used by Mindray authorized personnel or skilled/trained clinical professionals.

GRAPHICAL SYMBOLS

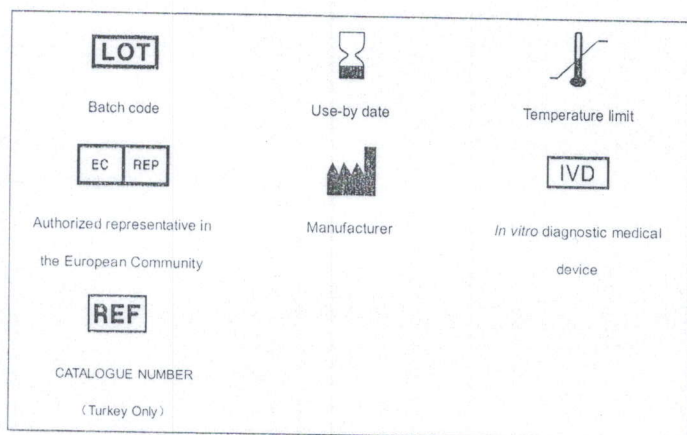IV

M-30D DILUENT

1. Product Name

M-30D DILUENT.

2. Intended Use

For use on Mindray BC-20/BC-21/BC-10/BC-11/BC-3000Plus/BC-3000CT/BC-3200/BC-3300/BC-3 BC-20s/BC-21s/BC-30s/BC-31s/BC-30/BC-31/BC-30e/BC-10e hematology analyzer.

For use as an azide-free, filtered isotonic solution for counting and sizing blood cells.

3. Active Ingredients

Sodium Chloride.....	3.0-5.5g/L
Sodium Sulfate Anhydrous.....	7.5-11.5g/L
Buffering Agents.....	1.0-3.0g/L
Anti-fungal and Anti-bacterial Agents.....	0.8-2.5g/L

4. Warnings and Precautions

Do not inhale and/or ingest. In case of inhalation or ingestion, seek medical advice immediately. Avoid skin contact. In case of skin contact, flush the affected area with plenty of water immediately. Avoid eye contact. In case of eye contact, flush the affected area with plenty of water and seek medical advice immediately.

5. Storage and Usage

The product must be stored in the temperature of 2°C to 30°C (35°F to 86°F), and the relative humidity must not exceed 90%. The storage room must be well ventilated and without corrosive gas. The shelf life is 2 years.

The product must be used in the temperature of 15°C to 30°C (59°F to 86°F), and it stays valid for 60 days after being opened.

Do not use expired products.

If product has been frozen (either partially or completely), warm it up to room temperature and then use it by gentle inversion. You can only proceed to test samples after you have confirmed that measured blank value is within the required range.

Dispose of waste product, unused product and contaminated packaging in accordance with governmental requirements.

The following conditions may produce interference to test results: expired reagent, reagent contaminated by dusts, unclear sample dilution, reagent mixed with those of other manufacturers.

1

M-30CFL LYSE

1. Product Name

M-30CFL LYSE

2. Intended Use

For use on Mindray BC-3000Plus/BC-3000CT/BC-3200/BC-3300/BC-3600/BC-20s/BC-21s/BC-30s/BC-31s hematology analyzer.

For use as a cyanide-free lytic reagent for quantitatively determining hemoglobin and for counting and sizing leukocytes. Use as directed in instrument operator's manual.

3. Active Ingredients

Quaternary Ammonium Salts.....	<50g/L
Nonionic Surfactant.....	<15g/L
Isopropanol.....	0.1-1.5ml/L
Ethanol.....	<1.5ml/L

4. Warnings and Precautions

Do not inhale and/or ingest. In case of inhaling or ingestion, seek medical advice immediately. Avoid contact with skin. In case of contact with skin, rinse immediately with plenty of water. Avoid contact with eyes. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

5. Storage and Usage

The product should be stored at 2°C to 30°C (35°F to 86°F) and keep container closed. The shelf life is 2 years.

If the product has been frozen (either partially or completely), warm it up to room temperature and then mix it by gentle inversion. Verify background results before analyzing patient samples.

The product should be used at 15°C to 30°C (59°F to 86°F) once opened, and it is stable for 60 days.

Dispose of waste product, unused product and contaminated packaging in accordance with governmental requirements.

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