READ THESE INSTRUCTIONS

1 This is a ZK3B-53L Category 3 protective coverall.

2 ZK3B-53L coverall is limited use clothing offering protection against biological agents and chemical substances.

3 The manufacturer is Portavita SRL see ref.2

4 The product number (ZK3B-53L).

5 Read this instruction sheet before use.

6 Size Code- see garment bag for size information

GARMENT USE

Check you have the right size garment. Use only where the risk of chemical exposure is low and the potential is defined as low risk. Check the garment carefully before wearing. When used with additional equipment, such as gloves and/or face mask, enhanced protection can be achieved by taping the garment to the additional equipment.

LIMITATIONS OF USE

The product is intended to be used as protective clothing for the protection of biological risk, and protection of the whole body against chemicals, against which a total barrier against the permeation of liquids (at the molecular level) is not necessary- acids and bases of low concentration, hydrocarbons, alcohols- exposure to light sprays, liquid aerosols, accidental splashes, protection against superficial mechanical injuries (abrasion, hanging) – can be used in normal environments without potentially explosive atmosphere and without risk of contact with open flame.

STEPS TO PUT ON THE GARMENT

1. Remove all personal items (jewelry, watches, cell phones, pens, etc.)
2. Move to the clean area at the entrance of the isolation unit.
3. By visual inspection, ensure that all sizes of the garment set are correct and the quality is appropriate.
4. Undertake the procedure of putting on the garment under the guidance and supervision of a trained observer (colleague).
5. Put on gloves (examination, nitrile gloves).
6. Step into the coveralls and pull up to the neck.
7. Put on face mask.
8. Put on face shield or goggles.
9. Put on head and neck covering surgical bonnet covering neck and sides of the head or hood.
10. Put on second pair of (preferably long cuff) gloves over the cuff.

SUITABILITY

Product performance data - See Ref. 1 the employer must carry out a safety assessment and provide personal protective equipment that is fit for purpose. Portavita SRL accepts no responsibility for the use of the garment in an improper way for advice on the garment or its usage applications, you can contact www.portavita.md

STORAGE

It should be in clean rooms at a temperature between -10 +20 , away from the action of sunlight and heat sources

PRODUCT DISPOSAL

Means of disposal will depend on any contamination and local regulations. The preferred method is incineration with energy recovery.

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