

Clino'wipes

WIPES IMPREGNATED WITH A
DETERGENT AND DISINFECTANT
SOLUTION



■ PRODUCT BENEFITS

- Active from 2 min. (EN 16615)
- Cleaning and disinfection of medical devices in one operation
- Complete virucidal activity (naked and enveloped viruses)
- Food contact formula
- Active on *Clostridium difficile* (impregnation solution)

www.sterifrance.com

Clino'wipes

WIPES IMPREGNATED WITH A DETERGENT AND DISINFECTANT SOLUTION

INSTRUCTIONS FOR USE

■ When using the wipes for the first time:

- Put on gloves and remove the lid of the box
- Remove the lid and take out the 1st wipe (in the center of the roll)
- Pass the wipe through the opening of the box lid

■ Use :

- 1- Thoroughly clean by mechanical wiping the surface of the material to be treated
- 2- Contact time: according to the desired antimicrobial activity
- 3- Do not rinse*
*Does not require rinsing, unless otherwise specified by the surface coating manufacturer or for surfaces in contact with foodstuffs, skin and mucous membranes.
- 4- Discard the support
- 5- Close the lid after each use
- 6- Repeat the operation if necessary

DISINFECTING PROPERTIES

Activity	Standards	Contact time	
Bactericidal	EN 16615	2 min.	
	EN 13727	5 min.	
	EN 1276		
	EN 13697		
EN 14561			
Yeasticidal	EN 16615	2 min.	
	EN 13624	5 min.	
	EN 1650		
	EN 13697	10 min.	
	EN 14562		
Fungicidal	EN 13624	10 min.	
	EN 14562		
Virucidal	EN 14476	<i>Norovirus</i>	5 min.
		<i>Adenovirus</i>	10 min.
		<i>Poliovirus</i>	
		<i>Rotavirus</i>	
Tuberculocidal	EN 14348	<i>M.terrae</i>	10 min.
Sporicidal	EN 13704	<i>Clostridium difficile</i>	15 min.

PRECAUTIONS OF USE

Refer to the Safety Data Sheet

FEATURES

- Aspect : white non-woven wipes
- pH of the impregnation solution: 7

COMPOSITION

- Dodecylamine
- Ethanol
- Isopropanol
- Surfactants
- Excipients

PACKAGING

- ① Box of 12 canisters of 150 wipes ref. **10795166**



CE
0459

Dangerous product - follow the precautions for use. Read the label and product information before use. Class IIb medical device.