iChemVELOCITY Wash Solution

REF 800-7217

TABLE OF CONTENTS

English	2
Français (FR)	4
Deutsch (DE)	6
Italiano (IT)	8
Español (ES)	10
Português Portugal (PT-PT)	12
Dansk (DA)	14
Svenska (SV)	16
Norsk (NO)	18
Ελληνικά (EL)	20
日本語 (JP)	22
中文 (ZH-CN)	24
Lietuviškai (LT)	26
Magyar (HU)	28
Polski (PL)	30
Čeština (CZ)	32
Slovenčina (SK)	34
한국어 (KO)	36
Türkçe (TR)	38
Русский (RU)	40
eesti keel (ET)	43
Hrvatski (HR)	45
Български (BG)	47
Română (RO)	49
Србија (SR)	51
Latviešu (LV)	53
Українська (UK)	55
Português Brasil (PT-BR)	57
Tiếng Việt (VI)	59

iChemVELOCITY Wash Solution

REF 800-7217

Rx Only in the U.S.A.

INTENDED USE

The iChemVELOCITY Wash Solution is a buffer solution for routine rinse of the iChemVELOCITY or DxU 810c Series of Automated Urine Chemistry Analyzers. This reagent is intended for *in vitro* diagnostic use with the iChemVELOCITY or DxU 810c Series of Automated Urine Chemistry Analyzers.

INTENDED USER

The device is intended to be used by a laboratory professional.

PRINCIPLE

The iChemVELOCITY Wash Solution is a ready-for-use solution specifically developed for the application in the iChemVELOCITY or DxU 810c Series of Automated Urine Chemistry Analyzers. Use of this Wash Solution properly provides the optimal operating condition for the System.

WARNING AND PRECAUTIONS

- The iChemVELOCITY Wash Solution has been determined to be nonhazardous under the guidelines issued by OSHA in 29 CFR 1910. 1200(d).
- Appropriate precautions in the collection, handling, storage and disposal of specimens should be exercised accordingly.
- Product is colorless, clear liquid with aromatic odor. Product that differs from these attributes should be discarded as they may have deteriorated.
- Avoid contact of solutions with eyes and skin. Refer to the Safety Data Sheet for first aid measures.
- Wear appropriate personal protective equipment when handling.
- Avoid long term exposure to skin or by inhalation. Refer to the Safety Data Sheet for first aid measures.
- Consult local or state authorities with regard to recommended method of disposal.

GHS HAZARD CLASSIFICATION

Not classified as hazardous

Safety Data Sheet is available at beckmancoulter.com/techdocs

NON-REACTIVE INGREDIENTS

- Water
- Stabilizers
- Preservatives

STORAGE AND STABILITY

- Store at 20°C to 28°C (68°F to 82.4°F).
- iChemVELOCITY Wash Solution is stable for twelve months from the date of manufacture.
- Open expiration: Unused solutions that remain in the original capped container are stable until the expiration date.
- Do not freeze
- Protect the solutions against light.
- Do not use solutions after the expiration date printed on the bottle label.

MATERIALS PROVIDED

- 2 x 7L iChemVELOCITY Wash Solution
- 2 x 1 µm Filter

MATERIALS REQUIRED, BUT NOT PROVIDED

- iChemVELOCITY Wash Solution Tubing Assembly

- iChemVELOCITY or DxU 810c Series of Automated Urine Chemistry Analyzers
- Personal Protective Equipment

PROCEDURE

Follow the directions in the iChemVELOCITY or DxU 810c Instructions for Use. The iChemVELOCITY or DxU 810c will automatically notify the operator when the Wash Solution liquid level is below minimal requirements.

TECHNICAL SUPPORT

For additional information, or if damaged product is received, call Beckman Coulter Customer Service at 800-854-3633 (USA or Canada) or contact your local Beckman Coulter Representative.

A Glossary of Symbols (PN C27648) is available at beckmancoulter.com/techdocs.

NOTICE TO USER

For a patient/user/third party in the European Union and in countries with identical regulatory regime (*Regulation 2017/746/EU on In vitro Diagnostic Medical Devices*); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

TRADEMARKS

Beckman Coulter, the stylized logo, and the Beckman Coulter product and service marks mentioned herein are trademarks or registered trademarks of Beckman Coulter, Inc. in the United States and other countries.

May be covered by one or more pat. -see www.beckmancoulter.com/patents.

REVISION HISTORY

Revision AA, 03/2022

Initial Release

iChemVELOCITY Wash Solution (Solution de lavage iChemVELOCITY)

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MATÉRIEL FOURNI

- 2 x 7 L iChemVELOCITY Wash Solution (Solution de lavage iChemVELOCITY)

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Beckman Coulter Ireland Inc., Lismeehan, O'Callaghan's Mills, Co. Clare, Ireland +(353) (0) 65 683 1100