Specific Proteins Calibrator

Package size

5×1 mL

Intended use

Specific Proteins Calibrator is used for the calibration of quantitative determination of specific proteins on Mindray BS measurement system.

Summary

Specific Proteins Calibrator contains the following analytes:

Complement C3 (C3)	Complement C4 (C4)	C- Reactive protein (CRP)
Immunoglobulin A(IgA)	Immunoglobulin G (IgG)	Immunoglobulin M(IgM)

By calibrating the selected analytes, Mindray BS-series chemistry analyzers can generate valid calibration factors, and then calculate the concentrations of each sample automatically after analysis. Mindray BS measurement system is composed of Mindray BS-series chemistry analyzers, Mindray reagent kits, calibrators and control materials.

Components

Specific Proteins Calibrator is a liquid calibrator based on human serum. The concentrations of the calibrator components are lot-specific.

Warnings and Precautions

1. For in vitro diagnostic use only.
2. Professional or trained person is required. Please take the necessary precautions for handling all laboratory reagents.
3. The calibrator values are lot-specific with the matched models listed in the value sheet.
4. Calibration is recommended when reagent lot is changed or certain maintenance or troubleshooting procedure is performed.
5. Please perform scheduled maintenance and standard operation including calibration and analysis to assure the performance of measurement system.
6. The calibrator was tested with Elisa methods and shown to be negative for antibodies to HIV and HCV and nonreactive for HBsAg. However, as no testing method can rule out the potential risk of infection with absolute certainty, so this material should be handled as a patient specimen¹.
7. Material safety data sheet is available on request.
8. Disposal of all waste material should be in accordance with local guidelines.

Preparation

1. Take the vial out of the freezer and allow reaching room temperature.
2. According to the calibration procedure described in the instrument manual, use the Calibrator value and Dilution factor listed in the value sheet to set calibration parameter and execute calibration.
3. Mindray BS-series chemistry analyzers can generate a series of standard calibrator S2-S6 (S1:0.9% NaCl) automatically from a single standard. Details of setting dilution parameter are available on request.

Storage and Stability

The calibrator is stable up to expiration date indicated on the label when stored in unopened vial at 2~8°C and protected from light. Once opened, it is stable for 28 days when capped tightly and avoiding microbial contamination at 2~8°C.

Materials required but not provided

1. Mindray BS Chemistry analyzers and reagent kits.
2. General laboratory equipment and 0.9% NaCl.

Traceability of Calibrator

The calibrator values assigned by Mindray standard transfer procedure and routine method are listed in the value sheet. The traceability process is based on ISO 17511², each analyte (measurand) in this calibrator is traceable to the available reference material. Traceability information is given below. Details of traceability are available on request.

Analytes	Reference methods/materials
C3, C4, IgA, IgG, IgM	ERM-DA470K
CRP	ERM-DA472

Quality control

Use Mindray control as sample to monitor the status of calibration. The control result should be within the defined range as shown in the control value sheet. If the control result falls outside the range, the measurement system should be checked. For instance, position of reagent or sample placed on the analyzer; expiration date or storage condition of the calibrator, reagent and control; the parameter setting in software; or performance of the analyzer.

References

1. Biosafety in Microbiological and Biomedical Laboratories; U.S. Department of Health and Human Services; US Government Printing Office; Washington: 2007.
2. ISO 17511:2003. In vitro diagnostic medical devices—Measurement of quantities in biological samples—Metrological traceability of values

mindray

assigned to calibrators and control materials.

Graphical symbols

In Vitro Diagnostic
medical device

Batch Code

European
Conformity

Authorized representative in
the European Community

Use By

Consult
Instructions for use

Biological
Risks

Temperature
Limit

Manufacturer

Catalogue
number

© 2024 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. All rights reserved

Manufacturer: Shenzhen Mindray Bio-Medical Electronics Co., Ltd.

Address: Mindray Building, Keji 12th Road South, High-Tech Industrial Park, Nanshan, Shenzhen, 518057 P.R.China

E-mail Address: service@mindray.com

Website: www.mindray.com

Tel: +86-755-81888998

Fax: +86-755-26582680

EC-Representative: Shanghai International Holding Corp.
GmbH(Europe)

Address: Eiffestraße 80, Hamburg 20537, Germany

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