

5.1 Registered Contents of Radiography Programs



NOTE

Several examples of radiography programs are factory-installed.

Before using the equipment, set appropriate radiography programs according to the applications.




NOTE

When setting radiography programs for pediatric patients, include the name such as "child" or "infant" in the programs or steps.

The contents registered in radiography programs are shown below.

Radiography program

- Name
- Body thickness
- Technique
- Focus
- kV, mA or mAs, sec
- AEC (optional)
 - AEC ON/OFF
 - Density
 - Photo pickup field
 - Speed
- Comment
- Radiography step name
- Communication number

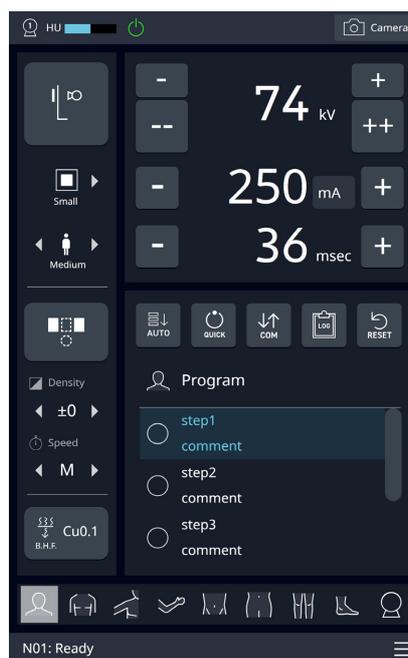

NOTE

When the camera option is combined, perform the following settings.

- Camera image zoom ratio
- ON/OFF of the motion detection function
- The threshold value of the body motion detection