



Better Care, Better Life

JINHUA JINGDI MEDICAL SUPPLIES CO., LTD.

Professional manufacturer for medical and healthcare disposables



JINGDI MEDICAL

www.jhjingdi.com



ADD: Ditian Function, Xiaoshun, Jindong,
Jinhua, Zhejiang, China
地址: 浙江省金华市金东区孝顺镇低田功能区2幢

TEL: (86)0579-89116759
FAX: (86)0579-89116751

Website: www.jhjingdi.com
E-mail: jingdi@jhjingdi.com



Better Care, Better Life

JINHUA JINGDI MEDICAL SUPPLIES CO., LTD.



Profile

Jingdi is a manufacturer. Starting from 2004, we are dedicated to produce quality medical and homecare disposables under ISO 13485 and GMP system. Jingdi is certified/registered with CE and FDA.

Major products include Wound Plaster, Medical Tape, Sterile Wound Dressing, Alcohol Pad, First Aid Kit, Cooling Gel Patch, Kinesiology Tape, Hydrocolloid Dressing as well as Medical and Protective Masks. Adhesive bandages, tapes and sterile wound dressings are well-known legacy items favored both domestically and internationally.

Our success is largely rooted in strong customer loyalty, competitiveness and strict quality consistency contributed by our staff of sufficient knowledge, integrity, diligence and frugality. Contact us today, you are warmly welcome to visit our workshops, laboratories and Research & Development center.


Stay with Jingdi, let's make better life with better care together.

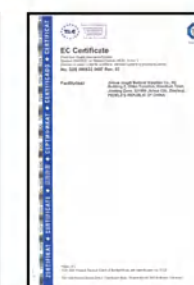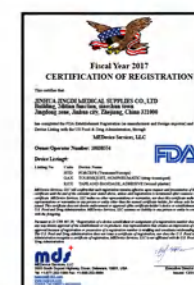

ADHESIVE BANDAGE



Fabric
Item No: AB-001
Size: 10x5cm
Package: 10pcsx100boxes

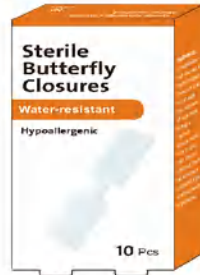

Waterproof
Item No: AB-002
Size: 45x9mm
Package: 10pcsx100boxes



Fabric
Item No: AB-003
Size: 76x38mm&50x45mm
Package: 20pcsx100boxes



Waterproof
Item No: AB-005
Size: 76x51mm
Package: 10pcsx100boxes



Waterproof
Item No: AB-006
Size: Dia22mm/Dia25mm
Package: 50pcsx100boxes



Waterproof
Item No: AB-007
Size: 76x25mm
Package: 20pcsx100boxes



Waterproof
Item No: AB-004
Size: 72x19mm
Package: 30pcsx100boxes



Waterproof
Item No: AB-008
Size: 56x19mm&76x19mm&76x25mm
Package: 40pcsx100boxes



First Aid Kit
Item No: AB-009
2 Antiseptic cleaning wipes
4 non-woven pad
2 Fabric Bandages 76x19mm
4 Fabric Bandages 56x19mm



Item No: WP-001
Material: PE/PVC/EVA/PU Transparent
Package: 20/50/100pcs/box 300/100/100boxes/carton



Item No: WP-003
Material: Fabric/Elastic Fabric/Cotton
Package: 20/50/100pcs/box 300/100/100boxes/carton



Item No: WP-005
Size: 4x10cm/6x10cm/8x10cm customizable
Material: Fabric
Package: 10pcs/box 100boxes/carton




Item No: WP-007
Size: Spot22mm/40X10mm/72X19mm/76X19mm customizable
Material: PE/PVC/EVA/PU
Package: 50/75/100pcs/box 50/100/200boxes/carton



Item No: WP-002
Material: Cartoon PE/PVC
Package: 100/200/300boxes/carton




Item No: WP-004
Size: 4x10cm/6x10cm/8x10cm customizable
Material: Plastic
Package: 10pcs/box 100boxes/carton

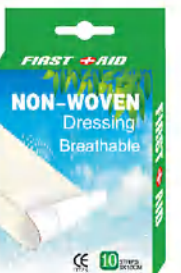

Item No: WP-006
Size: 4x10cm/6x10cm/8x10cm customizable
Material: Non-woven
Package: 10pcs/box 100boxes/carton




Item No: WP-008
Size: Spot22mm/40X10mm/72X19mm/76X19mm customizable
Material: Fabric/Elastic Fabric/Cotton
Package: 50/75/100pcs/box 50/100/200boxes/carton

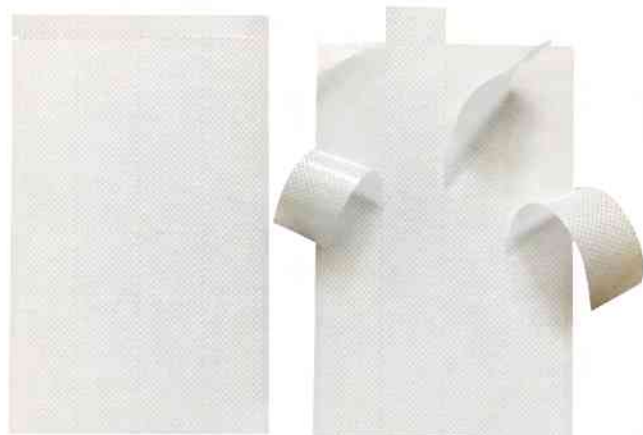

Item Name	Item No	Size	Item No	Size	Item No	Size	Item No	Size	Item No	Size	Item No	Size	
BANDAGES	B-001	72*19mm	B-010	38*38mm	B-019	62*25mm (bottle shape)	B-028	75*22mm	B-037	38*38mm (with a hole)	B-046	100*60mm	
	B-002	70*18mm	B-011	56*19mm (square angle)	B-020	76*51mm	B-029	65*25mm	B-038	40*10mm	B-047	100*50mm	
	B-003	70*17mm	B-012	56*19mm (round angle)	B-021	65*40mm	B-030	57*25mm	B-039	46*10mm	B-048	100*40mm	
	B-004	76*25mm	B-013	76*19mm	B-022	50*45mm	B-031	72*30mm	B-040	43*10mm	B-049	5x5cm	
	B-005	72*25mm	B-014	72*22mm (square angle)	B-023	74*24mm	B-032	40*20mm	B-041	60*50mm			
	B-006	72*22mm	B-015	72*20mm (square angle)	B-024	72*50mm	B-033	89*27mm	B-042	62*35mm			
	B-007	72*23mm	B-016	72*19mm (square angle)	B-025	72*38mm (square angle)	B-034	98*38mm	B-043	55*56mm			
	B-008	dia22mm	B-017	76*38mm (round angle)	B-026	72*38mm (round angle)	B-035	97*82mm	B-044	56*66mm			
	B-009	dia25mm	B-018	76*45mm	B-027	75*25mm	B-036	66*18mm	B-045	octangle			


STERILE WOUND DRESSING(Non-woven/PU)

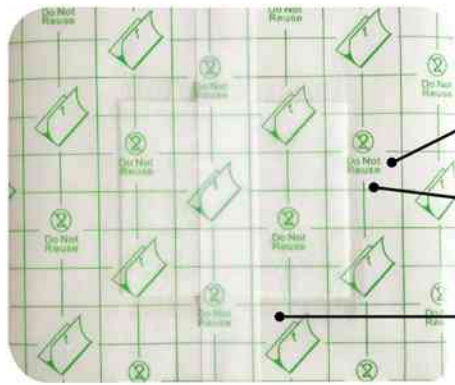

PE frame style decreases extra tension on skin, prevent the skin hurt.

Extra-thin PU film is comfortable to the skin.

Special welt design could make the back lining easier to peel off.

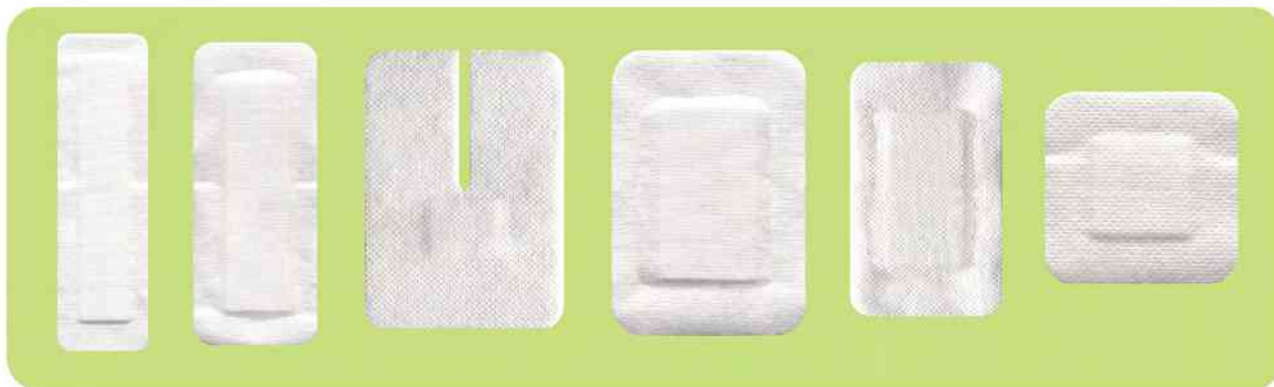



STERILE WOUND DRESSING(Non-woven/PU)

Highly breathable microporous material

Highly absorbent cushion

Elastic non-woven material

Medical hypoallergenic adhesive

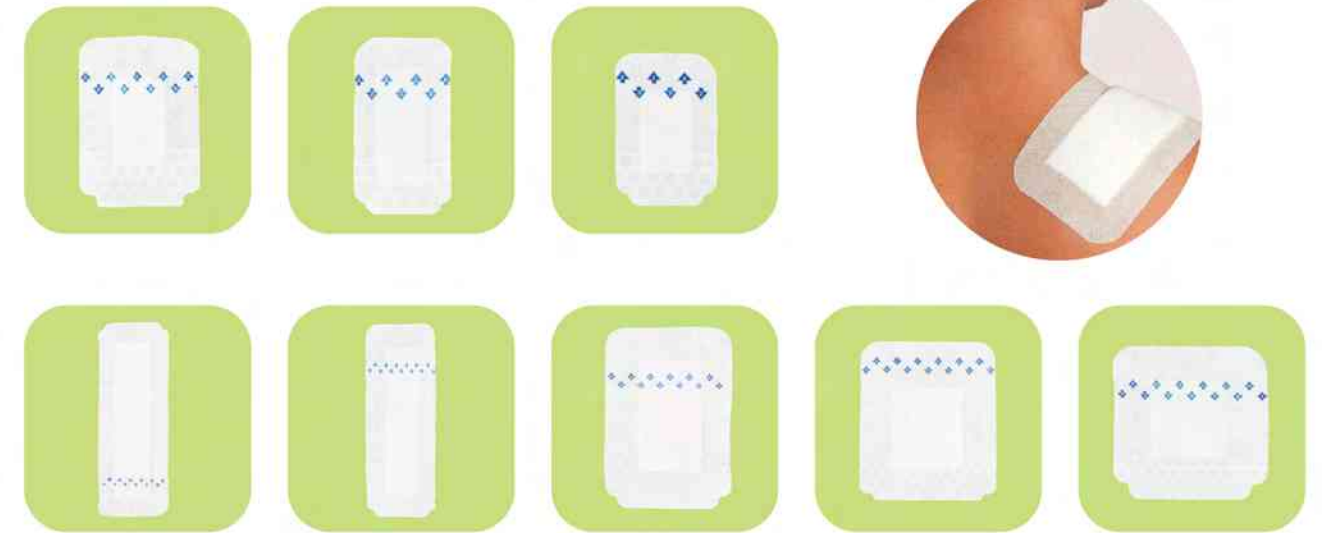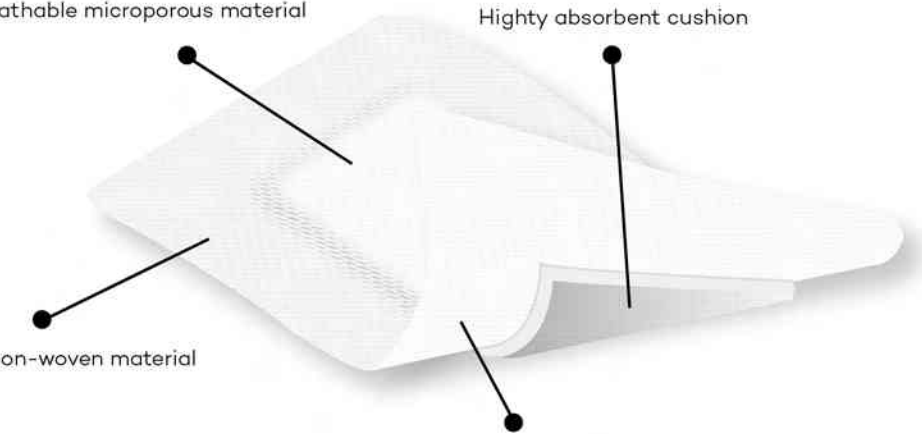

Item Name	Item No	Size	Package	Item No	Size	Package	Item No	Size	Package
Wound Dressing	D-001	5cmx5cm	50pcs*24boxes/carton	D-011	6cmx10cm	50pcs*24boxes/carton	D-021	9cmx35cm	50pcs*12boxes/carton
	D-002	5cmx7cm	50pcs*24boxes/carton	D-012	7cmx8cm	50pcs*24boxes/carton	D-022	10cmx10cm	50pcs*24boxes/carton
	D-003	5cmx8cm	50pcs*24boxes/carton	D-013	7cmx10cm	50pcs*24boxes/carton	D-023	10cmx12cm	50pcs*24boxes/carton
	D-004	5cmx8.3cm	50pcs*24boxes/carton	D-014	8cmx10cm	50pcs*24boxes/carton	D-024	10cmx15cm	50pcs*24boxes/carton
	D-005	5cmx9cm	50pcs*24boxes/carton	D-015	8cmx15cm	50pcs*24boxes/carton	D-025	10cmx20cm	50pcs*24boxes/carton
	D-006	5cmx10cm	50pcs*24boxes/carton	D-016	9cmx10cm	50pcs*24boxes/carton	D-026	10cmx23cm	50pcs*18boxes/carton
	D-007	5cmx15cm	50pcs*24boxes/carton	D-017	9cmx15cm	50pcs*24boxes/carton	D-027	10cmx25cm	50pcs*18boxes/carton
	D-008	6cmx7cm	50pcs*24boxes/carton	D-018	9cmx20cm	50pcs*24boxes/carton	D-028	10cmx30cm	50pcs*12boxes/carton
	D-009	6cmx8cm	50pcs*24boxes/carton	D-019	9cmx25cm	50pcs*18boxes/carton	D-029	10cmx35cm	50pcs*12boxes/carton
	D-010	6cmx9cm	50pcs*24boxes/carton	D-020	9cmx30 cm	50pcs*12boxes/carton	D-030	15cmx15cm	50pcs*24boxes/carton



Better Care, Better Life



For protection and absorption

Non-adherent Dressing Pad

Ideal for covering & protecting wound

Non-adherent dressing pad has an absorbent layer to help the wound keeping clean and dry, the ventilated outer film allows fluid to pass into the absorbent center while providing a non-stick contact layer against the wound. Can cut to any sizes.

Non-adherent


Ventilated film allows the wound to breathe and provides an ouchless removal

Cushions & preprotects

Soft and flexible material covers wound and minimizes disruption

Absorbent

Cotton/polyester core pulls excess fluid away from the wound

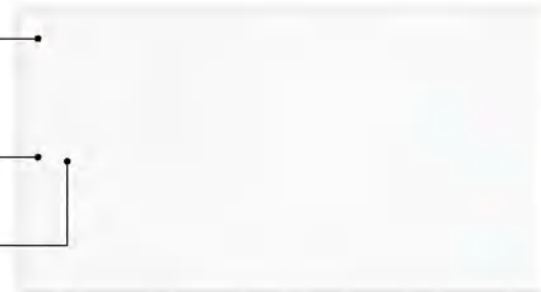

Corn Plaster


Gentle and natural corn removal
Moisturises and softens skin providing pain relief

Active ingredients

Salicylic acid, polyvinyl alkyl, ether adhesive, titanium dioxide, liquid paraffin, antioxidant, red iron oxide and black iron oxide.

Possible side effects

Irritation may occur. If it is severe stop use.

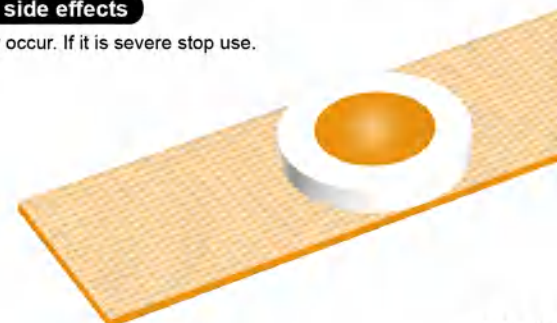

Instructions for use:

- 1.Ensure the skin is clean and dry.
- 2.Place firmly the medicated round part over the center of corn, adhesive next to skin.
- 3.Repeat treatment daily until corn can be removed.
- 4.Do not use for more than two weeks, unless your doctor tells you to.



Better Care, Better Life



I.V. and Catheter Fixation Dressing

It can be used to protect I.V. catheter sites and wounds, to secure devices to the skin and to maintain a moist environment for wound healing.

- Provides a waterproof, sterile barrier to external contaminants including liquids, bacteria and viruses.
- The dressing provides continuous visibility to the I.V. insertion site.
- Adhesive is friendly on skin.
- Dressing flexes with skin for greater patient comfort.
- High breathable, the skin can breath freely.

Photo	Size	Application advice	Notes	Photo	Size	Application advice	Notes
	8.5x11.5cm	CVC/PICC External Jugular Vein	Border design improve the adhesion, reduce edge-lift Dot coated border improve the Breathability Reduce the incidence of phlebitis caused by i.v. catheters		10x12cm 9x3.5cm	Both secure the i.v. injection site and catheter	The fixation of the catheter reduces the medical risks, such as the retention needle falling off due to accidental pull-out.
	6.5x7cm	Arterial catheter, PICC, CVC, epidural catheter, PIV (peripheral intravenous catheter PICC, CVC, PIV)			6x7cm		
	3.5x4.5cm	Peripheral lines - Paediatric			9x3.5cm		
	5x7.7cm	Peripheral lines - Paediatric	Notes: without paper frame		6x7cm	Suitable for wing type or with pot retention needle fixation.	U notch designed basic type S peeling design on paper frame makes it easy to peel off the backing paper
	7x9cm	Arterial catheter, PICC, CVC, epidural catheter, PIV (peripheral intravenous catheter PICC, CVC, PIV)			7x9cm		
	8.5x10.5cm	Peripheral and midline venous catheters, Central venous catheters (including subclavian, jugular, femoral and PICCs), Implanted infusion devices (ports), Dialysis catheters, Pulmonary artery catheters, Arterial catheters	Non-woven reinforced design improve the adhesion Dot coated border improve the Breathability Reduce the incidence of phlebitis caused by i.v. catheters		4.4x6cm	For Pediatric	Basic paper frame type S peeling design on paper frame makes it easy to peel off the backing paper
	7x8.5cm	Peripheral lines - Paediatric			6x7cm	PIV(peripheral intravenous catheter)	
	10x12cm				10x12cm	CVC/CVP cervical veins, epidural catheters, central veins	
	5x5.7cm	Peripheral lines - Paediatric	Notes: without paper frame		6x8cm	Suitable for wing type or with pot retention needle fixation.	U notch designed without frame Absorption pads keep the retention area clean and tidy
	7x9.5cm	For use on peripheral and central venous catheter fixation			6x8cm	Suitable for wing type or with pot retention needle fixation.	U notch designed Nonwoven type not waterproof Absorption pads keep the retention area clean and tidy



MEDICAL ADHESIVE TAPE



Non-Woven Tape(Skin/White Non-Woven)

Item No:	Size	Package	Carton size	MOQ
MA-001	1.25cm x 5m	24rolls/box,30boxes/ctn	25x24.5x28cm	1500Boxes/ Each Size
MA-002	2.5cm x 5m	12rolls/box,30boxes/ctn		
MA-003	5cm x 5m	6rolls/box,30boxes/ctn	40.5x24.5x25cm	
MA-004	7.5cm x 5m	6rolls/box,30boxes/ctn		
MA-005	10cm x 5m	6rolls/box,30boxes/ctn	40.5x36x22.5cm	



Transparent Pe Tape(Transparent)

Item No:	Size	Package	Carton size	MOQ
MA-006	1.25cm x 5m	24rolls/box,30boxes/ctn	27x26.5x28cm	1500Boxes/ Each Size
MA-007	2.5cm x 5m	12rolls/box,30boxes/ctn		
MA-008	5cm x 5m	6rolls/box,30boxes/ctn	27x26.5x40.5cm	
MA-009	7.5cm x 5m	6rolls/box,30boxes/ctn		
MA-010	10cm x 5m	6rolls/box,30boxes/ctn	44x26.5x32.5cm	



Silk Tape (Silk)

Item No:	Size	Package	Carton size	MOQ
MA-010	1.25cm x 5m	24rolls/box,30boxes/ctn	27x26.5x28cm	1500Boxes/ Each Size
MA-011	2.5cm x 5m	12rolls/box,30boxes/ctn		
MA-012	5cm x 5m	6rolls/box,30boxes/ctn	27x27x40.5cm	
MA-013	7.5cm x 5m	6rolls/box,30boxes/ctn		
MA-014	10cm x 5m	6rolls/box,30boxes/ctn	27.5x27x53cm	

MEDICAL ADHESIVE TAPE



Zinc Oxide Adhesive Tape(Skin/White Cotton)

Item No:	Size	Package	Carton size	MOQ
MA-021	1/2" x 4 rolls	3" x 1 rolls	1500Boxes/Each Size	MOQ
MA-022	1" x 3 rolls	12" x 10yds		
MA-023	2" x 2 rolls			



Zinc Oxide Adhesive Tape(Skin/White Cotton)

Item No:	Size	Package	Carton size	MOQ
MA-015	1.25cm x 5m	24rolls/box,30boxes/ctn	32x32x30cm	1500Boxes/ Each Size
MA-016	2.5cm x 5m	12rolls/box,30boxes/ctn		
MA-017	5cm x 5m	6rolls/box,30boxes/ctn	53x32x25cm	
MA-018	7.5cm x 5m	6rolls/box,30boxes/ctn		
MA-020	10cm x 5m	6rolls/box,30boxes/ctn	53x32x33cm	



Zinc Oxide Plaster with plastic cover(Skin/White Cotton)

Item No:	Size	Package	Carton size	MOQ
MA-032	1.25cm x 5m	24rolls/box,30boxes/ctn	36.5x36x42.5cm	1500Boxes/ Each Size
MA-033	2.5cm x 5m	12rolls/box,30boxes/ctn	36.5x36x34.5cm	
MA-034	5cm x 5m	6rolls/box,30boxes/ctn	36.5x36x31cm	
MA-035	7.5cm x 5m	6rolls/box,30boxes/ctn	36.5x36x44.5cm	
MA-036	10cm x 5m	6rolls/box,30boxes/ctn	59.5x36x34.5cm	

MEDICAL ADHESIVE TAPE



Silk / Zinc Oxide Perforated Plaster				
Item No:	Size	Package	Carton size	MOQ
MA-024	10cm x 5m	1roll/box,80boxes/ctn	45.5x29x25cm	1500Boxes/ Each Size
MA-025	10cm x 5m	10rolls/box,8boxes/ctn	36.5x36.5x38.5cm	
MA-026	18cm x 5m	1roll/box,80boxes/ctn	51.5x41.5x21.5cm	
MA-027	18cm x 5m	10rolls/box,10boxes/ctn	53x51.5x22cm	



Zinc Oxide Tinplate Tape(Skin/White Cotton)				
Item No:	Size	Package	Carton size	MOQ
MA-028	1.25cm x 5m	12rolls/box,60boxes/ctn	54x48x27cm	1500Boxes/ Each Size
MA-029	2.5cm x 5m	12Rolls/box,30boxes/ctn	36.5x36.5x38.5cm	
MA-030	5cm x 5m	6Rolls/box,30boxes/ctn	36.5x36.5x33.5cm	
MA-031	7.5cm x 5m	6Rolls/box,20boxes/ctn	36.5x24.5x45.5cm	



PU/Non-woven fixing Tape			
Item No:	Normal Size	Package	Carton size
MA-032	5cm x 10m	1roll/box,160boxes/ctn	61.5x39x23cm
MA-033	10cm x 10m	1roll/box,80boxes/ctn	39x31.5x43cm
MA-034	15cm x 10m	1roll/box,60boxes/ctn	39x31.5x47.5cm
MA-035	20cm x 10m	1roll/box,40boxes/ctn	39x31.5x42.5cm
MA-036	20cm x 10m	1roll/box,40boxes/ctn	40.5x32.5x42.5cm
MA-037	30cm x 10m	1roll/box,25boxes/ctn	40.5x40.5x42.5cm

COHESIVE BANDAGE



Description:

Hand-tear non-woven cohesive bandage
 Length and width can be made as per request
 One color printing on fabric can be customized according to customer's idea
 Print your logo on the fabric to present your company to all users impressively

Benefits:

Print lovely pattern on the bandage to relax users
 Quality cohesive bandage performance
 Also for Pet Hospital/clinic

Application:

Securing pads and dressings, protective bandage
 Pressure bandage to promote circulation and healing
 Sprains and strains
 partial immobilization
 Post-operatively leg wrap
 Caution: This product contains natural latex which may cause allergic reactions



Non-Woven/Cotton Cohesive Bandage(printing)			
Item No	Size	Package	MOQ
S-001	2.5cm x 4.5m	24Rolls/box,24boxes/ctn	1500Boxes/Each Size
S-002	5cm x 4.5m	12Rolls/box,24boxes/ctn	
S-003	7.5cm x 4.5m	12Roll/box,16boxes/ctn	
S-004	10cm x 4.5m	12Roll/box,8boxes/ctn	
S-005	15cm x 4.5m	12Roll/box,8boxes/ctn	

SPORTS TAPE/BANDAGE



Kinesiology Tape(95% Cotton+5%Spandex)				
Item No	Size	Package	Carton size	MOQ
S-006	2.5cm x 5m	12rolls/box,20boxes/ctn	39x30.5x35cm	5Cartons/ Eac Size
S-007	3.8cm x 5m	8rolls/box,20boxes/ctn	39x30.5x35cm	
S-008	5cm x 5m	6rolls/box,20boxes/ctn	39x30.5x35cm	
S-009	7.5cm x 5m	4rolls/box,20boxes/ctn	39x30.5x35cm	
S-010	10cm x 5m	12rolls/box 6boxes/ctn	48 5x33 5x36cm	



Sport Tape(Cotton)			
Item No	Size	Package	MOQ
S-011	2.5cm x 13.7m	2.5cm: 48Rolls/box,6boxes/ctn	1500Boxes/ Each Size
S-012	3.8cm x 13.7m	3.8cm: 32Rolls/box,6boxes/ctn	
S-013	5cm x 13.7m	5.0cm: 24Rolls/box,6boxes/ctn	
S-014	5cm x 13.7m	7.5cm: 16Rolls/box,6boxes/ctn	



EAB Tape(Cotton)			
Item No	Size	Package	Carton Size
S-015	5cm x 4.5m	144rolls	46x31x36cm
S-016	7.5cm x 4.5m	96rolls	46x31x36cm
S-017	10cm x 4.5m	96rolls	46x31x46cm
S-018	15cm x 4.5m	48rolls	46x31x36cm
S-019	5cm x 3m	144rolls	38.5x25.5x36cm
S-020	7.5cm x 3m	96rolls	38.5x25.5x36cm
S-021	10cm x 3m	96rolls	38.5x25.5x46cm
S-022	15cm x 3m	48rolls	38.5x25.5x36cm



ELASTIC BANDAGE(80%cotton+20%spandex)			
Item No	Size	Package	Carton Size
S-023	5cm x 4.5m	720rolls	52x33x43cm
S-024	7.5cm x 4.5m	480rolls	52x33x43cm
S-025	10cm x 4.5m	360rolls	52x33x43cm
S-026	15cm x 4.5m	210rolls	52x33x43cm



PBT BANDAGE(45%viscose+55%polyester with woven edges)			
Item No	Size	Package	Carton Size
S-027	5cm x 4.5m	720rolls	43x33x36cm
S-028	7.5cm x 4.5m	480rolls	43x33x36cm
S-029	10cm x 4.5m	360rolls	43x33x36cm
S-030	15cm x 4.5m	210rolls	43x33x36cm



CREPE BANDAGE(80%cotton+20%spandex)			
Item No	Size	Package	Carton Size
S-031	5cm x 4.5m	720rolls	57x33x48cm
S-032	7.5cm x 4.5m	480rolls	57x33x48cm
S-033	10cm x 4.5m	360rolls	57x33x48cm
S-034	15cm x 4.5m	210rolls	57x33x48cm



HIGH ELASTIC BANDAGE(85%polyester+15%rubber)			
Item No	Size	Package	Carton Size
S-035	5cm x 4.5m	720rolls	52x32x43cm
S-036	7.5cm x 4.5m	480rolls	52x32x43cm
S-037	10cm x 4.5m	360rolls	52x32x43cm
S-038	15cm x 4.5m	210rolls	52x32x43cm

SPORTS TAPE/BANDAGE

COOLING GEL PATCH

COOLING GEL PACTH

Item No: C-001
 Size: 12x5cm/11x4cm/6x4cm
 Material: Non-woven+Gel+Film
 Package: 1patchx3bagsx240boxes/ctn
 MOQ: 30000bags



1 Remove the cold paste and uncover the transparent film



2 Stick it on your forehead or arms where it needs to cool down

COOLING GEL PACTH(Color change)



the gel color will be purple if the temperature is blow 38°C



the color will turn pink if the temperature higher than 38°C



ALCOHOL PAD



Alcohol Pad(Non-Woven Fabric+Alcohol)

Item No	Size	Package	MOQ
A-001	6cm x 3cm	100pcsx100boxes/carton	500000PCS
A-002	6.5cm x 3cm	200pcsx50boxes/carton	
A-003	6cm x 6cm	100pcsx100boxes/carton	
A-004	9cm x 8cm	50pcsx100boxes/carton	
A-005	10cm x 10cm	30pcsx100boxes/carton	



Povidone-Iodine Prep Pad(Non-Woven Fabric+Povidone-Iodine prep)

Item No	Size	Package	MOQ
A-007	6cm x 3cm	100pcsx100boxes/carton	500000PCS
A-008	6.5cm x 3cm		



Wound



Tableware



Glasses



Mobile phone



Better Care, Better Life

POST-COLLECTION DRESSINGS

Round Adhesive Band-aid Packed With Dispenser Box



Application: Recommended for hemostasis and protection of the epidermis of wounds and punctures with needles. Indicated for medical or home use.

Composition: Made of polystyrene, absorbent pad, glassine paper and AC glue.

Non-sterile.

Size: Ø 25 mm

Storage: Store in a cool, dry and low light place.

Disposal: Dispose of the product in an appropriate location for potentially contaminated materials.

Instructions for use:

1. Check the integrity of the packaging and shelf life.
2. Remove the adhesive and place the absorbent region in contact with the wound or puncture.

Precautions:

1. Do not touch the absorbent region of the dressing.
2. Clean and disinfect the wound before using the dressing.
3. Do not reuse the product.
4. Use on minors must be accompanied by the supervision of an adult.
5. Do not use if it is out of expiry date.
6. If the dressing is wet, perform replacement with another dressing.
7. To maintain hygiene, change the dressing if it's necessary.



Skin disinfection



Better Care, Better Life

Wound Skin Closure Strips

For closing small wound.

Used in hospitals to help improving cosmetic results.

Application:

The wound skin closure strips can inhibit the tension on both side of skin, It can control the expansion of the wound in time when use in the early stage of wound healing.



Instruction for use:

1. Clean the wound with saline water and dry the wound area.
2. Stick the wound skin closure to the wound vertically from the middle of the wound.
3. Apply the wound skin closure from two ends of the wound to the middle part to avoid wound opening.
4. Stick to the rest parts of wound.
5. The medical band aid can be left in place for several days.



Wound Skin Closure (Elastic Bandage)

First aid dressing for closing small wounds

Application:

It's used for skin surface suture and pull, wound tension, effectively reducing the lateral tension on both sides of the wound, preventing scar widening at the suture and reducing the probability of scar formation.




Instruction for use:

1. Clean the wound and surroundings.
2. Take out the dressing and position it over the central of the wound.
3. Press down the edges of dressing firmly and remove the release paper.
4. Pull the dressing from two ends and disinfect the wound after it is closed.
5. Secure the edges of dressing with adhesive tape if long time stay on the skin.
6. Use an extra sterile dressing to cover if the wound is large.



CAPSICUM PLASTER

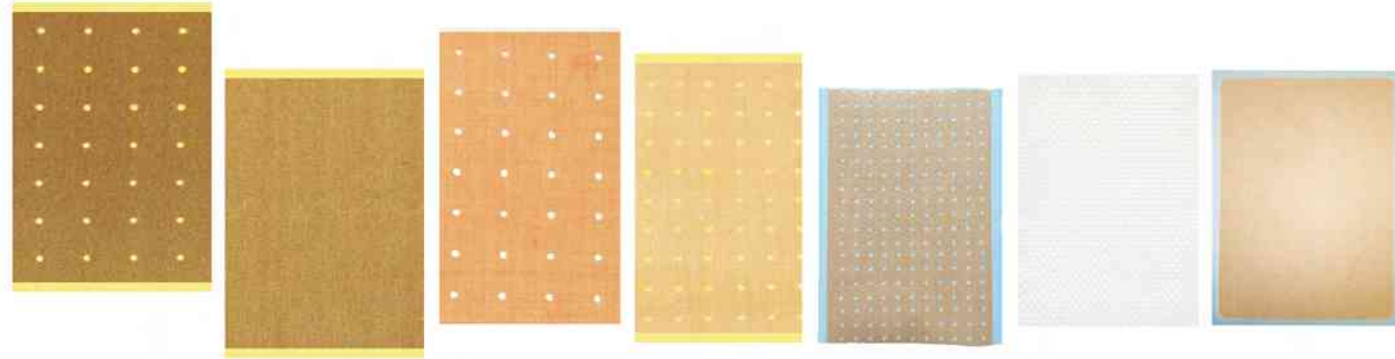

Capsicum Plaster				
CP-001	6x10cm	50bags/box	12boxes/CTN	CTN Size:32x21.5x17cm
CP-002	7x10cm	50bags/box	12boxes/CTN	CTN Size:32x21.5x17cm
CP-003	10x18cm	50bags/box	12boxes/CTN	CTN Size:32x30x25cm
CP-004	12x18cm	24bags/box	50boxes/CTN	CTN Size:57x36x25cm



HYDROCOLLOID DRESSING



Application

- Low or moderate exuded.
- Crura venous ulcer.
- I-II degree.
- Small superficial burn wound.
- Post operation.
- Donated skin area.
- Superficial wound and beauty operation wound.
- Period of granulation and epithelialisation of chronic wound.

Characteristics

- Good absorbency for exudation from the wound.
- Keep the wound moist and accelerate the wound healing; reduce the pain and avoid the secondary damage to the wound.
- Waterproof, permeable and prevent the wound from bacteria outside.
- Excellent alesticity and self-adhesive, easy for application.
- Change the dressing when it turns white.
- Easy for peeling off and it will not stick to the wound and keep patients comfortable.



Better Care, Better Life



Silicone Dressing


Silicone dressing consists of a gentle silicone wound contact layer, a super absorbent pad layer, and a vapor permeable and waterproof film, is an ideal dressing for advanced wound exudate management. The multi-layers construction facilitates dynamic fluid management to provide optimal moist wound environment, which promotes wound healing, minimizes trauma and reduces pain during dressing changes.

Benefits

- Conformable dressing - provide super gentle silicone contact layer for fragile skin.
- Minimize trauma and pain during dressing change.
- Easy to apply and remove - the dressing can be repositioned easily without losing its adherent properties.
- Shower proof dressing allowing patients to shower with the dressing in place.
- Customized shape design to adapt different requirements. Maintains its integrity during use

Application:

- Pressure ulcer • Leg ulcer • Non-infected diabetic foot ulcer
- Donor sites • Skin abrasions • Post-operative wound dressing
- Lacerations and abrasion

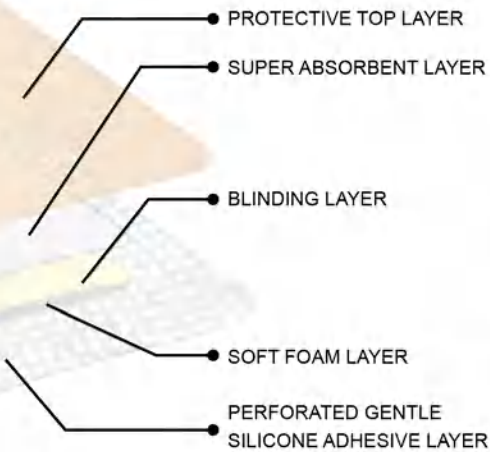

Foam Dressing


Foam dressing is a highly absorbent dressing made from hydrophilic polyurethane foam, ideally designed for heavily exuding wounds. The soft flexible foam absorbs large amount drainage, ensures patients comfort; the top backing film protect wounds against external bacterial.

Application:

- Foam dressing is indicated for the moderate to heavily exuding wounds
- Pressure ulcer
- Leg venous ulcer
- Diabetic foot ulcer
- First and second degree burns
- Skin grafts and donor site
- Post-operative and traumatic wounds



Better Care, Better Life

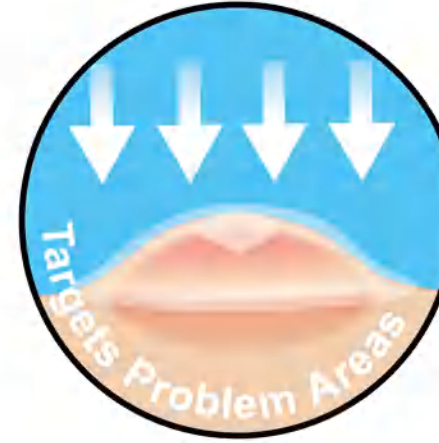

Hydrocolloid Acne Patch


Cover blemishes invisibly

Ultra-thin invisible
Suppress acne
Easy makeup
Waterproof



Instructions for use:

1. Cleanse the problem area and dry.
2. Remove desired size patch from film, apply over blemish. Try not to touch the adhesive side with your fingers.
3. Leave patch on for 6-8 hours for best results. Patch will turn to white.
4. Remove and discard used patch; replace with new patch, if needed.

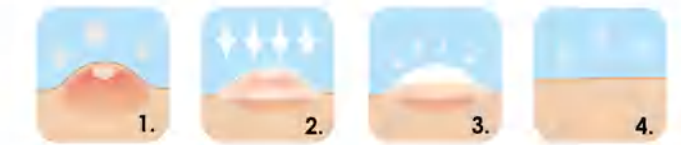

Hydrocolloid Blister Plaster

Blisters are fluid filled lesions that develop in response to repeated friction on the skin. What blisters can do is to:

Prevent: protecting feet from rubbing.

Relieve: providing instant pain relief.

Protect: protecting the blister from further friction.

Fast healing: offer closed fast healing environment.



Cushions and protects



Instructions for use:

1. Clean and dry the skin.
2. Peel off the release paper.
3. Apply the plaster on the skin.
4. Press the edges of plaster firmly.





Item No: F-001

List

1.First Aid Bandage 10cmx4m	3pcs
2.First Aid Bandage 8cmx4m	1pc
3.PBT Bandage 6cmx4m	2pc
4.PBT Bandage 8cmx4m	3pc
5. Triangular Bandage 96x96x136cm	1pc
6.Wound Dressing 10x10cm	1pc
7.First Aid Dressing 6x10cm, Non-woven, 8pcs/pack	1pc
8.PVC Gloves, 4pcs/pack	1pc
9.Non-woven Adhesive Tape 2.5cmx5m	2pcs
10.Emergency Blanket 160x210cm	2pcs
11.List Contents	1pc
12.Burn Dressing 60x80cm	4pcs
13.Burn Dressing 40x60cm	1pc
14.Scissor	1pc
15.First Aid box 260x170x80mm	1pc



Item No: F-002



Item No: F-003



Item No: F-004



Item No: F-005



Item No: F-006



Item No: F-007



Item No: F-008



Item No: F-009


DISPOSABLE FACE MASK

3 LAYER MASK

Size: 17.5x9.5cm/14.5x9.5cm/12.5x9.5cm


Material: Non-woven, melt blown, non-woven fabric

- Easy to handle adaptable nose-clip
- Filtering efficiency (BFE) $\geq 95\%$ according to the EN14683
- Easy to fix with an elastic band
- Does not contain glass-fibres
- Latex-free
- Non sterile

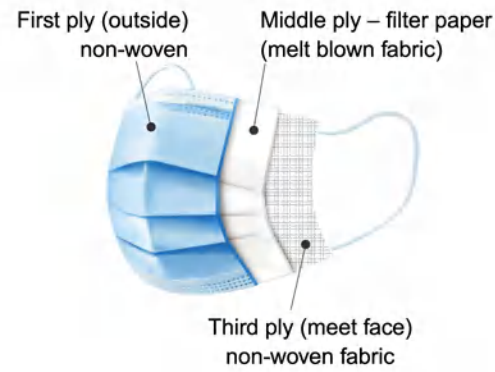

PROTECTIVE MASK

FFP2 / KN95

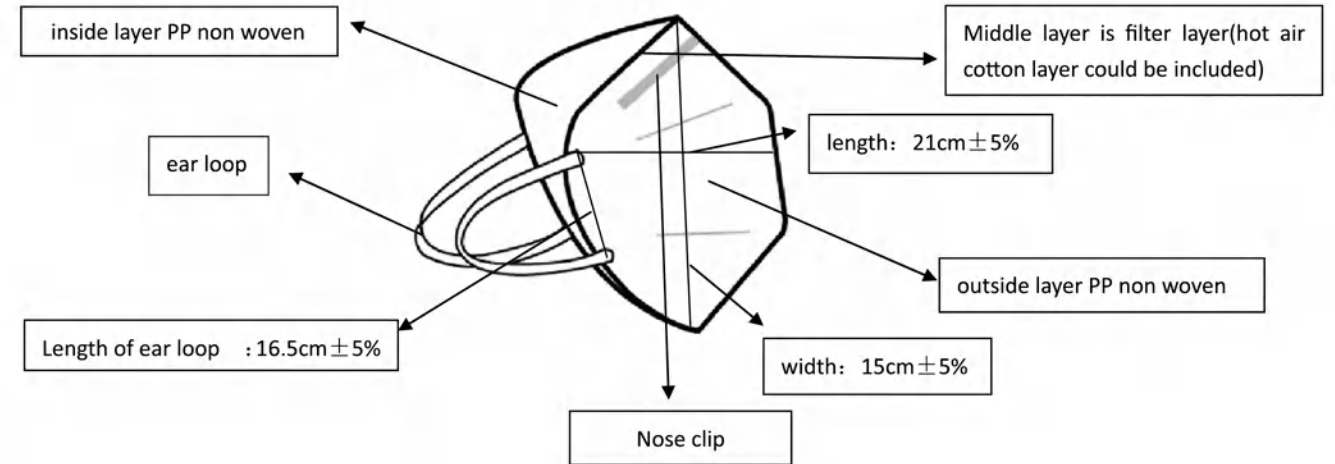

JUMBO ROLL OF RAW MATERIALS



Elastic Fabric

Item No: J-001



Cotton Fabric

Item No: J-002



Transparent PE

Item No: J-003



Release paper

Item No: J-004



Absorbent Pad

Item No: J-005



Wrapping Paper

Item No: J-006



Blue PE

Item No: J-007



Non-woven

Item No: J-008



**Non-woven/PE
Small Roll**

Item No: J-009

PLASTIC PRODUCTS



Item No: P-001



Item No: P-002



Item No: P-003



Item No: P-004

Plastic Transparent Cover Item No: P-005



Tweezer Item No: P-006





Better Care, Better Life

MACHINE

AUTOMATIC PACKAGE MACHINE FOR WOUND PLASTER

MAX.size of masterial roll: 380mm
MAX.width size of the material tape: 85mm
MAX.size of the plaster: 78mm
Plaster Orientation: NO
MAX.size of the wrapper paper: 95mm
Produce Speed: 0-720pcs/minute.
Wrapping Paper Orientation: NO
Machine Power: 3.5KW
Power Voltage: 380V
Machine size: 2800*750*1900mm
Machine Weight: 1500kgs



Better Care, Better Life

MACHINE

REWINDING MACHINE FOR TAPE ROLL

Rewinding width: 500-1100mm
Maximum diameter: 150mm
Rewinding speed: 150m/min
Tube core diameter: 25-50mm
Electric tension: 380V
Power of motor: 2.75KW
The using of air pressure: 2-5pa



ADHESIVE COATING MACHINE

Stylle: THB-560/THB-1100
The width of the coated: 560mm/1100mm
Amount of glue: 20-200g
Coating speed: 0-60m
Unwinding diameter: 1000mm
Rolling diameter: 80mm
The feeding device: 3"air expansion core
Correction device: gold medal optional
Installed power: 18KW/28KW
Electric source: 380V
The length of the whole machine: 4500mm-6800mm
The width of the whole machine: 1500mm/2000mm
The height of the whole machine: 1500mm-1900mm
Controlled temperature: 60°C-230°C



SLITTING MACHINE FOR TAPE ROLL

Effective width: 1000mm
Cutting width: 5-1000mm
Tube core diameter: 25-50mm
Maximum cutting diameter: 100mm
Electric tension: 380V
The power of the whole machine: 2KW




Blister Prevention Tape

Item No: O-001



Wound Skin Closure

Item No: O-002

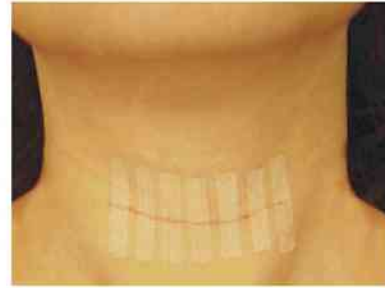

Silicon Plaster

Item No: O-003



Nasal Plaster

Item No: O-004



Corn Plaster

Item No: O-005



Silicone Foam Dressing

Item No: O-006

