


### 4.5.2 Pre-Defined Text Menu

You can set and manage text used for markers from the **Macro Exposure** mode and the **Viewer** mode.

- Add or delete text using **Add / Delete** buttons.
- You can adjust the order of **Pre-Defined Text** indicated on the **Label** tab by using **Up / Down** buttons.

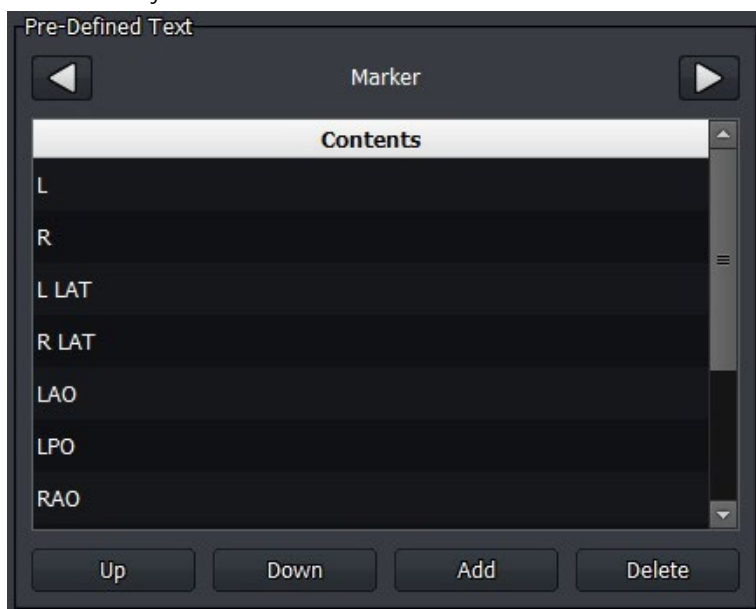

Marker	Position	Projection	Measure	Time	Others
					PORTABLE
L	ERECT	AP	0°- 90°	15/30 SEC	CROSS-TABLE
R	SUPINE	PA	(Every 5° interval)	1/2/3/5 MIN	WT BEARING
L LAT	PRONE	LAT		10/15/20 MIN	STRESS
R LAT	R LAT	OBL		30/45/60 MIN	DISTAL
LAO	DECUBITUS	AXIAL		2/3/4/5 HOUR	PROXIMAL
RAO	FLEXION	TANGENTIAL		12/24 HOUR	INTERNAL
L DECUB	EXTENSION				EXTERNAL
R DECUB	SITTING				CRANIAL
	LATERAL				INSPIRATION
	OBLIQUE				EXPIRATION
					CAUDAL
					SCOUT
					POST CONTRAST
					POST VOIDING




- The groups cannot be added, deleted or edited.

### 4.5.3 Image Tool Menu

You can assign icon buttons to be shown in the **Tool** panel in the **Viewer** mode and the macro exposure mode as basic tools.

- The color of selected icon becomes blue and will appear as a default tool.
- The unselected icons are provided as popup buttons.

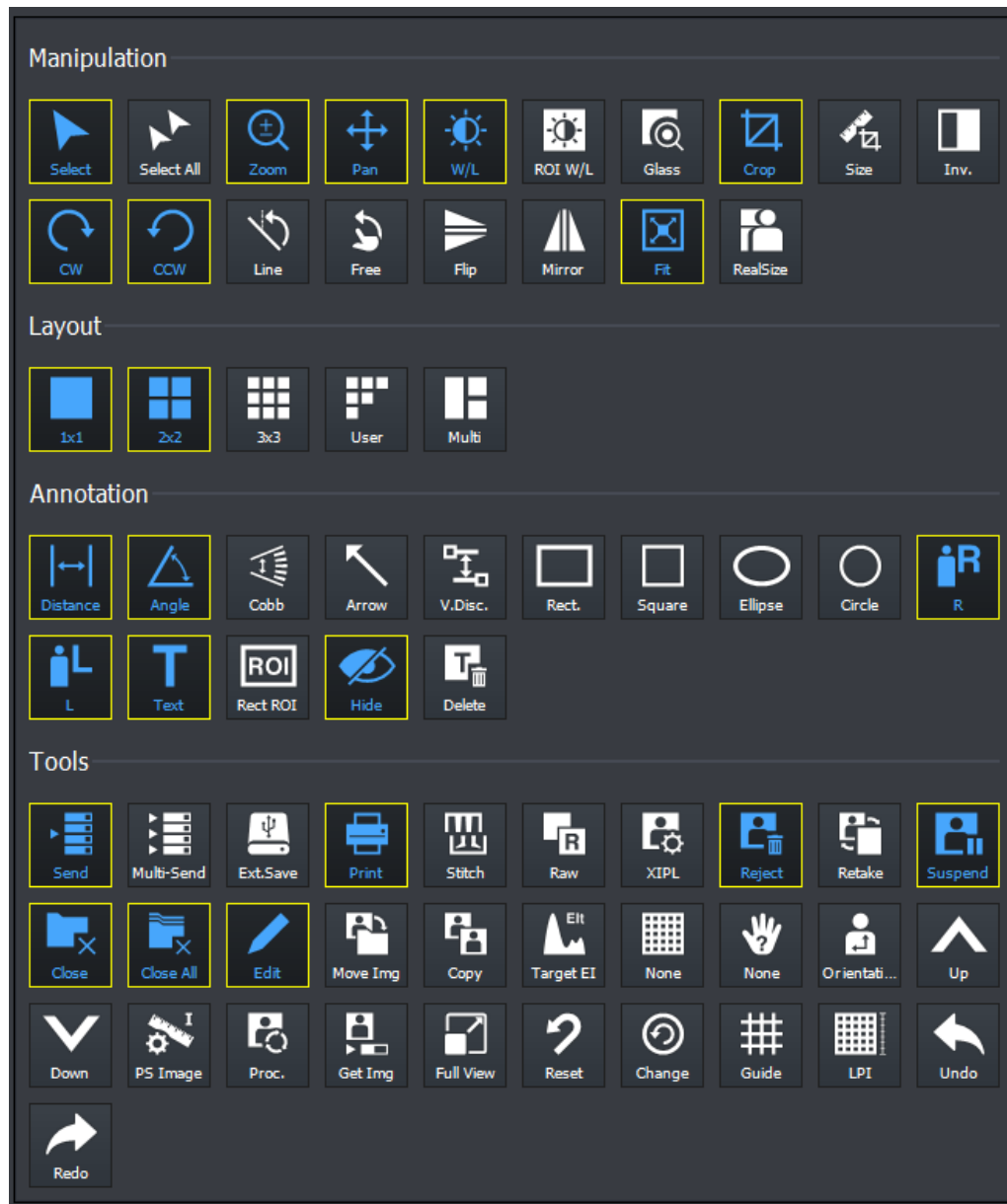

- The icon size is changed depending on the number of icons you chose.