


- (1) Independent probe connecting unit (optional)
- (2) USB (3.0) connector
- One of the following: RVS control unit (optional), Security box (optional), or storage drawer
- (4) Physiological signal unit (optional)
- Dummy connectors^{*1}
- (6) Foot switch connector
 - Alphanumeric keyboard (optional)
 -) ECG clip bar
 - Probe connector
 Connectors 1 to 4 (from top to bottom)

*1.

This is a connector for temporarily fastening a probe. No image is displayed, even if a probe is connected to this connector.

3. Operation panel

NOTE: The push button-type integrated rotary encoders have the key name on top and the rotary encoder names on the bottom. Information on how to use the rotary encoders is provided at the end of this section.

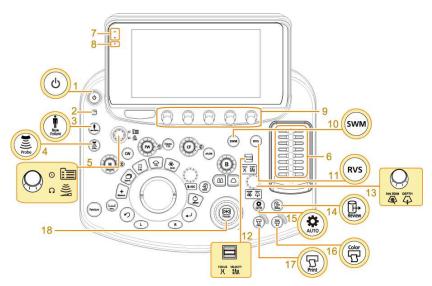

Operation panel diagram