


3.2.2. Rear panel of the device - description of outlets

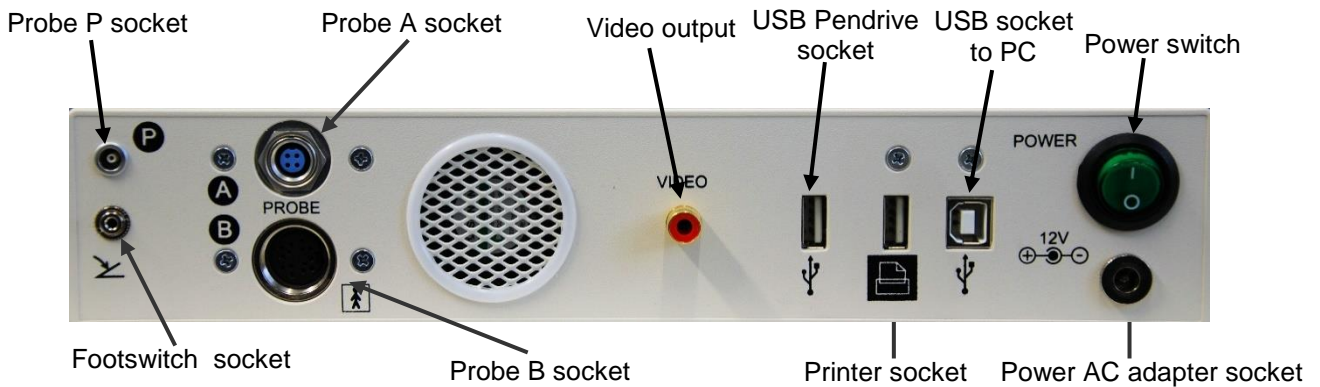

3.2.3. Probe connections

A probe

The biometry OA-probe socket is located on the back panel. Before inserting the connector rotate it slightly to find the good position and then inserting the connector and screw on the knurled ring of the plug clockwise to secure.

Specific probe holder for the biometry probe with immersion tube holder and special element for testing, are on the right side of the front panel. Turn the holder to the vertical until reach the fixing point (latch lock). Check the stability of the probe holder

B probe

The OB-probe socket is located on the back wall. Inserting the connector and screw on the knurled ring of the plug clockwise to secure.

Specific probe holder is located on the left hand size of front panel.

P probe

The pachymetry OP-probe socket is located on the back wall. The biometry OP probe is equipped with a push-pull type connector with a locking system. To connect the probe insert the plug as far as it goes. Pull the connector body to disconnect the probe (do not pull the cable).

Specific probe holder for the pachymeter straight probe is located on the left hand size of front panel.

3.2.4. Diagram of connecting components

