CERTIFIED COMPANY UNI EN ISO 9001 & UNI EN ISO 13485

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120ML URINE CONTAINERS WITH DEVICE FOR VACUUM TEST TUBE COLLECTION

Urine containers manufactured in medical Polypropylene (PP) and graduated up to 100ml. With screw cap manufactured in Polyethylene yellow colour complete by device for vacuum test tubes collection. Label on the cap for sampling system protection and specimen identification. Needle for sampling has a secure protection manufactured in synthetic rubber. This system makes an hygienic and clean urine collection due the closed system. The health operator must no longer open the containers that reaches the laboratory of analysis. Sterile Device. Latex free.

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CE Marked product - manufactured in compliance with 98/79/CE Directive and D.lgs 332 dtd 08/09/2000

CARATTERISTICHE PRINCIPALI		TECHNICAL FEATURES
Stato microbiologico	STERILE / STERILE	Microbiological status
Materiale impiegato contenitore	POLIPROPILENE / POLYPROPYLENE	Raw material – container
Materiale impiegato tappo	POLIETILENE / POLYETHYLENE	Raw material - cap
Volume nominale contenitore (ml)	120	Nominal volume container (ml)
Scala graduata (ml)	MIN. 20ML – MAX 100ML	Graduated scale (ml)
Incrementi scala graduata (ml)	20	Intervals graduated scale (ml)
Temperature tollerate contenitore	MIN -10°C MAX +121°C	Temperature range - container
Temperature tollerate tappo	MIN -50°C MAX +80°C	Temperature range - cap
Dimensioni contenitore (mm)	Ø 58 x 72	Dimensions - container (mm)
Dimensione tappo (mm)	Ø 61 x 53	Dimensions – cap (mm)
Peso contenitore (gr.)	8,2	Weight - container (gr.)
Peso tappo (gr.)	7,25	Weight – cap (gr.)
Validità del prodotto	5 ANNI / YEARS	Shelf life

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*Intended purpose is “IN VITRO MEDICAL DEVICE” adapted to contain a human biological sample (urine) in order to perform diagnostic analysis laboratory. **For professional use only.***

National classification of medical devices (CND - For Italian law) > W05010203 (Containers for urine collection)

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DO NOT REMOVE the protective label on the container cap: There is a needle under the label, danger of puncture!
Don't urinate in the cone under the label !

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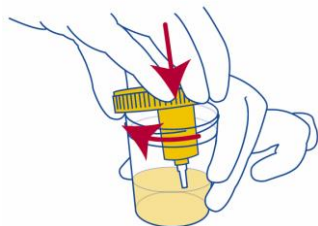
Unscrew the cap of the container not touching with the hands the interior of the container or cap

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Collect the first morning urine or follow the doctor's instructions

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Close the container and tighten the cap tightly. Write their data on the label and apply it on the container. Deliver the container to the laboratory.

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Collect urine sample in the collector with vacuum and sampling system with the cap tightly closed. Identify the sample with patient data

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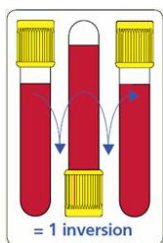
Remove the protective sticker from the cap and insert the tube into the adapter until the cap of the tube is not completely penetrated.

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
Wait until the tube has collected the correct urine volume, and then remove it from the container adapter. Identify the sample with patient data.

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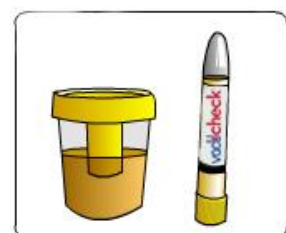In case of tube with preservative, reverse it gently for 5-8 times after sampling in order to obtain a homogeneous mixture of urine and a preservative.

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Re-apply the protective sticker on the container cap and dispose it and the container according to the regulations on force

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Keep out of flame or heat sources which might damage the product

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Do not use after expiry date or if packing is opened

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Do not re-use: Disposable device

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Do not vary the intended purpose of the product

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Store in dry place, Temperature range: min -10°C max +50°C

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Disposal: use appropriate personal protective equipment and act according to applicable regulations

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Before use with particular substance check the resistance / compatibility chart on our catalogue

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