Anexa nr.22 Cîntar mecanic cu taliometru, pentru adulti 130740, S7200HR, Fazzini,Italia
Nr. de inregistrare AMDM: DM000579029

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Base size mm 375x275

TECHNICAL FEATURES:

Mechanical column scale with eye-level beam. Large anti-slip feet surface.

4 adjustable feet and 2 castors for easy movement.

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GB "FAZZINI" wishes to thank you for purchasing one of its range of products and you are invited to read this booklet thoroughly. This booklet contains all of the information necessary to use the device properly. Please adhere closely to the notices contained within and conserve the booklet, without making changes, in a place where it is easy to find and is accessible to the operator. The contents of this manual can be changed, without warning or additional obligations, so that the manufacturer can include variations and improvements. It is prohibited to copy or translate any of this booklet without the written permission of "FAZZINI".

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INSTRUCTION MANUAL

WARNING

READ THE INSTRUCTION MANUAL BEFORE USING THE DEVICE

Cautions

Please unpack the carton carefully and remove all the packages and fillers before starting assemble.

The scale is accurate and high-sensitive with large weighing range, it is widely used in enterprises, schools, hospitals, clinics and sports sectors. It is also the basic examination device for medical and health sectors.

This precision device is very easy to be fitted together because all the major parts are pre-assembled in the factory. Please unpack the carton carefully and remove all the packages and fillers before starting assemble.

ASSEMBLING INSTRUCTIONS

1. Following items are packed in the carton:

- a) Column with head
- b) Platform base
- c) Height rod
- d) Carrying wheels

2. Note: if the column bracket is bent, straighten it before assembling.

- a) Slip the column into the column bracket on the platform base.
- b) Hold the column straightly so that the column can enter into the opening on the column bracket
- c) Screw the column to the column bracket (by using two M6 truss head screws in the front of the column and four at the back).

3. Lay down the scale, with column horizontal to the floor.

- a) Lift the lever
- b) Push the lever
- c) Turn and hook the draught rod inside of the column with the lever

4. Installation for Height Rod:

- a) Pull off the height rod from the carton box
- b) Unscrew the two Philips screws fitted with spacers in gray or black plastic from the holes in the front of the column, using a screwdriver, not supplied with the scale
- c) Extend, without pulling out, the part of height rod just to free the holes which need to match it to the front part of the balance of the column
- d) Insert the screws, previously unscrewed, in the 2 through holes, placing the spacers between height rod and column and screw them with the screwdriver to tighten moderately
- e) To calibrate the scale, move both the weights in position 0 (zero), then rotate the worm calibration screw on the left side of the weights in a clockwise or counterclockwise until the measuring arm is in horizontal position, and that points on the reference mark on the right side.

For accurate weighing, please place the scale on level.

5. Instruction for use of the height rod:

- a) Before stepping onto the scale platform, the spoon should be placed to the horizontal position and be raised over the height of the person to be measured
- b) The person may now step onto the scale platform. The spoon should be held horizontally and above the person's head
- c) Lower the spoon carefully while keeping it horizontally till it touches the top of the person's head gently. If the person is shorter than 101,5 cm., push the latch to the right and lower the spoon till it rests horizontally on the top of the person's head
- d) Get the height of the person as follows:
 - If the back of the spoon points to the outer height rod, then the height it points is the person's height.

- If the back of the spoon points to the inner height rod, then the person's height should be read from the top to the outer height rod (see the arrow on the outer height rod)
- e) Raise the spoon horizontally above the person's head, the person may now step off the scale platform. Hold the spoon horizontally till the person leaves the scale
- f) Put back the spoon to the vertical position and adjust the height rod back to the rest position (i.e. the spoon should be locked in the original place in the range of the inner height rod and the inner height rod should be at its lowest position).

Disposal

With reference to the disposal of the appliance, please see current waste disposal regulations

Fazzini s.r.l. reserves all the rights to change this handbook with no prior notice further to manufacturing changes to the appliance, which has been built in compliance with the following standards: device in conformity with **93/42 CEE – 2007/47**

Fazzini s.r.l. is committed to the monitoring of the medical appliances it distributes on the market, and therefore shall be most grateful of being informed of any problems or suggestions relating to its products, to be sent in writing to: fax 0039/0227409242 / e-mail fazzini@fazzini.it please state your consensus to data processing

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Manufactured and imported by FAZZINI S.R.L.

Fazzini s.r.l.
 S.S.PADANA SUPERIORE, 317
 20090 VIMODRONE - ITALY
 Tel : ++39.02.2651521 Fax : ++39.02.26515221
 e-mail : fazzini@fazzini.it
 web : www.fazzini.it

1. Installation Instructions

Your scale is a precision instrument and should be handled accordingly. Remove all parts carefully to prevent scratching the finish. In assembling the parts, do not use force since the scale has been assembled at our plant and all parts will fit properly if instructions are followed.

Carton Contents: 1 Pillar/Beam, 1 Base, 1 wheels assembly, 1 measuring ruler, 8 screws and 8 washers.

- a. Fix the column to the base using the 4 screws & washers in back and 2 screws & washers in front supplied. Fix the wheels assembly to the base using the 2 screws & washers. (See Fig. 1)
- b. Lay down the scale, with the column horizontal to the floor.
Lift the lever and push the lever,
Turn and hook the draught rod
Inside of the column with lever. (See Fig. 2)
- c. Fix the measuring ruler to the two hexagonal-head screws pre-assembled on the column. (See Fig. 3)
Fix the measuring ruler by tightening the hexagonal-head screws using the relative wrench. Don't tighten too much!

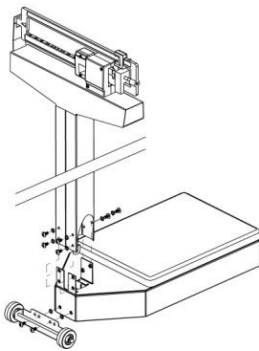
Fig.1

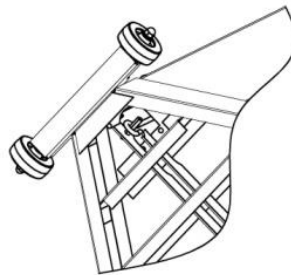
Fig.2

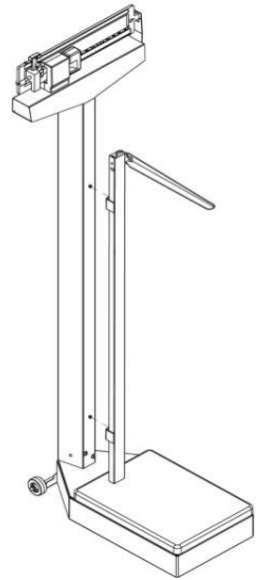
Fig. 3

2. Fault Analysis

SYMPTOM	POSSIBLE CAUSE	CORRECTIVE ACTION
The weighing is incorrect	1. Zero is not adjusted before use. 2. The Scale Base is not in a free status. 3. The hook of the Sensing Weight Lever is not hooked to Knives of the Long Lever Assembly during installation. 4. The Lower Poise Weight is not moved to the appropriate notch.	1. Adjust Zero (Refer to the manual). 2. Move the Scale Base from side to side, back and forth, up and down to make the parts in a free status. 3. Operate according to the manual. 4. Move the Lower Poise Weight to the notch.