GUVERNUL  
REPUBLICII  
MOLDOVA

SERVICIUL FISCAL DE STAT

# CERTIFICAT

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# Operation Manual

---

## Hematology Analyzer Reagent (M-30 series)

English (1~2)

French (3~4)

Spanish (5~6)

Portuguese (7~8)

Italian (9~10)

Turkish (11~12)

© 2009-2018 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. All rights Reserved.  
For this Operation Manual, the issue date is 2018-08.

## Intellectual Property Statement

SHENZHEN MINDRAY BIO-MEDICAL ELECTRONICS CO., LTD. (hereinafter called Mindray) owns the intellectual property rights to this Mindray product and this manual. This manual may refer to information protected by copyright or patents and does not convey any license under the patent rights or copyright of Mindray, or of others.

Mindray intends to maintain the contents of this manual as confidential information. Disclosure of the information in this manual in any manner whatsoever without the written permission of Mindray is strictly forbidden.

Release, amendment, reproduction, distribution, rental, adaptation, translation or any other derivative work of this manual in any manner whatsoever without the written permission of Mindray is strictly forbidden.

**mindray**

**MINDRAY**

are the trademarks, registered or otherwise, of Mindray in China and other countries. All other trademarks that appear in this manual are used only for informational or editorial purposes. They are the property of their respective owners.

## **Responsibility on the Manufacturer Party**

Contents of this manual are subject to change without prior notice.

All information contained in this manual is believed to be correct. Mindray shall not be liable for errors contained herein or for incidental or consequential damages in connection with the furnishing, performance, or use of this manual.

Mindray is responsible for the effects on safety, reliability and performance of this product, only if:

- 1) Use of this product is conducted as instructed in this operation manual by Mindray authorized personnel or skilled/trained clinical professionals;
- 2) The product is stored and transported as instructed in the operation manual and the labeling;
- 3) The product is used only on the specified models.

## **Warranty**

Free service will be provided to:

- Products that are covered by the warranty terms of Mindray.

Pay service will be provided to:

- 1) Expired products;
- 2) Products within the warranty period, but are damaged by improper use or human error, stored in improper environment, or damaged by force majeure or other reason not caused by the product itself.

## **Return Procedure**

Follow the procedure below to return products to Mindray if needed.

Contact Mindray customer service department, inform them the product name and lot No. (the name and lot No. are labeled on the outer package of the product). If the name and lot No. are not legible, your return request will not be accepted. Please specify the product name, model, lot No. and the return reason.

## Company Contact

<b>Manufacturer:</b>	Shenzhen Mindray Bio-Medical Electronics Co., Ltd.
<b>Address:</b>	Mindray Building, Keji 12th Road South, Hi-tech industrial park, Nanshan, Shenzhen 518057, P.R. China
<b>Website:</b>	www.mindray.com
<b>E-mail Address:</b>	service@mindray.com
<b>Tel:</b>	+86 755 81888998
<b>Fax:</b>	+86 755 26582680

<b>EC-Representative</b>	<b>Shanghai International Holding Corp. GmbH(Europe)</b>
<b>Address</b>	Eiffestraße 80, Hamburg 20537, Germany
<b>Tel</b>	0049-40-2513175
<b>Fax</b>	0049-40-255726

### Warning

If the reagent accidentally gets into your mouth, seek medical assistance immediately; if the reagent spills on your skin, wash it off with plenty of water immediately; if the reagent spills into your eyes, wash it off with plenty of water and seek medical assistance immediately.

### Note:

This product must only be used by Mindray authorized personnel or skilled/trained clinical professionals.

## GRAPHICAL SYMBOLS

Batch code

Use-by date

Temperature limit

Authorized representative in  
the European Community

Manufacturer

*In vitro* diagnostic medical  
device

# M-30D DILUENT

## 1. Product Name

M-30D DILUENT.

## 2. Intended Use

For use on Mindray BC-20/BC-21/BC-10/BC11/BC-3000Plus/BC-3000CT/BC-3200/BC-3300/BC-3600/BC-20s/BC-21s/BC-30s/BC-31s/BC-30/BC-31/BC-30e/BC-10e hematology analyzer.

For use as an azide-free, filtered isotonic solution for counting and sizing blood cells.

## 3. Active Ingredients

Sodium Chloride.....3.0-5.5g/L

Sodium Sulfate Anhydrous.....7.5-11.5g/L

Buffering Agents.....1.0-3.0g/L

Anti-fungal and Anti-bacterial Agents.....0.8-2.5g/L

## 4. Warnings and Precautions

Do not inhale and/or ingest. In case of inhalation or ingestion, seek medical advice immediately.

Avoid skin contact. In case of skin contact, flush the affected area with plenty of water immediately.

Avoid eye contact. In case of eye contact, flush the affected area with plenty of water and seek medical advice immediately.

## 5. Storage and Usage

The product must be stored in the temperature of 2°C to 30°C (35°F to 86°F), and the relative humidity must not exceed 90%. The storage room must be well ventilated and without corrosive gas. The shelf life is 2 years.

The product must be used in the temperature of 15°C to 30°C (59°F to 86°F), and it stays valid for 60 days after being opened.

Do not use expired products.

If product has been frozen(either partially or completely), warm it up to room temperature and then mix it by gentle inversion. You can only proceed to test samples after you have confirmed that the measured blank value is within the required range.

Dispose of waste product, unused product and contaminated packaging in accordance with governmental requirements.

The following conditions may produce interference to test results: expired reagent, reagent contaminated by dusts, unclean sample dilution, reagent mixed with those of other manufacturers.

# M-30CFL LYSE

## 1. Product Name

M-30CFL LYSE

## 2. Intended Use

For use on Mindray BC-3000Plus/BC-3000CT/BC-3200/BC-3300/BC-3600/BC-20s/BC-21s/BC-30s/BC-31s hematology analyzer.

For use as a cyanide-free lytic reagent for quantitatively determining hemoglobin and for counting and sizing leukocytes. Use as directed in instrument operator's manual.

## 3. Active Ingredients

Quarternary Ammonium Salts.....	<50g/L
Nonionic Surfactant.....	<15g/L
Isopropanol.....	0.1-1.5ml/L
Ethanol.....	<1.5ml/L

## 4. Warnings and Precautions

Do not inhale and/or ingest. In case of inhaling or ingestion, seek medical advice immediately.

Avoid contact with skin. In case of contact with skin, rinse immediately with plenty of water.

Avoid contact with eyes. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

## 5. Storage and Usage

The product should be stored at 2°C to 30°C(35°F to 86°F) and keep container closed. The shelf life is 2 years.

If the product has been frozen(either partially or completely), warm it up to room temperature and then mix it by gentle inversion. Verify background results before analyzing patient samples.

The product should be used at 15°C to 30°C(59°F to 86°F) once opened, and it is stable for 60 days.

Dispose of waste product, unused product and contaminated packaging in accordance with governmental requirements.

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