

DETROCID ACTIV- USER MANUAL

DETROCID ACTIV

- **PERACETIC ACID BASED MEDICAL INSTRUMENT AND ENDOSCOPE PRECLEANING, DISINFECTION, STERILIZATION PRODUCT**
- **CONCENTRATE – POWDER**

Usage Area

It is used for cleaning and disinfection of general and dental surgical instrument and equipment, healthcare equipment and anesthesia equipments, rigid and flexible endoscopes, laparoscopic instruments, incubators. Detrocid Activ has multi property; enzymatic cleaning, disinfection, sterilization.

Usage form

- For sterilization, prepare %2 solution (add 20 gr product into 1 lt water and mix during 3-5 minutes).
- Immerse instruments into solution completely.
- After effect time (10 min), take out instruments and rinse with water.
- Prepared solution life is 24 hours. In the case of intensive contamination, it is recommended to renew solution earlier.

Usage Concentration

Water Amount	% 2 – Solution
1 litre	20 gr
5 litre	100 gr
10 litre	200 gr
15 litre	300 gr
20 litre	400 gr

Ingredient

Peracetic acid occurring from reaction of disodium carbonate and TAED with water

First aid Informations

- Eyes; In the case of eye contact, wash plenty of water around 5 min.
- Skin: Wash your skin immediately.
- Ingestion: Drink water and apply your doctor.

DETRO HEALTHCARE KİMYA SANAYİ A.Ş.

☎ 212 659 77 62 📞 212 659 77 63

✉ info@detrox.com.tr - www.detrox.com.tr