All4Labels
GLOBAL PACKAGING GROUP

RAKO Etiketten GmbH & Co. KG
Möllner Landstr. 15
22969 Witzhave

Phone: +49 4104 693 0
Fax: +49 4104 693 110
Mail: info@all4labels.com
Web: www.all4labels.com

Product Name	Mat.No.	Index	Dimensions	Package	Country	EAN
thermosept NKZ	20003125	A	280 x 105 mm	HE-KA	DE, EN, FR, PL	-
Application Area / Category	Label No.	Version	Techn. Spec.	Date	Operator	Font Size
Instrumente / MP	305064112_00	05	015	12.03.20	NA	6 pt./90%

Sprache

Pantone 3005

1

Pantone 3125

2

Pantone 032

3

Schwarz

4

5

6

7

8

9

10

Alt: 20000641

Exclusion of liability for the implementation of statutory provisions and color liability.

This PDF is not color-coded, it only serves to check text, status and color distribution. Compliance with applicable statutory provisions with regard to font sizes, industry-sector-specific labeling requirements, trademark, competition and licensing laws must be checked and ensured by the customer.

Please use the overprint option for PDF preview.

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ACHTUNG
WARNING
ATTENTION
UWAGA

UN3265

CORROSIVE LIQUID, ACIDIC, ORGANIC, N.O.S.
(Citric acid)

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thermosept® NKZ

EN citric acid-based neutraliser for the machine processing of medical instruments
• phosphate-free • excellent material compatibility

► **Application areas:** For professional use only. Liquid neutraliser, for use after alkaline cleaning, during the reprocessing of surgical, microsurgical and MIS instruments, stainless steel containers, rigid endoscopes, anaesthetic equipment, theatre footwear and baby bottles. Also suitable for use after neutral cleaning with thermosept® RKN-zym to obtain optimum cleaning performance. Can be used in stand-alone models or conveyor-type machines ► **Use-concentration:** After alkaline cleaning: 0.1 - 0.2 % (i.e. 1 - 2 g/l). Do not exceed this dose. After neutral cleaning: 0.1 % (i.e. 1 g/l). Dosage takes place according to machine pump type ► **Composition:** Organic acid ► **Hazard notes and safety advice: WARNING: H290** May be corrosive to metals. **H319** Causes serious eye irritation. **P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P337+P313** If eye irritation persists: Get medical advice/ attention. ► Any serious incidents related to the product are to be reported to the manufacturer and competent authority.

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