OXY CLEANER 6%

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OXY CLEANER 6%

MEDICAL SURGICAL DEVICE

Pag. 1 di 2

1. COMMERCIAL NAME

OXY CLEANER 6%

2. QUALIFICATION

Authorized for use in PT2 and PT4. Registration of the Ministry of Health n. 20298

Ready-to-use liquid disinfectant for floors, walls and hard surfaces

3. MANUFACTURER AND OWNER A.I.C

HOUND CLEANER Srl, Milano

4. COMPOSITION:

100 grams of product contains: Hydrogen peroxide (H₂O₂) (CAS 7722-84-1) 6 gr, demineralized water until 100 grams.

5. PHYSICO-CHEMICAL CHARACTERISTICS:

Appearance	clear colourless liquid
density at 20°	1,019
pH as it is	4,0 ± 0,3

6. PROPERTIES:

OXY-CLEANER 6% is a disinfectant for floors, walls and hard surfaces for the food industry, community and sports facilities. Spray evenly with a special trigger on the surfaces to be treated only after thorough cleaning of the surface, use new gloves for each application. The active principle is active against: Bacteria (Gram (+) and Gram (-), Fungi (Candida albicans, Aspergillus niger) and molds.

7. METHOD OF USE

The product is used as it is by applying it to the non-porous surface. Contact times and activity: contact time of at least 5 minutes for the disinfectant activity.

Standard	Microrganism	Contact time	Activity	Conditions
EN1276 LogR >5	<i>Staphylococcus aureus</i> ATCC 6538	5 minutes	Bactericidal	Dirty conditions
	<i>Escherichia coli</i> ATCC10536			
	<i>Pseudomonas aeruginosa</i> ATCC 15442			
	<i>Enterococcus hirae</i> ATCC 10541			
EN1650 LogR >4	<i>Candida albicans</i> ATCC 10231	15 minutes	Fungicidal	Dirty conditions
	<i>Aspergillus brasiliensis</i> ATCC 16404			

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The test were carried out by laboratory respecting the standard conditions and the rate kill Logs After application, rinse the treated surface with drinking water if intended for contact with food.

8. WARNINGS:

Do not ingest - Keep locked up and out of reach of children - Do not contaminate food or feed or containers intended to contain them with the product - Do not use for uses other than those indicated, avoid direct or indirect contact with food products - Store in a cool and well ventilated place - Keep container tightly closed - In case of contact with skin, wash immediately with plenty of water and soap - In case of contact with eyes, rinse immediately with plenty of water and seek medical advice - In case of accident or if you feel unwell seek medical advice immediately (show the label if possible).
DO NOT DISPOSE OF THE CONTAINER IN THE ENVIRONMENT AFTER USE. NOT TO BE SOLD IN BULK
However, consult the safety data sheet.

VALIDITY: 12 months at room temperature and in the original packaging.

FOR PROFESSIONAL USE

9. PACKAGING

FM0884 5L Tank x4 card board pieces FM1101 1L
Bottles x12 card board pieces

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