



PRODUCT SPECIFICATIONS

Product Components

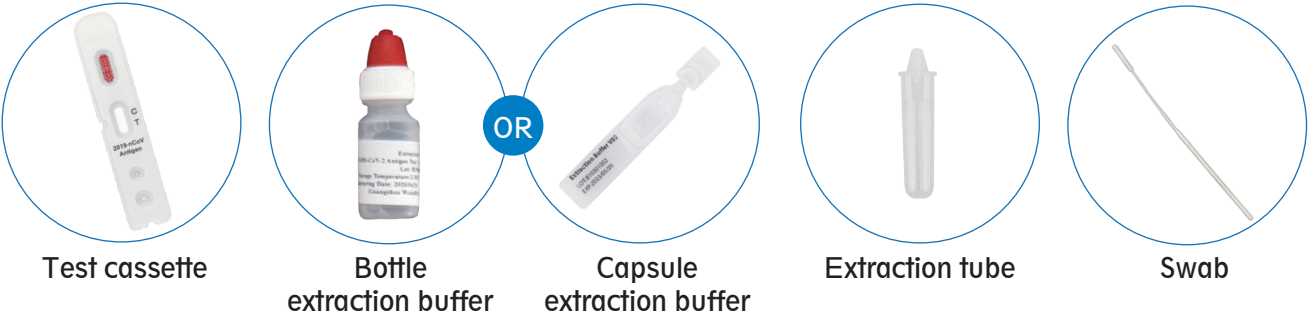

Operation procedure

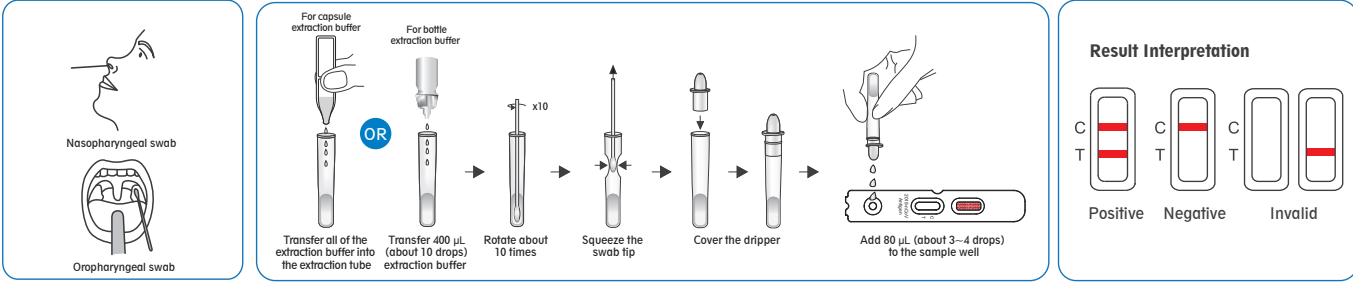

* The detail of this component (e.g. picture, operation procedure) will be provided upon request.

Performance

Reagents		PCR		Total
		Positive	Negative	
Wondfo 2019-nCoV Antigen Test (Lateral Flow Method)	Positive	208	1	209
	Negative	4	361	365
Total		212	362	574

Sensitivity: 98.11% (95%CI: 95.24%~99.48%)
Specificity: 99.72% (95%CI: 98.47%~99.99%)
Total agreement: 99.13% (95%CI: 97.98%~99.72%)

Order information

Catalog No.	Product Name	Packing Size	Sample Type	Storage Condition	Shelf Life	Qualification
W196	2019-nCoV Antigen Test (Lateral Flow Method)	1T/5T/20T	Nasopharyngeal swab or oropharyngeal swab	2~30 °C	12 months	CE

WONDFO BIOTECH
WeAreWorkingForYourHealth

Wondfo

Guangzhou Wondfo Biotech Co., Ltd.
Add: No. 8 Lizhishan Road, Science City, Luogang District, 510663, Guangzhou, P.R. China
Tel: +86-20-32296083 Fax: +86-20-32296063
E-mail: sales@wondfo.com.cn Websites: en.wondfo.com.cn
Wondfo reserves all rights. All contents of this brochure are copyrighted by Wondfo.



Wondfo

WONDFO
2019-nCoV
ANTIGEN
TEST

Speed Up the **COVID-19** Control !

WONDFO 2019-nCoV ANTIGEN TEST



Direct detection
of the virus



Instant results
within 15mins



Easy to use, no
equipment required



Room temperature
storage (2~30°C)




Non-invasive sampling
(sample type: nasopharyngeal
or oropharyngeal swab)



Early detection of COVID-19
(WHO recommends the testing period
is from 3 days before to 5-7 days
after symptoms onset)

CURRENT DIAGNOSTIC METHODS FOR COVID-19

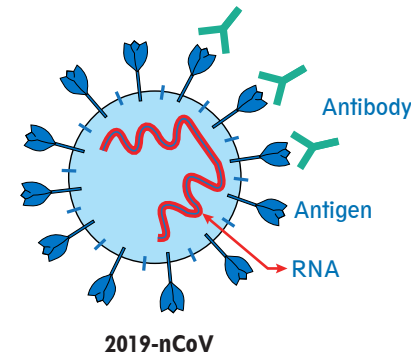

Antigen test

Detect the antigen of the virus, indicating the active viral infection.

RT-PCR

Detect the RNA of virus, indicating the active viral infection.

Antibody test


Detect the antibody generated by immune response after viral infection, indicating the active or past viral infection.

WHEN TO USE ANTIGEN TEST?

Releasing profile

Levels of 2019-nCoV virus and antibodies after infection

*For illustrative purpose only

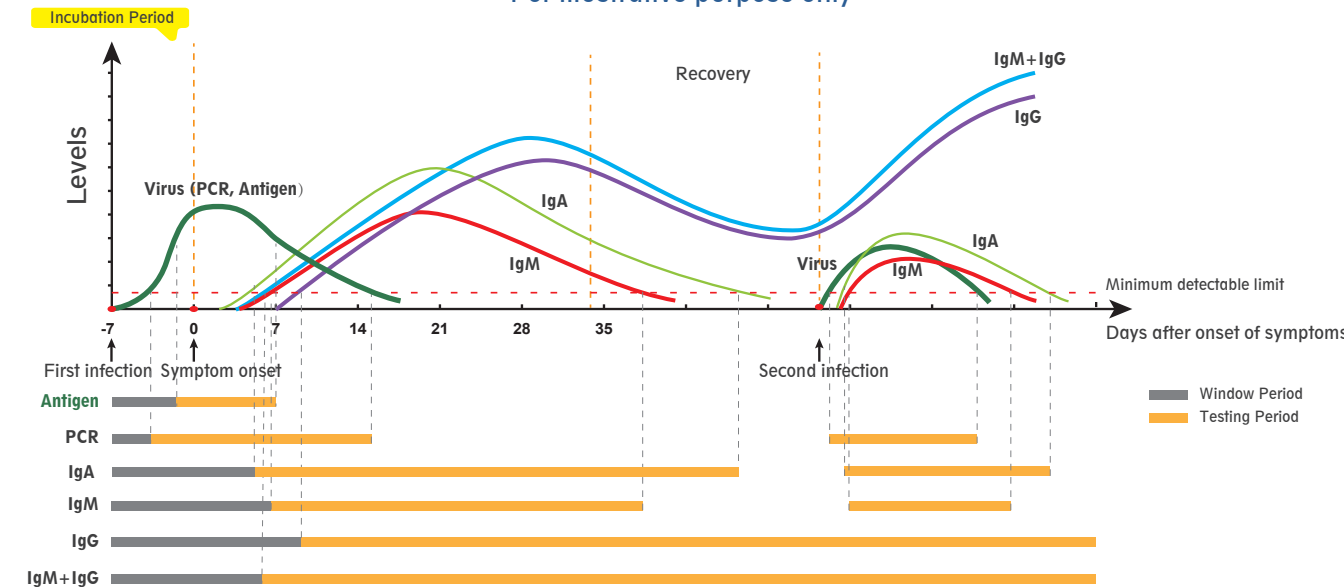

* Incubation Period: 1-14 days, mostly 3-7 days
* Antibody Window Period: 5-10 days after onset of symptoms

* The minimum detectable limit varies with methodology and sensitivity of test
* IgG antibody test can be referred as one of discharge criteria for recovering COVID-19 patients


ANTIGEN TEST ADVANTAGES

Antigen test OVER RT-PCR

- Short turn-around time
(Antigen test: 20mins vs. RT-PCR: 2hours)
- Inexpensive cost, no equipment required and simple operation make antigen test suitable for point-of-care (POC) setting usage.

Antigen test OVER Antibody test

- Detect the virus directly, allowing the early detection of COVID-19
- Non-invasive sampling
(sampling type: blood vs. swab)

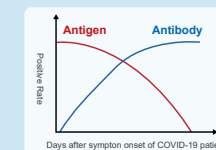


ANTIGEN TEST APPLICATION

Similar to RT-PCR, the detection of antigen indicates the active infection. Under the circumstance that the area(s) is still undergoing widespread community transmission and/or the area with limited RT-PCR resources, antigen can be used for aiding in the diagnosis of COVID-19 suspect patients.




* American CDC also recommends to use rapid antigen tests for screening testing in high-risk congregate settings where the immediate result is required.

Result interpretation

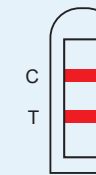

POSITIVE

The patient is undergoing active 2019-nCoV infection. Further isolation is required.

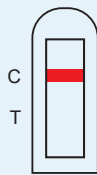


NEGATIVE

The patient should be further evaluated by RT-PCR, especially if the result of the antigen test is inconsistent with the clinical context.

ANTIGEN TEST OFFICIAL GUIDELINES



Antigen-detection in the diagnosis of novel coronavirus (2019-nCoV) infection using rapid immunoassays

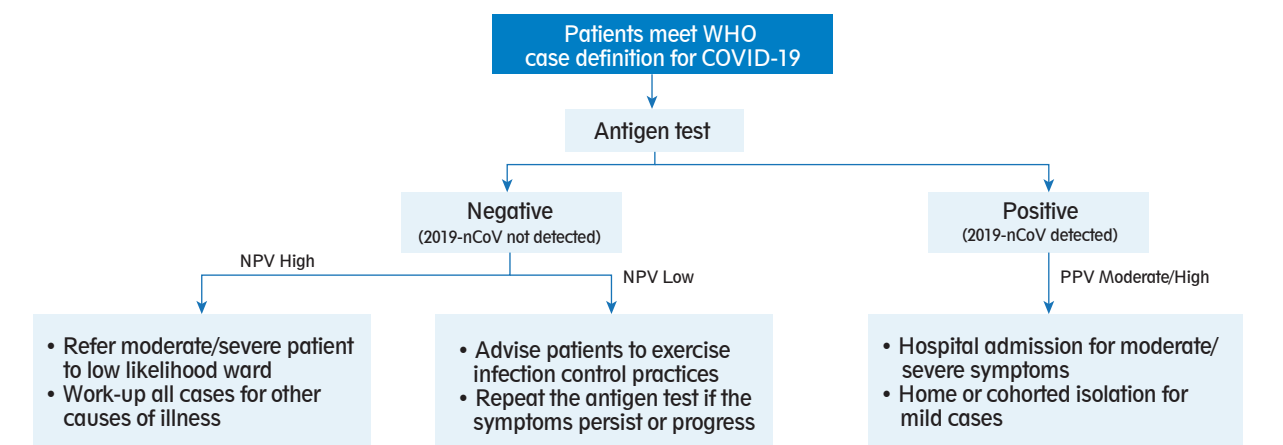

NPV- negative predictive value PPV- positive predictive value
*The value for NPV and PPV is decided based on products performance and disease prevalence in applied scenarios.

Other antigen test related guidelines

- Interim Guidance for Rapid Antigen Testing for 2019-nCoV, American CDC (8-16-20)
- Considerations for Use of 2019-nCoV Antigen Testing in Nursing Homes, American CDC (8-27-20)
- Antigen-Detection in the Diagnosis of 2019-nCoV Infection Using Rapid Immunoassays, WHO (9-11-20)
- Considerations for Implementation of 2019-nCoV Rapid Antigen Testing, APHL (9-2-20)