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Test tubes conical bottom for a complete sample collection. Light blue screw cap which guarantees the perfect leak-proof, with milled rim allowing a safe and easy opening/closing. Graduated scale and writing surface serigraphed. Excellent high and low temperatures resistance and chemical agents. Manufactured in PP, particularly transparent. Autoclavable at 121°C and freezable up to -80°C. R.C.F. max: 10,000 x g (fixed angle rotor). Latex free device. Sterile, non-pyrogenic.

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CE Marked product - manufactured in compliance with 98/79/CE Directive and D.lgs 332 dtd 08/09/2000

CARATTERISTICHE PRINCIPALI		TECHNICAL FEATURES
Stato microbiologico	STERILE / STERILE	<i>Microbiological status</i>
Materiale impiegato provetta	POLIPROPILENE / POLYPROPYLENE	<i>Raw material test tube</i>
Materiale impiegato tappo	POLIETILENE ALTA DENSITA' / HIGH DENSITY POLYETHYLENE	<i>Raw material cap</i>
Temperature tollerate provetta	MIN -80°C MAX +121°C	<i>Temperature range test tube</i>
Temperature tollerate tappo	MIN -50°C MAX +80°C	<i>Temperature range cap</i>
Scala graduata	5 - 50 ML	<i>Graduated scale</i>
Incrementi scala graduata	2,5 ML	<i>Intervals graduated scale</i>
Dimensioni provetta	Ø 30 x 115 MM	<i>Dimensions test tube</i>
Diametro interno provetta	Ø 29 MM	<i>Tube inner diameter</i>
Dimensione tappo	Ø 33.5 x 13.5 MM	<i>Dimensions cap</i>
Peso provetta (g)	8.9	<i>Weight test tube (g)</i>

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Classificazione EDMA > 51091001 - Other containers for samples of human origin

Basic UDI-DI (Global Model Number – GMN): 805577609FT02STETUBE01Y8

In Vitro Diagnostic Medical Device "specimen receptacles" adapted to contain a human biological sample (blood, urine, faeces, pus, sputum, saliva, semen, cells, nails, hair) to perform in vitro diagnostic analysis laboratory. For professional use only.

National classification of medical devices (CND) > W050301020102 (Samples analyses, plastic tubes without additives).

EDMA code > 51091001 - Other containers for samples of human origin

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- For professional use only -

Use only for laboratory diagnostic analysis of human biological samples in healthcare settings.

Device without measuring function.

Any presence of a graduated scale is intended for indicative purposes and without a metrological function.

Do not vary the intended purpose of the device. Please contact the Manufacturer in the event of doubts concerning the intended purpose.

Before use check the perfect closure of the device if applicable.

Handle using personal protective equipment: risk of contamination.

After collection, store the sample as prescribed by the analytical method and deliver to the Laboratory within the maximum time specified by the same.

Do not use the Device in case of evident signs of breakage, injuries, cracks that could compromise its correct use.

Keep out of flame or heat sources which might damage the device.

Do not use after expiry date or if packing is opened.

Do not re-use: Disposable Device. Any reuse could cause sample contamination.

If sterile (ionizing radiations) or aseptic Device: Microbiological status in undamaged pack.

Do not re-sterilize.

Use the Device only with compatible and / or supplied accessories (caps, supports, etc.)

Do not put under direct sun rays; store in a dry, cool place (U.R. max 75% at 26°C)

Store in dry place, temperature range: min. -10°C (14°F) max +50°C (122°F)

Disposal according to applicable regulations, risk of infection.

Before use with particular substances check the resistance/compatibility chart in our catalogue.

Test tubes: centrifuge rotation speed (max 20 minutes, room temperature):

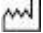
PMMA r.p.m 3.000 (R.C.F. min 1.500 x g) – PS r.p.m 3.000 (R.C.F. min 1.500 x g) – PP r.p.m 5.000 (R.C.F. min 4.000 x g) - TPX r.p.m 4.500 (R.C.F. min 3.200 x g).

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Aptaca S.p.A. Regione Monforte, 30 - 14053 Canelli (Asti) Italy