

## 2.3 Radiography Program Screen

This screen displays the radiography programs.

Press the Tab keys to switch between the radiography program pages on the screen.

Press a Radiography Program key to switch to the radiography step screen.

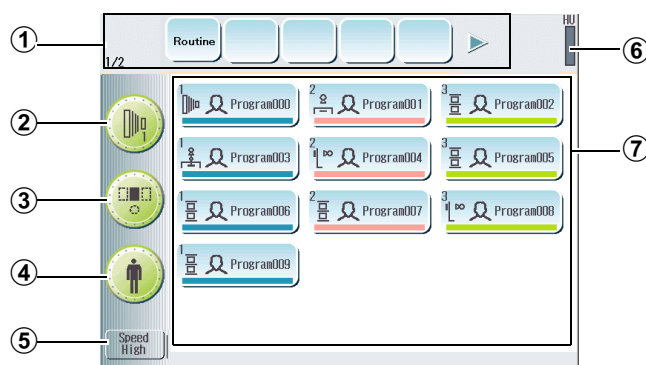


Figure 2.4 Radiography Program Screen

### ① Tab Keys

The radiography programs groups are displayed as an icon or text.

Press a key to switch between the radiography program pages on the screen.

The ◀ (left) and ▶ (right) keys are displayed if ten Tab keys have been configured.

Press the ◀ (left) key beside the Tab keys to display the first half of the Tabs.

Press the ▶ (right) key beside the Tab keys to display the second half of the Tabs.

### ② Technique Key

Displays the radiography technique.

Press this key to change the technique.

### ③ Photo Pickup Field Key

Displays the AEC photo pickup field.

Press this key to select the photo pickup field.

### ④ Body Thickness Correction Key

Displays the body thickness correction conditions.

Press this key to select the correction conditions.

### ⑤ Speed Key

Displays the speed (film sensitivity).

Press this key to change the speed.

### ⑥ Heat Unit Gauge

Displays the X-ray tube accumulated heat units.

### ⑦ Radiography Program Key

Displays the name of radiography programs that include multiple radiography steps.

Press this key to display the radiography steps.