


R-SW touch screen control panel

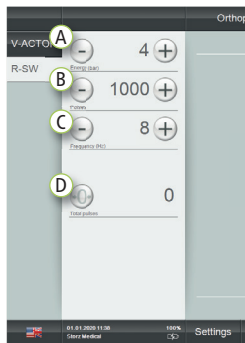

- A Setting the intensity level
- B Setting the number of pulses
- C Setting the frequency
- D Resetting the pulse counter/ display of pulses administered


R-SW handpiece control panel



- E Trigger button
- F Reduce intensity level
- G Increase intensity level
- H Reduce frequency
- I Increase frequency
- J Intensity display/pulse limit/ pulse counter/frequency


Reset pulse counter by pressing both (F) and (H).

V-ACTOR® (optional) touch screen control panel

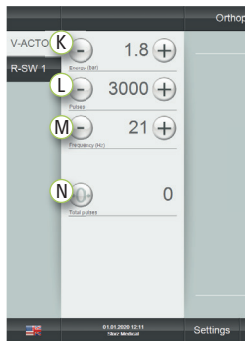

- K Setting the intensity level
- L Setting the number of pulses
- M Setting the frequency
- N Resetting the pulse counter/ display of pulses administered

Main screen

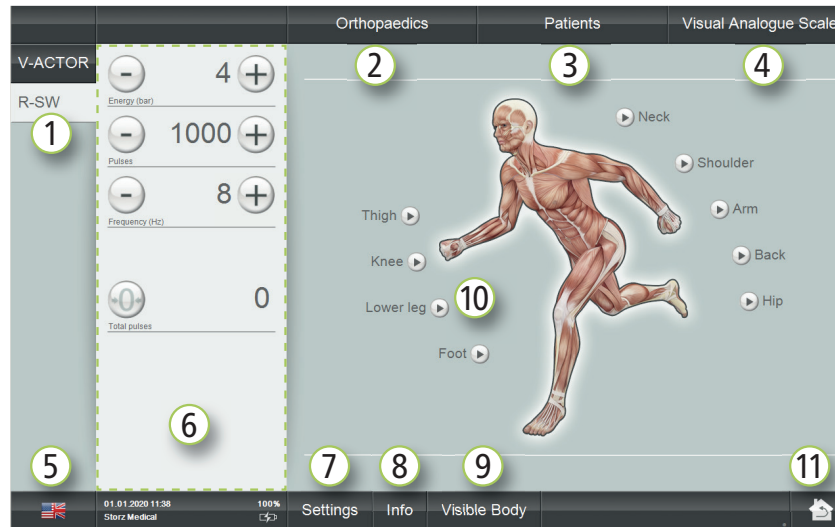

- 1 Operating modes (selectable only with handpieces connected), active mode is highlighted in white
- 2 Indication selection/management
- 3 Patient records/management
- 4 Visual Analogue Scale
- 5 Language selection
- 6 Control panel
- 7 System settings
- 8 Info (version and device data)
- 9 Visible Body® (digital anatomy atlas)
- 10 Indications selection/management via treatment zones
- 11 Return to main screen

R-SW key handpiece functions

(only when a touch screen is not attached)

- Pressing both (G) and (I) takes you to the main menu.
- Pressing (F) and (G) switches between menu items (Pulse limit/Information).
- Pressing (I) takes you to the respective menu.



Pulse limit menu

- Press (F) or (G) to reduce or increase the pulse limit (increments of 50) and then confirm the nominal pulse value by pressing (I).
- Press (H) for the main menu.



Information menu

- Press (F) or (G) to switch between the information menu items.
- Press (H) for the main menu.

System information items are listed in the user manual.

Getting started

- Start by pressing the On/Off button (E) located on the front of the device. The start-up process takes up to 2 minutes.
- You can operate the device with either the handpiece or via the touch screen control panel (optional).
- Please confirm the flashing display on the handpiece by pressing the trigger button (E).

Recommended indications (touch screen)

- Press the home button (H) for the main screen.
- Select indication (2).
- Follow the on-screen instructions.



Step 1

- V-ACTOR®**
- Pulses 3000 per session
 - Intensity 1.8 – 3.0 bar
 - Frequency 21 – 50 Hz

Step 2a

- R-SW with D-ACTOR® transmitter**
- Pulses 4000 per session
 - Intensity 0.3 – 2.5 bar_{eff}
 - Frequency 14 – 16 Hz

Step 3

R-SW with radial transmitter

Step 2b

Step 2c

Alternative/complementary to Step 2a

- PERI-ACTOR®**
- Pulses 2000 per session (3 x per muscle chain)
 - Intensity 0.3 – 2.5 bar_{eff}
 - Frequency 16 – 21 Hz

- SPINE-ACTOR®**
- Pulses 2000 per session
 - Intensity 0.3 – 2.5 bar_{eff}
 - Frequency 12 – 16 Hz

»Advanced Technologies«

Treatment recommendations	Step 1: V-ACTOR®	Step 2a: R-SW	Step 2b: PERI-ACTOR®	Step 2c: SPINE-ACTOR®	Step 3: R-SW			
Indications	Vibration head	Transmitter	Transmitter	Transmitter	Pulses/session	Intensity (bar _{eff})	Frequency (Hz)	Transmitter
Achillodynia	V25	D20-S/D20-T*	scraper	–	2500	0.3 – 2.5	12 – 16	R15/C15*
Plantar fasciitis/heel spur	V25/V40	D20-S	knuckle	–	2500	0.3 – 2.5	12 – 16	R15/DI15*
Tibial stress syndrome	V25/V40	D20-S	scoop	–	2500	0.3 – 2.5	12 – 16	R15/F15*
TP/SWA – calf muscles	V25/V40	D20-S/D20-T*	scraper	–	2500	0.3 – 2.5	12 – 16	R15/DI15*
Patellar tendinitis	V25/V40	D20-S/D20-T*	scraper	–	2500	0.3 – 2.5	12 – 16	R15/C15*
TP/SWA – thigh muscles	V10/V25/V40	D20-S/D35-S*	scraper	–	2500	0.3 – 2.5	12 – 16	R15/DI15*
Trochanteric tendinopathy	V25/V40	D20-S/D20-T*	sphere	–	2500	0.3 – 2.5	12 – 16	R15/DI15*
TP/SWA – lumbago	V25/V40	D20-S/D20-T*	scraper	SPINE-ACTOR® 2/3	2500	1.0 – 2.5	12 – 16	R15/A6*
TP/SWA – dorsalgia	V25	D20-S	scraper	SPINE-ACTOR® 2/3	2500	0.3 – 2.5	12 – 16	R15/C15*
TP/SWA – cervical syndrome	V25	D20-S	scraper	SPINE-ACTOR® 2/3	2500	0.3 – 2.5	12 – 16	R15/T10*
Calcific tendinitis	V25/V40	D20-S/D20-T*	scraper	SPINE-ACTOR® 2/3	2500	0.3 – 2.5	12 – 16	R15/DI15*
Lateral/medial epicondylitis	V25/V40	D20-S	scraper	–	2500	0.3 – 2.5	12 – 16	R15/F15*
TP/SWA – cervical spine muscles (atlas)	V25	D20-S	scraper	–	400 – 600	0.3 – 0.8	8	ATLAS

TP/SWA (Trigger points/Shock wave acupuncture)

* (optional)