
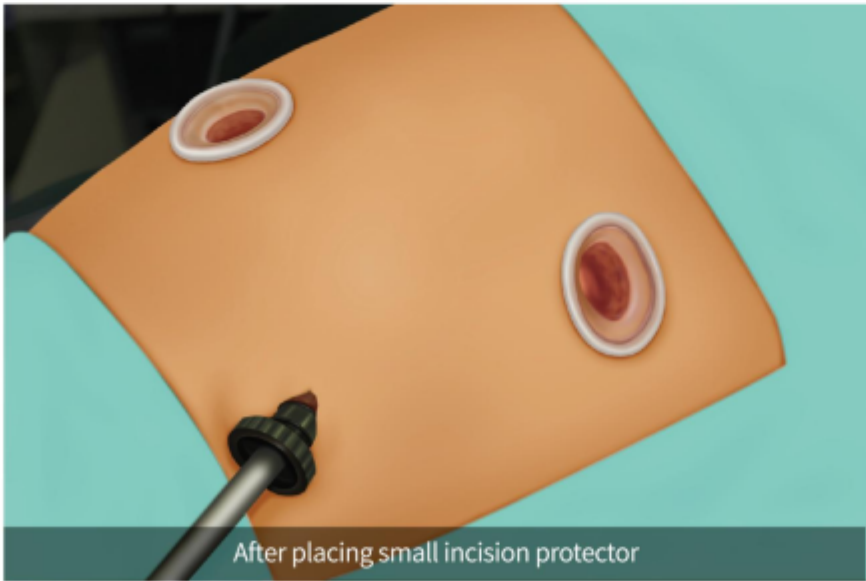
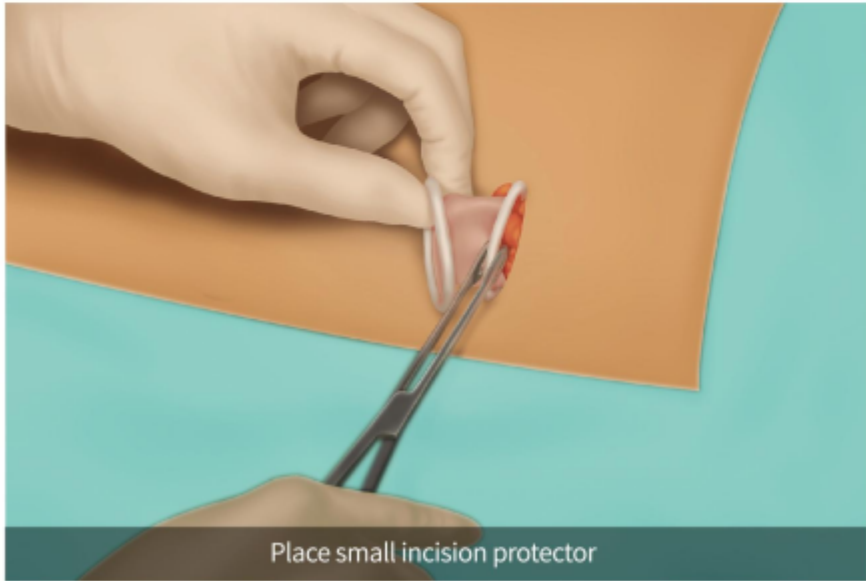


Product Demo


- 01 Make an appropriate sized surgical incision in the surgical area.
- 02 Insert the inner ring (Green) into abdominal cavity through the incision.
- 03 Roll the outer ring (White) inward at the position of 10 o'clock and 2 o'clock until it stops.
- 04 Through the tunnel of incision protection sleeve to operate.

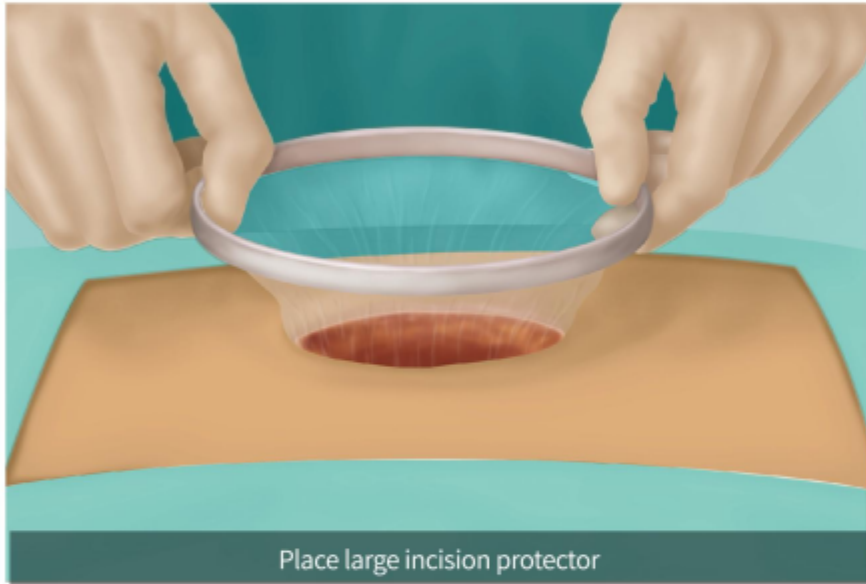

Order Information

Models		Size	Inner ring diameter (mm)	Tunnel diameter (mm)	Outer ring diameter (mm)	Tunnel length (mm)	Incision length (cm)
WZIPS-A	S	50/50-50/220 ⁽¹⁾	50	50	50	220	1.8-3
		60/70-60/220 ⁽²⁾	70	60	60	220	2-4
		60/70-60/150	70	60	60	150	2-4
		80/90-80/150	90	80	80	150	2.5-6
	M	120/130-120/220	130	120	120	220	5-9
	M&L	150/160-150/220	160	150	150	220	7-12
	L	180/190-180/220	190	180	180	220	9-14
		220/230-220/250	230	220	220	250	11-17
		250/260-250/100	260	250	250	100	17-20
		320/330-320/250	330	320	320	250	15-22
WZIPS-B	S	40/30-15/25	40	30	15	25	1.5
		50/40-25/25	40	50	25	25	2.5
		70/70-35/25	70	70	35	25	3.5
		70/60-45/25	60	70	45	25	1.5-2.5
		150/70-60/5	70	150	60	5	2-3

⁽¹⁾ and ⁽²⁾ are available for gastroschisis.
*Our sleeves are also applicable to thyroid surgery and caesarean section.
*OEM services are available.
*For more model recommendations, please consult us.

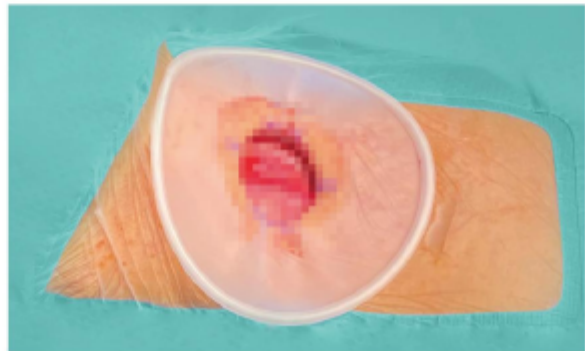

thyroid surgery



caesarean section