HumaTrol N

Quality Control Serum for Clinical Chemistry Assayed

Package Size

REF 13511 6 x 5 ml
IVD

Properties and Use

HumaTrol N is a lyophilised universal control serum prepared from bovine serum with assayed values for all important components of human serum. It may be used to control the precision and accuracy of manual and automated methods. Most parameters are in the normal range or in the borderline between normal and pathological.

Target Values

The maximum allowed range (for individual results) was calculated by the target value \pm the maximum permissible deviation. The values were determined in own quality control laboratories and in selected laboratories.

Instructions for Use

Reconstituting the Lyophilisate

Open the HumaTrol N vial carefully to avoid any loss of the substance. Accurately pipette 5.0 ml of distilled water to the contents of one vial. Close the vial carefully and allow standing, protected from light, for at least 30 minutes at room temperature. Then completely dissolve and thoroughly mix any undissolved substance still adhering to the flask and the stopper, by careful swirling, rocking or rotating. **Do not shake. Avoid foaming.**

Storage/Stability

If stored unopened in a refrigerator (2...8°C), the contents are usable until the expiration date printed on the package and the vial. The components in reconstituted HumaTrol N are stable at **2...8°C** for at least: anorganics, organics and enzymes **7 days**, bilirubin 4 days and acid phosphatase 2 days. Freshly dissolved HumaTrol N can be divided into portions and frozen once (-20°C) for at least one month. **Carefully mix thawed serum before use.**

In order to prevent contamination and to protect from light (Bilirubin, CK) we recommend storing the original bottle in a dark place and removing the necessary amount for a day's usage.

The activity of acid phosphatase definitely decreases at a neutral pH. Stabilisation is achieved by adding one drop (25 - 30 μ l) of 0.7 mol/l acetic acid to 1 ml of reconstituted control. 2 days usage is possible after stabilisation when stored at 2...8°C.

Glucose values may decline after 4 days.

For determining alkaline phosphatase, HumaTrol N should not be used earlier than 2 hours after reconstitution. Alkaline phosphatase stabilises within about 48 hours, the values may increase up to 20%.

Safety Notes

HumaTrol N has been prepared from healthy BSE-free cattle. Being of non-primate origin, the serum avoids most of the risks associated with sera based on human serum (e.g. Hepatitis B and C and HIV). Nevertheless the control serum should be handled as if potentially infectious.

Performance Characteristics

Typical performance data can be found in the Verification Report, accessible via

www.human.de/data/gb/vr/cs-hu.pdf or

www.human-de.com/data/gb/vr/cs-hu.pdf

If the performance data are not accessible via internet, they can be obtained free of charge from your local distributor.

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