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#### SAFETY CONTAINERS ROUND SHAPE

*This line includes round shape disposable and puncture resistant containers for the disposal of sharp and cutting waste and it comes in sever sizes.*

*The conical shape and the indent on the lid ease the disposal and the remove of needles, whilst the anti-defluxion flow wings on the lid prevent the outflow of the waste. All the models have temporary and definite closure and are fitted with solid handle; the three most volumious models are fitted with an ergonomic handle that eases use.*

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*Ecological plastic material (colored copolymer polypropylene compliant to REACH regulations and with masterbatch heavy metal free and cadmium free). The container body is yellow and the lid is red to allow the immediate detection and warning of the dangers of the content.*

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*Compliant to AFNOR NF X-30-500  
Compliant to ISO 23907  
Compliant to ADR - UN Certificate  
FDA (Food and Drug Administration) Approval K071517  
Compliant to German standard TRBA 250  
Autoclavable at +134°C for 18 minutes*

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- *Ergonomic handle for the more spacious models*
- *Anti-defluxion wings that prevent the outflow of the waste*
- *Machanism for unhooking insulin needles*
- *Irreversible permanent closure*
- *Maximum filling level indicator*
- *Various supports available*
- *No emission of any harmful gasses during the inceneration process. Once filled, the containers have to be disposed through incineration, releasing no toxic emissions*
- *Suitable to autoclave at 134° for 18 min*

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*Item designed for low volumes of waste (care tray, trolley, domestic care, doctor's office, etc.)*

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*Item designed for frequent use (Hospitals, Healthcare Centres, Clinics, etc.)*

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Item designed for frequent use (Hospitals, Helthcare Centres, Clinics, etc.)

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**ASSEMBLY:** Firmly press on all the perimeter, until the lid clicks on the body

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### IMPORTANT!

Before use, press on the anti-reflux fins in order to create a funnel-shaped opening. This device eases the insertion of the waste and creates a barrier to prevent from leakage.

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**TEMPORARY CLOSURE:** The container could be temporarily closed using the cap upside down.

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**PERMANENT CLOSURE:** The container could be closed permanently applying pressure on the cap until it clicks on the lid

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*During use, open and close the cap in temporary mode.*

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*Close the container permanently and irreversibly once full, positioning the lid on the permanent closure side.*

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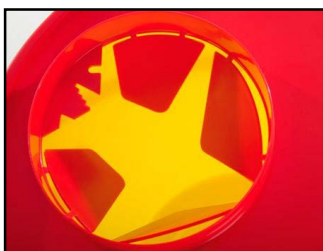
*Use the "V" removal device to separate needle and syringe*

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**Insulin pens removal:** Use the "marguerite" device and pull the pen bottom-up (leverage effect) before unscrewing

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**Hole for introduction of waste:** large orifice to allow the introduction among others, of tubes, scalpels, etc..

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*Do not reopen the closed container and sealed permanently*

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*Do not attempt to rescue objects already in the container*

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*Respect the filling level indicated: 80% of real capacity*

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*Containers are tested to not being perforated by sharp objects allowing safety to the operators, as long as the container is used properly according to instructions*

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*Keep out of flame or heat sources which might damage the product*

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*Do not use after expiry date or if packing is opened*

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*Do not re-use: Disposable device*

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*Do not vary the intended purpose of the product*

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*Store in dry place, Temperature range: min -10°C max +50°C*

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*Disposal: use appropriate personal protective equipment and act according to applicable regulations*

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*Before use with particular substances check the resistance / compatibility chart on our catalogue*

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#### SAFETY CONTAINERS SQUARE SHAPE

Disposable container for sharps and cutting objects disposal. Three sizes square shape containers, solid and stable shape, with universal trolley handling system, vertical and horizontal. Lid with temporary and definitive closure, hermetic even if turned upside down. The lid, apart from having a wide open, is fitted with indents for the removal of needles. Provided with ergonomic handle. Maximum filling level indicator. All the models have temporary and definite closure and are fitted with solid handle.

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*Ecological plastic material (colored copolymer polypropylene compliant to REACH regulations and with masterbatch heavy metal free and cadmium free). Yellow body, yellow lid and red plug in order to signal the dangerous content.*

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FDA (Food and Drug Administration) Approval K071517

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Autoclavabile a +134°C per 18 minuti

*Quality certification NF302, Compliant to NF X 30-511*

*Compliant to ISO 23907:2012*

*Compliant to ADR - UN Certificate*

*FDA (Food and Drug Administration) Approval K071517*

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**ASSEMBLY:** Place the lid on the body with the "FRONT" signal in the side of the label, pressing the opposite corners and sides

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**TEMPORARY CLOSURE:** Turn the red plug and cover the hole on the lid. At this stage closure is temporary.

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**PERMANENT CLOSURE:** First, place the red plug in the temporary closure position.

Place the yellow cap in the "arrow against arrow" position and press firmly. At this stage closure is permanent

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*Turn the movable part of the lid clockwise, opening it according to the size of the object to dispose*

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*Close the container permanently and irreversibly once full*

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*Use the "V" removal device to separate needle and syringe,*

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*Use the product only with the provided lid*

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*Do not use the product before fitting the lid*

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*Respect the filling level indicated*

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