

Export Options

Item	Description
Export Type	<ul style="list-style-type: none"> • DICOMDIR • BMP • JPEG • TIF • RAW • PNG
Burn Image Info	• Applies the patient or study information into pixels.
Anonymize Information	• Anonymizes patient information.
Burn Annotation	• Applies the annotation and shutter information to pixels.
Apply Changed W/L	• Applies the changing window level to pixels.
Burn Scale Bar	• Applies the scale bar to pixels.
Include Viewer	• Includes a viewer to display the exported DICOM file.
Include Report	• Includes a report in DICOM SR format.
Transfer Syntax	• Same as the transfer syntax used for sending study.




- Click on the **Stop** button to stop studies from being exported.



- The following options cannot be applied depending on the export type:
 - **DICOM DIR** - Burns Image Info., Burn Scale Bar
 - **BMP, JPEG, TIF, PNG** - Anonymizes Information, Include Viewer, Include Report, Transfer Syntax
 - **Raw** – Not all options can be applied.

3.4 DICOM Q/R Tab

Click on the  button in the **DICOM Q/R** tab to query or retrieve studies in the DICOM server which supports DICOM Q/R service.



- The **DICOM Q/R** tab follows the standard of DICOM Q/R SCU Service. From this dialog, you can query studies or retrieve images by accessing to the server which supports DICOM Q/R SCP Service.



- From the **DICOM Q/R** tab, you can query or retrieve studies in **QXLink** server by using the standard way of **DICOM Q/R** service. However, this way is slower than the way of direct access from the database tab in **QXLink** viewer to the database in **QXLink** server. Therefore, it is recommended to use the database tab when you query or retrieve studies. (This way is called as a non-standard way or a direct way.)

3.4.1 Search

1 Select search conditions and enter the search values.






2 Configure the search period.

3 Click on the  button.



- The case-sensitive matching of search conditions may vary depending on the DICOM Server.

3.4.2 Queue List Toolbar

Icon	Name	Description
	Add Queue	Adds the selected study to queue.
	Clear	Deletes selected items from the queue list.
	Clear All	Deletes all items in the queue list.
	Select All	Selects all items in the queue list.
	Retrieve	Retrieves selected studies from the queue list.