



# NT-proBNP

## Fast Test Kit

### (Immunofluorescence Assay)

#### User Manual

**REF** IF1002 for Getein1100  
IF2002 for Getein1600

#### INTENDED USE

NT-proBNP Fast Test Kit (Immunofluorescence Assay) is intended for *in vitro* quantitative determination of N-terminal B-type natriuretic peptide precursor (NT-proBNP) in human serum, plasma or whole blood samples. This test is used as an aid in the clinical diagnosis, prognosis and evaluation of Heart Failure (HF).

#### SUMMARY

N-terminal B-type natriuretic peptide precursor (NT-proBNP) is secreted from the left cardiac ventricle in response to volume and pressure overload. It's an inactive N-terminal fragment that split from BNP prohormone. NT-proBNP can be used to evaluate heart contractile, diastolic dysfunction, and ventricular segmental wall motion coordination. Besides, it has high sensitivity and negative predictive value (>97%). As a gold standard recommended by the European Society of Cardiology, American Heart Association, and American College of Cardiology for the diagnosis and prognosis of heart failure, NT-proBNP is used to indicate heart failure patient at the early stage, determine HF risk levels, monitor medical efficiency of HF drug, evaluate prognosis of HF patient and to distinguish dyspnea that caused by HF from other diseases. Furthermore, NT-proBNP is a risk assessment indicator for Acute Coronary Syndrome.

#### PRINCIPLE

The test uses an anti-human NT-proBNP monoclonal antibody conjugated with fluorescence latex and an anti-human NT-proBNP polyclonal antibody coated on the test line. After the sample has been applied to the test strip, the fluorescence latex-labelled anti-human NT-proBNP monoclonal antibody binds with the NT-proBNP in sample and forms a marked antigen-antibody complex. This complex moves to the test card detection zone by capillary action. Then marked antigen-antibody complex is captured on the test line by the anti-human NT-proBNP polyclonal antibody. The fluorescence intensity of the test line increases in proportion to the amount of

NT-proBNP in sample.

Then insert test card into Getein1100 Immunofluorescence Quantitative Analyzer/Getein1600 Immunofluorescence Quantitative Analyzer (hereinafter referred to as Getein1100 and Getein1600), the concentration of NT-proBNP in sample will be measured and displayed on the screen. The value will be stored in Getein1100/Getein1600 and available for downloading. The result can be easily transmitted to the laboratory or hospital information system.

#### CONTENTS

##### 1. A kit for Getein1100 contains:

- Package specifications: 25 tests/box, 10 tests/box
- 1) NT-proBNP test card in a sealed pouch with desiccant
  - 2) Disposable pipet
  - 3) User manual: 1 piece/box
  - 4) SD card: 1 piece/box
  - 5) Whole blood buffer: 1 bottle/box

##### 2. A kit for Getein1600 contains:

- Package specifications: 2×24 tests/kit, 2×48 tests/kit
- 1) Sealed cartridge with 24/48 Getein NT-proBNP test cards
  - 2) User manual: 1 piece/box
- Materials required for Getein1600:

- 1) Sample diluent: 1 bottle/box
- 2) Box with pipette tips: 96 tips/box
- 3) Mixing plate: 1 piece/box

##### 3. Sample diluent/Whole blood buffer composition:

Phosphate buffered saline, proteins, detergent, preservative, stabilizer.

##### 4. A test card consists of:

A plastic shell and a reagent strip which is composed of a sample pad, nitrocellulose membrane (one end of the membrane is coated with a fluorescence latex-labelled anti-human NT-proBNP monoclonal antibody, the test line is coated with another anti-human NT-proBNP monoclonal antibody and the control line is coated with rabbit anti-mouse IgG antibody), absorbent paper and liner.

**Note:** Components from different batches must not be interchanged.

#### APPLICABLE DEVICE

Getein1100 Immunofluorescence Quantitative Analyzer  
Getein1600 Immunofluorescence Quantitative Analyzer

#### STORAGE AND STABILITY

Store the test card at 4~30°C with a valid period of 24 months.  
Use the test card for Getein1100 within 1 hour once the foil pouch is opened.

For test card of Getein1600: if the cartridge is opened, it could be stable within 24 hours once exposed to air. If the test cards can't be used up at a time, please put the cartridge back to the foil pouch and reseal along the entire edge of zip-seal. The remaining test cards should be used up within 7 days.

Store the sample diluent/whole blood buffer at 0~30°C with a valid period of 24 months.

Store the sample diluent/whole blood buffer at 2~8°C for better results.

#### PRECAUTIONS

1. For *in vitro* diagnostic use only.
2. Do not use the kit beyond the expiration date.
3. Do not use the test card if the foil pouch or the cartridge is damaged.
4. Do not open pouches or the cartridge until ready to perform the test.
5. Do not reuse the test card.
6. Do not reuse the pipet.
7. Handle all specimens as potentially infectious. Proper handling and disposal methods should be followed in accordance with local regulations.
8. Carefully read and follow the manual to ensure proper test performance.

#### SPECIMEN COLLECTION AND PREPARATION

1. This test can be used for **serum, plasma and whole blood samples**. **Heparin and sodium citrate** should be used as the anticoagulant for plasma and whole blood. Samples should be free of hemolysis.
2. Suggest using serum or plasma for better results.
3. Serum or plasma can be used directly. For whole blood sample, one drop of whole blood buffer must be added before testing.
4. If testing is delayed, serum and plasma samples may be stored up to 1 day at 2~8°C or stored at -20°C for 3 months before testing (whole blood sample may be stored up to 3 days at 2~8°C).
5. Refrigerated or frozen sample should reach room temperature and be homogeneous before testing. Avoid multiple freeze-thaw cycles.
6. Do not use heat-inactivated samples.
7. **SAMPLE VOLUME (for Getein1100): 100 µl.**

#### TEST PROCEDURE

1. Collect specimens according to user manual.
2. Test card, sample and reagent should be brought to room temperature before testing.

#### For Getein1100:

- Confirm SD card lot No. in accordance with test kit lot No. Perform "SD card" calibration when necessary.
- Remove the test card from the sealed pouch immediately before use. Label the test card with patient or control identification.
- Put the test card on a clean table, horizontally placed.
- Using sample transfer pipette, deliver **100 µl** of sample into the sample port on the test card (for whole blood sample, one drop of whole blood buffer must be added after loading 100 µl sample on the test card).
- Reaction time: 10 minutes.** Insert the test card into Getein1100 and press "ENT" button or click on "Start" icon (for Android Getein 1100) after reaction time is elapsed. The result will be shown on the screen and printed automatically.

#### For Getein1600:

- Each cartridge for Getein1600 contains a specific RFID card which can calibrate automatically.
- Place the sample diluent at the correct position in Getein16-00.
- Place samples in the designed area of the sample holder, insert the holder and select the right test item, Getein1600 will do the testing and print the result automatically.

#### Notes:

- It is required to perform "SD card" calibration when using a new batch of kits for Getein1100.
- It is suggested to calibrate once for one batch of kits for Getein1100.
- Make sure the test card and the sample insertion is correct and complete.

#### TEST RESULTS

Getein1100/Getein1600 can scan the test card automatically and display the result on the screen. For additional information, please refer to the user manual of Getein1100/Getein1600.

#### EXPECTED VALUE

The expected normal value for NT-proBNP was determined by testing samples from 2,500 apparently healthy individuals. The 95<sup>th</sup> percentile of the concentration for NT-proBNP is 185 pg/ml and the 97.5<sup>th</sup> percentile of the concentration for NT-proBNP is 300 pg/ml. Because of the apparent difference of the concentration of NT-proBNP among different age groups, the reference values of the NT-proBNP are reported in groups. Details refer to Table 1. Clinical diagnosis value: refer to Roche criterion, details see Table 2.

Table 1 NT-proBNP reference value

Age Percentile	≤44	45-54	55-64	65-74	≥75	Statistic analysis
95	98.5	130	215	290	530	185
97.5	116	170	270	350	740	300

Table 2 Standard of excluding/diagnosing HF by NT-proBNP

Age	<50	50-75	≥75	Diagnosis of HF
NT-proBNP (pg/ml)	≥450	≥900	≥1800	High probability of HF
	300-450	300-900	300-1800	Low probability of HF, need to combine with other clinical evaluation
	<300	<300	<300	Exclude HF

It is recommended that each laboratory establish its own expected values for the population it serves.

#### PERFORMANCE CHARACTERISTICS

Measuring Range 100-35000 pg/ml  
 Lower Detection Limit ≤100 pg/ml  
 Within-Run Precision ≤10%  
 Between-Run Precision ≤15%

#### Method Comparison:

The assay was compared with ROCHE E170 and its matching NT-proBNP test kits with 232 serum samples. The correlation coefficient (r) for NT-proBNP is 0.988.

#### LIMITATIONS

- As with all diagnostic tests, a definitive clinical diagnosis should not be made based on the result of a single test. The test results should be interpreted considering all other test results and clinical information such as clinical signs and symptoms.
- Samples containing interferents may influence the results. The table below listed the maximum allowance of these potential interferents.

Interferent	Hemoglobin	Triglyceride	Bilirubin
Concentration (Max)	5 g/L	10 g/L	0.2 g/L

#### REFERENCES










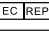



- de Lemos JA, McGuire DK, Drazner MH. B-type natriuretic peptide in cardiovascular disease. *Lancet* 2003; 362:316-322.
- Pfister R, Scholz M, Wielckens K, Erdmann E, Schneider CA. The value of natriuretic peptides NT-pro-BNP and BNP for the assessment of left-ventricular volume and function. A prospective

study of 150 patients. *Deutsche medizinische Wochenschrift* (1946) 2002; 127(49):2605.

- EN ISO 18113-1:2011 In vitro diagnostic medical devices - Information supplied by the manufacturer (labelling) - Part 1: Terms, definitions and general requirements.
- EN ISO 18113-2:2011 In vitro diagnostic medical devices - Information supplied by the manufacturer (labelling) - Part 2: In vitro diagnostic reagents for professional use.

#### DESCRIPTION OF SYMBOLS USED

The following graphical symbols used in or found on NT-proBNP Fast Test Kit (Immunofluorescence Assay) are the most common ones appearing on medical devices and their packaging. They are explained in more details in the European Standard EN ISO 15223 -1:2016/ISO 15223-1:2016.

Key to symbols used			
	Manufacturer		Use-by date
	Do not re-use		Date of manufacture
	Consult instructions for use		Batch code
	Temperature limit		In vitro diagnostic medical device
	Contains sufficient for <n> tests		Authorized representative in the European Community
	CE mark		Do not use if package is damaged
	Catalogue number		

Thank you for purchasing NT-proBNP Fast Test Kit (Immunofluorescence Assay). Please read this user manual carefully before operating to ensure proper use.

Version: WIF03-S-08

 Getein Biotech, Inc.  
 Add: No.9 Bofu Road, Luhe District, Nanjing, 211505, China  
 Tel: +86-25-68568508  
 Fax: +86-25-68568500  
 E-mail: tech@getein.com.cn  
 overseas@getein.com.cn  
 Website: www.bio-gp.com.cn

 Lotus NL B.V.  
 Add: Koningin Julianaplein 10, 1e Verd, 2595AA, The Hague, Netherlands.  
 Tel: +31645171879(English)  
 +31626669008(Dutch)  
 E-mail: peter@lotusnl.com