

HyPort B30

Medical Supply Unit

**Installation & Service Manual**

# Copyright



The device bears CE mark indicating its conformity with the provisions of the Council Directive 93/42/EEC concerning medical devices and fulfills the essential requirements of Annex I of this directive.

The device complies with the requirements of IEC/EN 60601-1-2:2007 “Electromagnetic Compatibility-Medical Electrical Equipment”.

© 2015 Nanjing Mindray Bio-Medical Electronics Co., Ltd. All rights Reserved.

For this Installation Manual, the Issued Date is 2015-08 (Version: 1.0).

## Intellectual Property Statement

NANJING MINDRAY BIO-MEDICAL ELECTRONICS CO., LTD. (hereinafter referred to as Mindray) owns the intellectual property rights to this Mindray device and this manual. This manual may refer to information protected by copyrights or patents and does not convey any license under the patent rights or copyright of Mindray, or of others.

Mindray intends to maintain the contents of this manual as confidential information. Disclosure of the information in this manual in any manner whatsoever without the written permission of Mindray is strictly forbidden.

Release, amendment, reproduction, distribution, rental, adaption and translation or any other derivative work of this manual in any manner whatsoever without the written permission of Mindray is strictly forbidden.



are the trademarks, registered or otherwise of Mindray in China and other countries. **HyPort**, **HyPort** are the trademarks, registered or otherwise, owned by Nanjing Mindray Bio-Medical Electronics Co., Ltd in China and other countries. All other trademarks that appear in this manual are used only for informational or editorial purposes. They are the property of their respective owners.

# 5 Installing Double Pendant

---

## 5.1 Notes

---

### CAUTION

- The installation and commissioning of each part shall follow the steps specified in this chapter. Otherwise, there may be safety risks.
  - Prior to the installation, make sure all electrical connections to the mains supply are cut off!
-

## 5.7 Installing the Covers

1. Place the 2 parts of the rotary ring onto the fixing holders and then use 2 M3×6mm countersunk head screws (1) to fix the two parts together. The rotary rings should be installed on both of the bases.

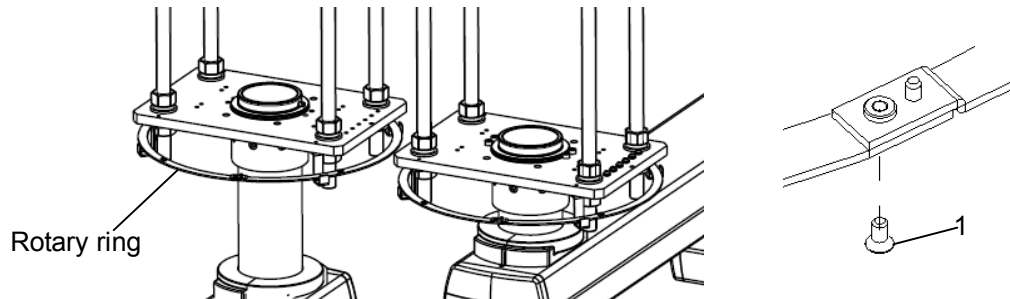


Figure 5-2

2. Hook the 2 parts of the square ceiling cover together and then use 4 M4×12mm countersunk head screws to fix them.


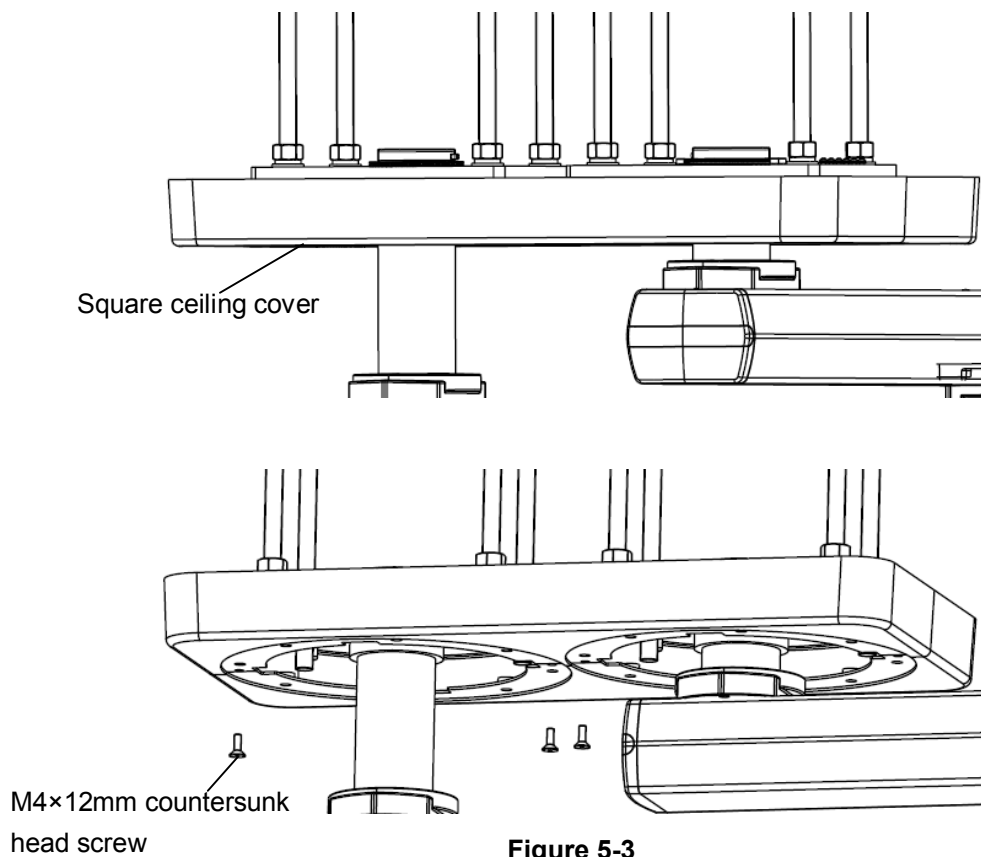



Figure 5-3

3. Install the rubber junction seal onto the square ceiling cover.

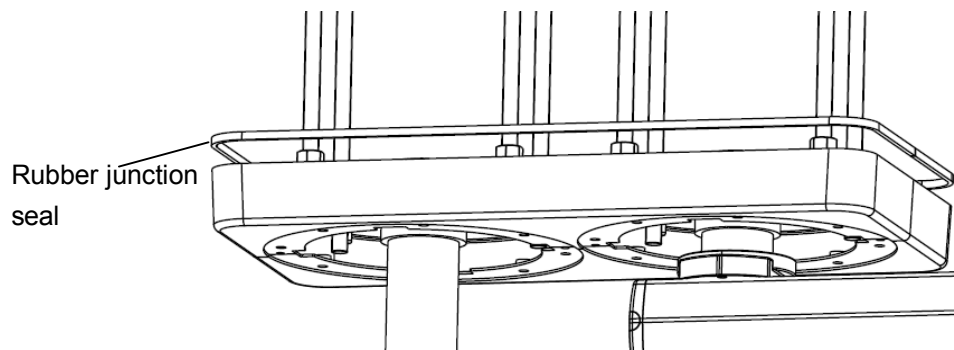


**Figure 5-4**

4. Use 8 M4×35mm pan head screws with washers (2) to fix the square ceiling cover to the rotary ring and then tighten the screws. Make sure the ceiling cover is fully engaged with the false ceiling.

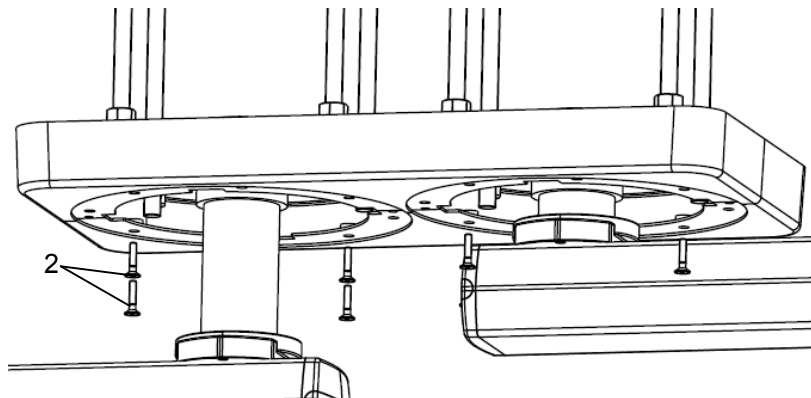


**Figure 5-5**

5. Hook the 2 parts of the round decorative cover (3) together and then screw it onto the ceiling cover through the grooves. Make sure the joint of the square ceiling cover is aligned with that of the round decorative cover. Install the other round decorative cover in the same way.

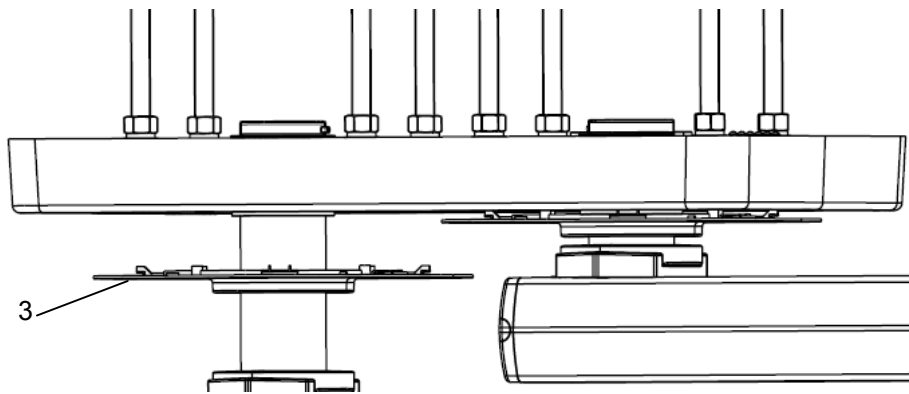

**Figure 5-6**

6. The procedures of selecting the ceiling cover's length and rest installation are consistent with that of the single pendant. For the rest installation procedure, see chapter 3.7 for details.

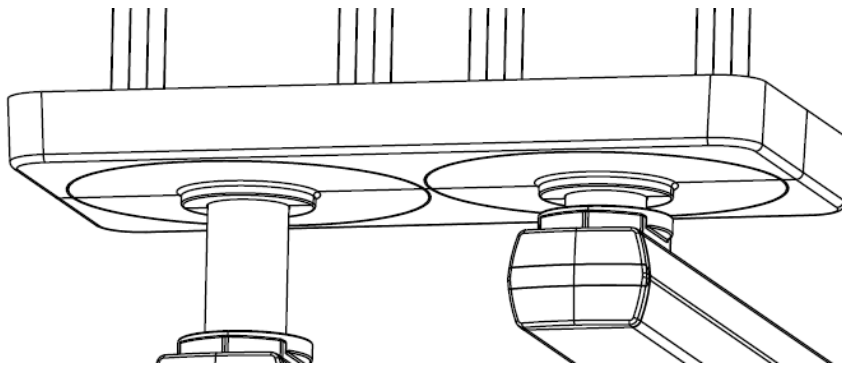

**Figure 5-7**