



Product Specification of:

Medical Oxygen Sensor

Type: M-07

Part Number: 410009

Nuova ID: E-77/0

RoHS compliant and SVHC free

Meets the applicable requirements of ISO 80601-2-55

Comes with CE marking. further regulatory registrations available upon request

Produced under EN ISO 13485 Quality Management System

DOCUMENT PURPOSE

The purpose of this document is to present the performance specification and key features of the sensor.

This document should be used in conjunction with the Operating Manual of the instrument and the Product Safety Data Sheet of the sensor.

KEY FEATURES

Long-life galvanic gas sensor with high signal stability at low cross sensitivity to anesthesia gases combined with superior linearity over the entire measurement range.







Superior Linearity



Wide Measurement Range



Separate Temperature Compensation Network

This data sheet is subject to change without prior notice

Specification_M-07_Rev_3

Page 1 of 5

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany



Type: M-07

Part Number: 410009

Nuova-ID: E-77/0



TECHNICAL SPECIFICATIONS

		-			_		_		-
ΝЛ	EA			v	ы	٧Л		м	
IVI	-	101	u	г		VΙ	_	w	

Operating Principle:	Partial pressure electrochemical cell	
Measurement Range:	0 100 Vol.%O ₂	
Initial Output Signal:	7.5 13.0 mV	at dry ambient air
Response Time t90:	< 12 s	
Signal Drift (long-term):	≤ ± 1	at ambient air
Linearity Error:	≤ ± 3	at 100 Vol.%O ₂ applied for 5 min
Zero Signal Offset:	≤ 0.3 Vol.%O ₂	at 100 Vol.%N ₂ applied for 5 min
Repeatability Error:	≤ ± 1 Vol.%O ₂	at 100 Vol.%O ₂ applied for 5 min
Influence of Gas Humidity:	- 0.03 % of signal per %rH	
Signale Temperature Compensation	NTC on sensor PCB	
Cross-Sensitivity:	fullfills DIN EN ISO 80601-2-55	
ELECTRICAL		
Electrical Connector	4P4C Handset Modular Jack 4 Position (RJ11), Type AMP	
Recommended Load Resistor:	≥ 10 kOhm	
MECHANICAL		
MECHANICAL Weight:	≤ 28 g	
	≤ 28 g PA, PPS, PTFE, Stainless Steel NBR	

ENVIRONMENTAL

Operating Temperature Range:	10 40 °C	
Ambient Pressure Range:	700 1250 hPa	
Ambient Humidity Range:	up to 100 %rH	non-condensing

LIFETIME

Expected Operating Life:	3 years	at ambient air, depending on application
Nominal Sensor Life:	500 000 Vol.%h O ₂	at ambient air, depending on application

Important Note: All characteristics are based on conditions at 25 °C, 50 %rH, 1013 hPa and a gas flow of > 2.5 L_s/min. For sensor performance data under other conditions, contact ITG.

This data sheet is subject to change without prior notice

Specification_M-07_Rev_3

Page 2 of 5

IT Dr. Gambert GmbH :: Hinter dem Chor 21 :: 23966 Wismar :: Germany



Type: M-07

Part Number: 410009

Nuova-ID: E-77/0



STORAGE CONDITIONS IN UNOPENED ORIGINAL PACKAGE

Ambient Temperature Range: 15 ... 25 °C recommended

 $-20 \dots 50$ °C maximum (≤ 10 h)

Ambient Pressure Range: 700 ... 1250 hPa

Ambient Humidity Range: 50 ... 100 %rH recommended non-condensing

0 ... 30 %rH maximum one week

RELATED PRODUCTS

Product	Part Number	Other Specifics	
O ₂ Sensor M-07	41 00 09	-	
O ₂ Sensor M-07T	47 00 12	translucent housing	
O ₂ Sensor M-07S	41 01 09		
O ₂ Sensor M-07ST	41 02 09	translucent housing	

Cleaning and Disinfection

The sensor outer housing can be cleaned with a dry wipe. Do not wipe the sensor's gas entrance side. Do not use any chemical disinfectant or sanitizer on the sensor.

Poisoning

ITG sensors are designed to operate in a wide range of environments. For optimal sensor lifetime and performance it is important that exposure to high concentrations of solvent vapors is avoided during storage, installation into instruments and operation of the sensor. Do not use adhesives directly on or near the sensor as the solvents may cause stress corrosion on the plastic parts.

Intended Use

The electrochemical oxygen sensors for use in medical technology are used as accessories with a limited shelf life exclusively in conjunction with other medical products (ventilators, incubators, anesthesia machines, portable oxygen monitors, oxygen therapy devices and gas mixing units) so that they can fulfill the manufacturer's intended purpose.

The oxygen sensors measure the oxygen partial pressure in gas mixtures.

Commissioning and use is carried out exclusively by expert clinic or service personnel. Direct contact with these items occurs during installation or maintenance by trained personnel.

Stabilisation Time

When installing a new sensor refer to the instrument manual for stabilization time before calibration.

If not specified otherwise wait at least 15 minutes to ensure the sensor has stabilized in the instrument.

Calibration Interval

ITG sensors are designed to have minimal signal drift over their functional lifetime. For optimal performance and maximum measurement accuracy however they should be calibrated before each use.

This data sheet is subject to change without prior notice

Specification_M-07_Rev_3

Page 3 of 5

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany



Type: M-07

Part Number: 410009

Nuova-ID: E-77/0



If the Sensor is dropped

If a sensor is dropped, please check for visible mechanical damage of the sensor or if the sensor is leaking electrolyte. If this is the case, do not try to install the sensor into the device. Take safety precautions and immediately dispose the cell.

If the sensor shows no visible damage it should be placed in quarantine for at least 24 hours. Afterwards a follow-up check made by a two point calibration should be done.

Installation in Device

A gas tight sealing is ensured when the sensor is screwed in hand-tight. Don't use any mechanical tools to install the sensor.

Using excessive force may damage the sensor.

Optimal mounting position of the sensor is when the gas sensing area faces downwards. A horizontal position is acceptable. It is not recommended to use the sensor with the gas sensing area facing upward.

Connection should be made via recommended electrical and mechanical connectors only. The specified load resistance must be taken into account. Soldering the sensor will damage it and void the warranty. Please contact ITG for further information.

Since temperature has an influence on the output signal it is not advisable to place any heat or vibration sources (i.e. electric- pumps or valves, coolers, etc.) in close proximity to the cell.

Avoid proximity of the cell to any EMC radiating units as those emit frequencies which might interfere with the sensor's electrical connection and connecting cable.

RFI/EMI Susceptibility

ITG sensors contain metal parts and might be susceptible to RFI or EMI. Before use in MRI environments please contact ITG for further information.

Disposal

At the end of the sensors lifetime the sensor should not be disposed of in normal public waste as it may contain hazardous materials and caustic electrolyte (for more information refer to the PSDS). Please contact your local authorities for environmental legislation to relevant local waste disposal.

This data sheet is subject to change without prior notice

Specification_M-07_Rev_3

Page 4 of 5

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany



Type: M-07

Part Number: 410009

Nuova-ID: E-77/0



TECHNICAL DRWAING


Sensor	ITG	Color			Electrical pin assignment			
	Part number	Сар	Housing	Threaded ring	Pin 1	Pin 2	Pin 3	Pin 4
M-07	410009	white	white	white	nc	plus (+)	minus (-)	NTC (+)

Top view 1 2 3 4 Electrical connector: 4P4C Modular Jack Type AMP

Ø 11 gas inlet diameter

Bottom view

Front view

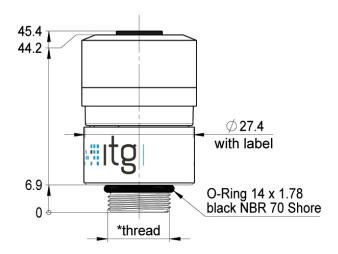

Dimension unit: mm Dimension tolerances: linear ±0.5 mm , diameter ±0.3 mm

This data sheet is subject to change without prior notice

Specification_M-07_Rev_3

Page 5 of 5

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany