

# SILICONE NASAL CPAP MASK



## USER MANUEL KULLANMA KILAVUZU



www.kngmed.com & info@kngmed.com

ENG 1



### ➤ Intended Use:

Nasal Masks ( *active & fix* ) are designed for the use of adult patients for whom the CPAP, Bi-PAP therapy is prescribed by a healthcare professional or respiratory therapist. The Nasal Masks are used with the CPAP, Bi-PAP, Auto-PAP therapy devices for single patient use in home or multi-patient use in medical centers.

### Remember!

An exhalation port is built into the Nasal Masks. Therefore, a separate exhalation port is not required.

### Remember!

Do not block vent holes and also exhalation ports of the mask.



### ➤ Before First Use:

- ✓ Read and understand the instructions.
- ✓ Remove all packaging before using the Nasal Masks.
- ✓ Hand wash mask.
- ✓ Inspect all parts of your Nasal Masks for damage; replace that are visibly deteriorated due to cracking, crazing, tears, etc.
- ✓ Clean patient's face.
- ✓ Always keep the vent holes clear and open.




✓ Do not wear this Nasal Mask unless the CPAP system is turned on and operating properly.

## Nasal CPAP Masks Information & Symbols

www.kngmed.com & info@kngmed.com



Keep away from sunlight.

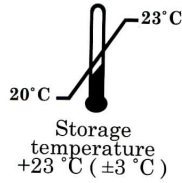

Storage temperature  
+23 °C (±3 °C)



Warning & Cautions



Consult instructions for use



LOT, REF numbers, production and expiration dates on the box.

2 ENG



### Warnings & Precautions:



- ✓ Discontinue use and contact your healthcare professional, if skin redness, irritation or discomfort occurs while using nasal or full-face mask.
- ✓ Discontinue using Nasal Masks if you have any adverse reaction, and consult your physician or sleep therapist.
- ✓ This mask should not be used in the presence of impaired cardiac sphincter function, excessive reflux, impaired cough reflex, and hiatal hernia and if the patient is uncooperative, obtunded, unresponsive, or unable to remove the mask by themselves.




- ✓ The Nasal Masks should fit comfortably. Verify that the mask is the correct size.



- ✓ Oxygen supports combustion. Oxygen should not be used while smoking or in the presence of an open flame.



- ✓ At low PAP pressures the flow through the exhalation port may not be poured out completely and rebreathing may occur.

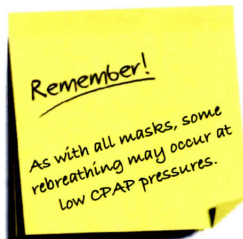

- ✓ If oxygen is used with device, the oxygen flow must be turned off when the device is not operating. Otherwise, Oxygen accumulated in the device enclosure will create a risk of fire.



- ✓ Do not use the masks, If you are using a pill that potentially causes vomiting.

4 ENG



## ➤ Step by step usage directions

www.kngmed.com & info@kngmed.com



**Step 1.** Before fitting your mask, remove the tubing connector from the mask by pressing the side clips. Connect the tubing connector to air tubing.



**Step 4.** Attach the headgear clips on the cheek.



**Step 2.** Before fitting your mask, attach the hook and loop fastener on the forehead.



**Step 5.** Fasten the headgear clips on the cheek and forehead.



**Step 3.** Hold the mask against your face and pull the headgear over your head.



**Step 6.** Adjust the mask to minimize air leakage as comfortable as possible.



ENG 6



## ➤ Technical Details:

Therapy Pressure	: 4 cmH <sub>2</sub> O to 20 cmH <sub>2</sub> O
Operating Temperature	: +41°F to 104°F (+5°C to +40°C)
Operating Humidity	: 15% to 95% non-condensing

### Limited Warranty:


KNGMED warrants that your nasal mask systems (including mask frame, silicone cushion, headgear and tubing) shall be from defects in material and workmanship from the date of purchase for the period of 90 days. This warranty is only available to the initial consumer. It is not transferable.

This limited warranty does not cover:

- any damage caused as a result of improper use, abuse, modification or alteration of the product,
- repairs carried out by any service organisation that has not been expressly authorized by KNGMED to perform such repairs,
- any damage or contamination due to cigarette, pipe, cigar or other smoke.

Warranty replaces all other expressed or implied warranties, including any implied warranty of merchantability or fitness for a particular purpose.

KNGMED shall not be responsible for any incidental or consequential damages claimed to have resulted from the sale, installation or use of any KNGMED product.

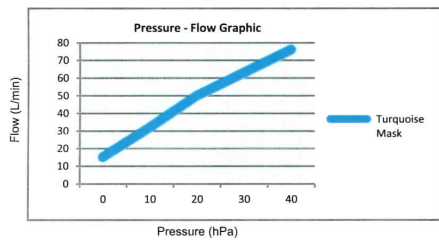

KNGMED Medical Co. Ltd.  
10007 Sok. No:26/B Ulukent,  
Menemen Izmir TURKIYE  
Phn: +90 232 833 42 62  
Fax: +90 232 833 42 63  
[www.kngmed.com](http://www.kngmed.com)  
[info@kngmed.com](mailto:info@kngmed.com)  
[kng.info@yahoo.com](mailto:kng.info@yahoo.com)

IFU.TE 7.8.1/Rev.01/08.06.2020



ENG 7

www.kngmed.com & info@kngmed.com



### ➤ Cleaning Instructions:

#### Cleaning the Mask:

- ✓ Clean the nasal masks before first use and daily.
- ✓ The headgear does not need to be removed for daily cleaning of the nasal masks.
- ✓ Hand wash in warm water with a mild dishwashing detergent.
- ✓ All components should be rinsed well with drinking quality water and allowed to air dry out of direct sunlight.

**Remember!**  
Do not use bleach, alcohol or cleaning solutions containing conditioners, moisturizers or alcohol.

#### Cleaning the headgear:

- ✓ The headgear should be wash weekly or as needed. The headgear does not need to be removed for daily cleaning of the mask.

**Remember!**  
\*Do not place the headgear into the dryer.  
\*Do not use autoclaving or disinfection liquids to clean headgear.



#### For Multi-patient use in the Hospital:

- For disinfection of the masks, can be use autoclaving (134°C and 3 min.)

#### Disposal:

- Dispose of in accordance with local regulations for medical waste.