

OZONOSAN[®] alpha plus

OPERATOR'S MANUAL

English edition



German issue completed 01/2018
This English issue completed 01/2018

Subject to technical changes

Dr. J. Hänsler GmbH, Nordring 8, 76473 Iffezheim Tel. 0049-07229 – 3046-0 Fax - 3046-30
www.ozonosan.de info@ozonosan.de

5.11.2 Accessories for local applications

Article	Art. No.
Local ozone gas application	
Plastic bag for limb gassing 300 x 600mm 23 ³ / ₅ x 11 ⁴ / ₅ ",	2001
Plastic bag for limb gassing 400 x 1200mm 47 ¹ / ₄ x 15 ³ / ₄ "	2002
Connection line 140 cm	3154
Velcro strap 25 x 350mm	2007
Low pressure ozone gas application	
Bell shaped suction vessel 60mm [2,36"] in diameter	3029
Ozone resistant disposable PVC tubes (internal diameter 6 mm [1/4"])	1126

5.11.3 Cleaning and disinfection of recommended accessories

NOTE

All parts coming into direct contact with the patient are to be disposable. This is with the exception of the low-pressure (subatmospheric) suction cups for topical ozone gas application.

First clean these with a mild detergent suitable for plastics before disinfection; we recommend disinfectants in accordance with the continuously updated list (in: "Guidelines for Testing Chemical Disinfectants"*) prepared by the German Society for Hygiene and Microbiology** as being suitable agents for this purpose***.

* *Richtlinien für die Prüfung chemischer Desinfektionsmittel*

** *Deutsche Gesellschaft für Hygiene und Mikrobiologie*

*** published by: *mhp-Verlag GmbH*, Wilhelmstrasse 42, D-65185 Wiesbaden, Germany)