

## Ultra Vac Set Closed Wound Suction Unit (Hemosuc)

**Code No.: 9007**

### 1- Description:

- Suitable for close wound drainage under negative pressure post operatively with the option to operate one or two catheter simultaneously.



### 2- Advantages & Features:

- Graduated Bellow chamber.
- Bellow unit with connector.
- Connecting tube with clamp.
- Curved trocar needle with catheter.
- Spare perforated Redon drain catheter.

### 3- Indications:

- Drainage or collection of fluid from surgical wound.

### 4- Contraindications:

- Not to be used in patients with known hypersensitivity to any of the materials used.

### 5- The purpose:

- Suitable for close wound drainage under negative pressure post operatively with the option to operate one or two catheter simultaneously.

## 6- Instruction for Use:

# ULTRAVAC SET

## CLOSE WOUND SUCTION UNIT

**ULTRAVAC SET** has been specially designed to offer the Medical Fraternity an effective device for close wound drainage under negative pressure with the option to use one catheter or two catheters simultaneously.

### UNIT CONSISTS OF :

1. Bellow unit with connector and hanger.
  2. Connecting tube with clamp and "Y" connector.
  3. Curved needle with matching catheter.
  4. Spare perforated catheter.
- ( Catheter sizes: FG 10 , 12 , 14 , 16 , 18 )

### INSTRUCTION FOR USE :

1. (a) Place the perforated tube in the wound with the help of curved needle , as ( Fig 1&2 ) Ensure that the entair perforated part of the catheter is well within the wound to avoid any air leakage during drainage.

(b) The length of the perforation should be reduced according to the size of the wound.

(c) Catheter is marked 5 cm from the last perforation to ascertain the depth of catheter in the wound.

2- Press the bellow container with both hands. (Fig. 3)

3- Connect the connecting tube ( duly clamped off ) to the bellow container. (Fig. 4)

4- Connect the perforated catheter with connecting tube (Fig. 5) and start drainage by releasing the clamp.

5- To re-charge the vaccum, start operation repeating instructions (2,3,4).

### FOR USE OF TWO CATHETERS SIMULTANEOUSLY

Remove protector cap provided at the "Y" connector and connect the spare perforated catheter with the "Y"connector as shown in (Fig. 6) and follow the rest of the directions as given above.


**STERILE EO**



**PYROGEN FREE**

**ECREP**  
Obelis SA  
1030 Brussels - Belgium  
www.obelis.net

**CE**  
0120

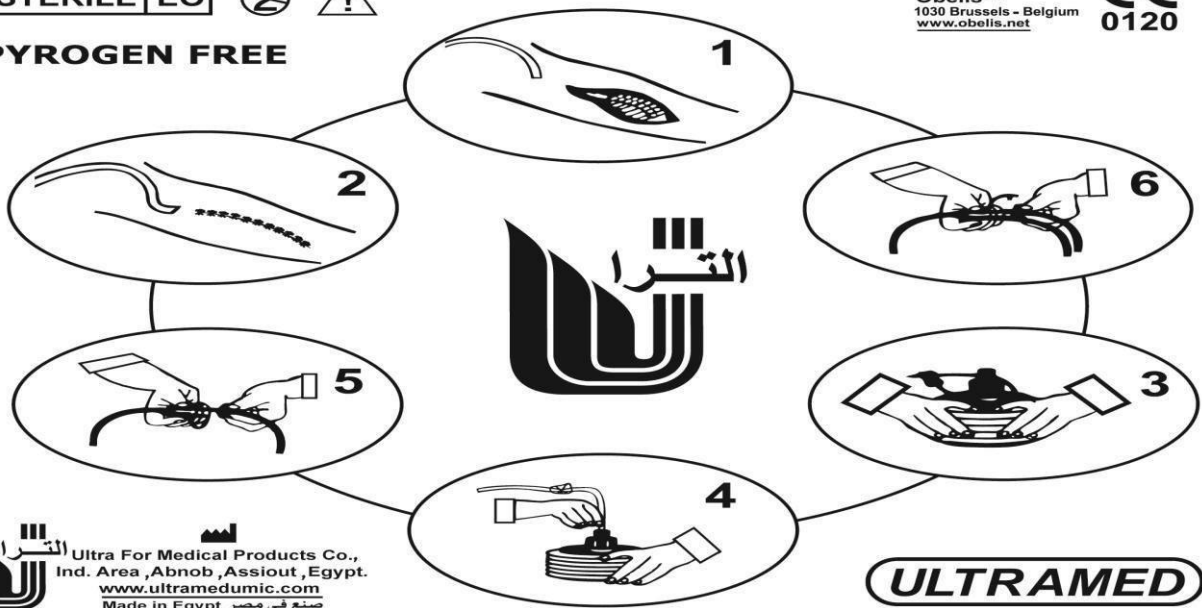

## 7- Care instructions:

- Do not use if package is open or damaged.
- Discard after single use.

## 8- Storage conditions:

- Not more than 5 cartoons on each other.
- Nice Ventilated place.

**ULTRA FOR MEDICAL PRODUCTS CO.**

- Out of Sun light.
- Room Temperature.

**9- Product code with size and measures:**

- Code No.: 9007.
- Size of Bellow Chamber: 600 ml, 400 ml.
- Sizes of Accessories: 10 FG to 18 FG .

Code	Size
9007-10	10
9007-12	12
9007-14	14
9007-16	16
9007-18	18

**10- List of product components:**

- Graduated Bellow chamber - Outer Cap - Inner Cap - Hanger - tube (Phthalate free and DEHP free) with clamp & connector - U-Connector - Gauge Connector & Gauge Closer - Curved trocar needle with catheter - Spare perforated Redon drain catheter .

**11- Sterility:**

- Sterilized by Ethylene oxide Gas (20% ETO +80 % Carbene dioxide) .

**12- Shelf life: 5 Years**

**Assiut Factory:** Part No. 304, 305, 306, 307, 308, 309, 310, 312, Arab El Awamer- industrial zone, Abnoub, Assiut, Egypt.. **Tel:** 002-088-4964333 (500), 002-088-4964666 (600), 002-088-4964777 (700), 002-088-4964888 (800), 002-088-4964999 (900)

**& Fax:** 002-088-4964222 **& Mob:** 002-01001558853, 002-01068832355

**Cairo Head Office:** 64, Nakhla El Motaay Triumph Square Heliopolis **Tel:** 022/4171621-4143794 **& Fax:** 022/4171613 **& Mob:** 01223988200

**Assiut Office:** 23, July Str. **Tel:** 088/2364111 – 2364222 **& Fax:** 088/2334964 **& Mob. :** 01223988202

**Alexandria Office:** 212 Abd El salaam Aref Str, Luran **Tel:** 03/5856202– 5856458 **& Fax:** 03/5828988 **& Mob:** 01223948666

**E-mail:** [ultra@elaggargroup.com](mailto:ultra@elaggargroup.com), [shady@elaggargroup.com](mailto:shady@elaggargroup.com)

**Website:** [www.ultramedumic.com](http://www.ultramedumic.com)