

Mucus Extractor

Code No.: 4004

1- Description:

- Mucus Extractor is designed for aspiration of secretion from oropharynx for microbiological examination mucus specimen.



2- Advantages & Features:

- Separate screw top lid for safe sealing and handling of mucus.
- Soft, frosted & kink resistant tubing (Phthalate free and DEHP free).
- Available with thumb control suction catheter.
- Transparent Container (Capacity: 25 ml.).
- Suction tube Length: 40 cm.
- Suction tube size: 10 FG.

3- Indications:

- To clear the secretion from the oral cavity.

4- Contraindications:

- Can damage the soft tissue of nasal route.
- It may cause sinusitis, esophagitis, gastro-ulceration and pulmonary infections.

5- The purpose:

- Mucus Extractor is designed for aspiration of secretion from oropharynx for microbiological examination mucus specimen.
- For single use.

6- Instruction for Use:

- Connect the funnel (connector) the suction machine.
- Put the plain tube into the mouth patient and suck the secretion from the funnel side.

**ULTRA FOR MEDICAL PRODUCTS CO.**

- The secretion is collected into the container.
- Put the plain tubing into the nostril of the patient and suck the secretion.
- Remove the tubing from the patient. Remove the lid of container.
- Fix the other lid supplied with the packing & send the container for the laboratory testing.

7- Care instructions:

- Do not use if package is open or damaged.
- Discard after single use.

8- Storage conditions:

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Room Temperature.

9- Product code with size and measures:

- Code No.: 4004
- Mucus Extractor with bacterial barrier filter at the suction end with code number 4004-01.
- Mucus Extractor without bacterial barrier filter at the suction end with code number 4004-02.
- Suction tube size: 6 FG to 14 FG.

10- List of product components:

- Cylinder (Capacity 25 ml) – tubing (Phthalate free and DEHP free) - Funnel (connector) – Top Cap –Bottom Cap

11- Sterility:

- Sterilized by Ethylene oxide Gas (20% ETO +80 % Carbone dioxide).

12- Shelf life: 5 Years.

Assiut Factory: Part No. 304, 305, 306, 307, 308, 309, 310, 312, Arab El Awamer- industrial zone, Abnoub, Assiut, Egypt.. **Tel:** 002-088-4964333 (500), 002-088-4964666 (600), 002-088-4964777 (700), 002-088-4964888 (800), 002-088-4964999 (900)

& Fax: 002-088-4964222 **& Mob:** 002-01001558853, 002-01068832355

Cairo Head Office: 64, Nakhla El Motaiy Triumph Square Heliopolis **Tel:** 022/4171621-4143794 **& Fax:** 022/4171613 **& Mob:** 01223988200

Assiut Office: 23, July Str. **Tel:** 088/2364111 – 2364222 **& Fax:** 088/2334964 **& Mob. :** 01223988202

Alexandria Office: 212Abd El salaam Airef Str, Luran **Tel:** 03/5856202– 5856458 **& Fax:** 03/58289888 **Mob:** 01223948666

E-mail: ultra@elaggargroup.com, shady@elaggargroup.com

Website: www.ultramedumic.com