TEHNOMEDICA

str.Ciuflea, 38/1 MD-2001, mun. Chișinău, Moldova tel./fax: (022)601 102, 601 087
e-mail <tehnomedica_md@yahoo.com> <tehnomedicamd@gmail.com>

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REFERENCES
NACHWEISE

For the references 1–9 of the safety and performance claims on Neuro-Patch®, please visit our website or use the QR code:

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<https://www.bbraun.com/en/products/b1/neuro-patch.html>

For the references 10–14 of the safety and performance claims on Lyoplant®, please visit our website or use the QR code:

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MODE OF APPLICATION

To achieve tension-free embedding, Neuro-Patch® and Lyoplant® should be cut as closely as possible to the defect size.

Neuro-Patch® and Lyoplant® should be fixed with non-absorbable suture material (polyester, polypropylene). Additionally, it may be secured with fibrin glue.

Atraumatic round-bodied needles allow suturing without significant damage to the implant.

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The main product trademark “Aesculap” and the product trademarks “Lyoplant” and “Neuro-Patch” are registered trademarks of Aesculap AG.

Subject to technical changes. All rights reserved. This brochure may only be used for the exclusive purpose of obtaining information about our products. Reproduction in any form partial or otherwise is not permitted.

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PRODUCT CHARACTERISTICS

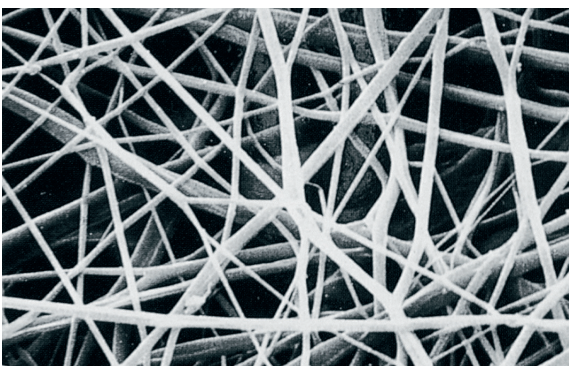
- High liquid tightness of the implant prevents CSF leakages ^{1,2,3,10}
- Tissue tolerant ^{4,5,11,12}
- High tensile strength of the implant prevents suture pull out ^{2,6,12,13}
- Good suturability ^{2,10,13}
- Adaptable to anatomical structures ^{1,2,7,10,14}

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AESCULAP® Neuro-Patch®

SYNTHETIC, SUTURABLE DURA SUBSTITUTION
SYNTHETISCHER, VERNÄHBARER DURAERSATZ

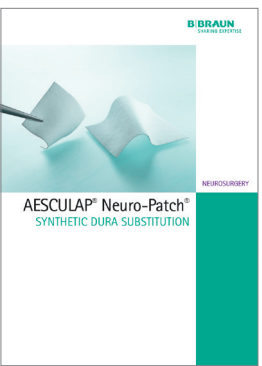
ELASTIC. WELL-TOLERATED. NON-ABSORBABLE. ^{4,5,8,9}

Neuro-Patch® is a type of fine fibred microporous fleece manufactured from a highly purified polyesterurethane.

An electronically controlled procedure has been developed for producing Neuro-Patch®, in which a dissolved polyesterurethane polymer is sprayed from specialised jets. This produces fibres which are captured at defined angles, producing a fleece-like structure.

The fine fibred microstructure supports the rapid infiltration of connective tissue cells. ^{5,9}

C87802

Product sample card including a product sample.

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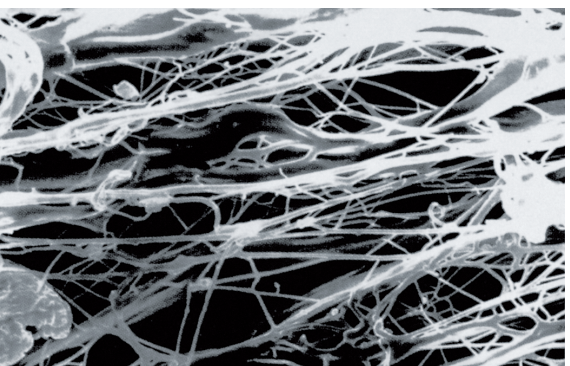
BIOLOGICAL, SUTURABLE DURA SUBSTITUTION
BIOLOGISCHER, VERNÄHBARER DURAERSATZ

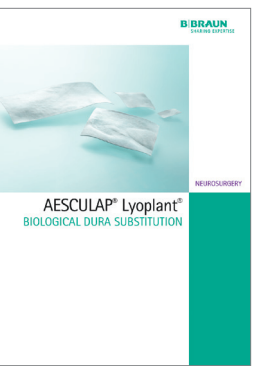
FUNCTIONAL. TEARPROOF. ABSORBABLE. ^{5,10,11,12,13}

Lyoplant® is a pure collagen implant that is produced from bovine pericardium. This highly purified collagen is known for its low propensity to cause immunological reactions. ^{11,12,13}

The special and comprehensive manufacturing process eliminates all non-collagenous substances such as fats, enzymes and other non-collagenous proteins.

A controlled lyophilisation (freeze drying) process supports that the loose fibre structure of Lyoplant® is preserved to offer appropriate conditions for integration after implantation.

After implantation, Lyoplant® slowly degrades enzymatically and is substituted by body's own connective tissue.

C87902

Product sample card including a product sample.

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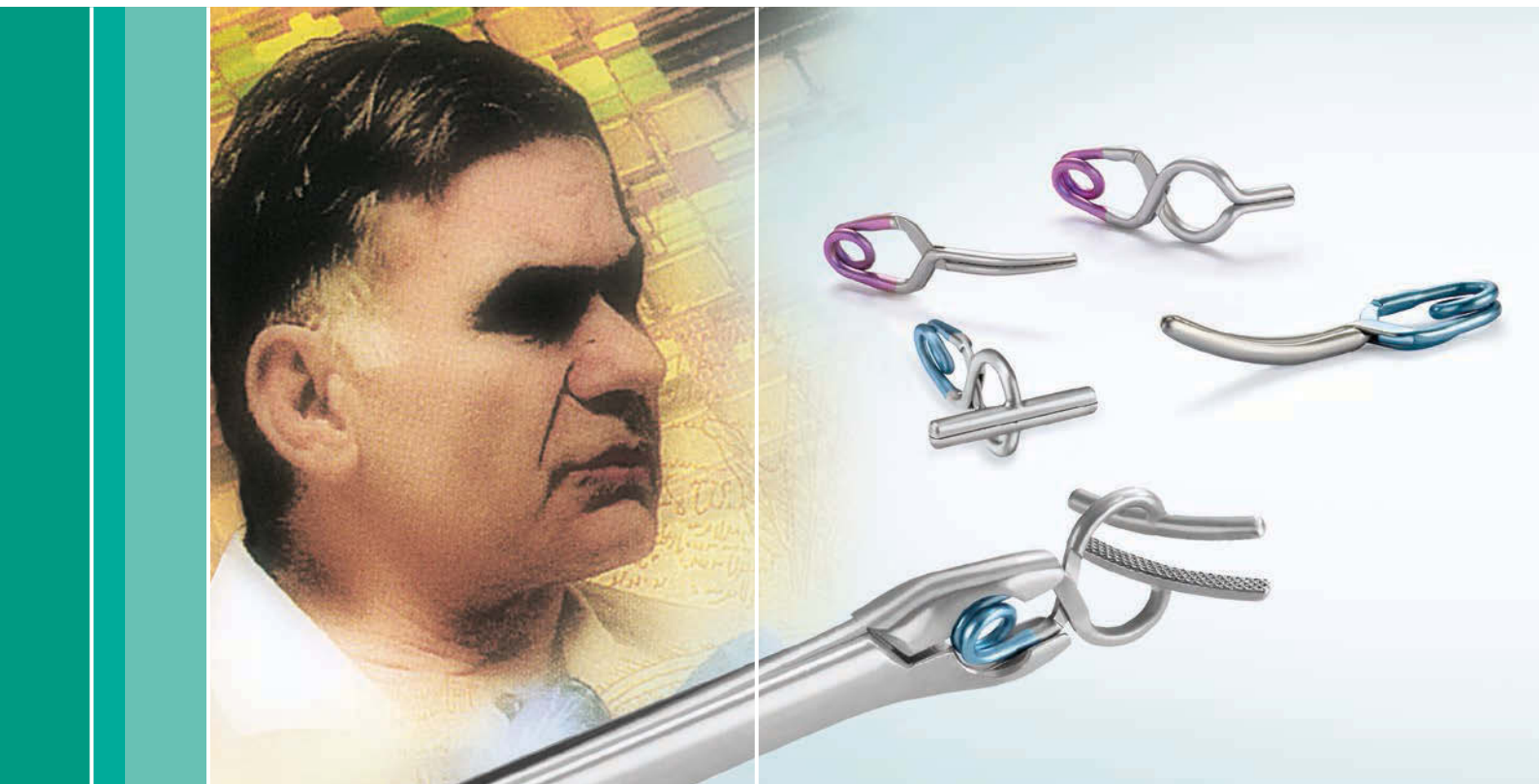
Aesculap Neurosurgery

1968

- First generation YASARGIL Aneurysm Clip made from implant steel with a half spring coil and no jaw guidance

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1970

- YASARGIL Aneurysm Clip with 1.5 spring coil and safety ring to enhance closing force and stability

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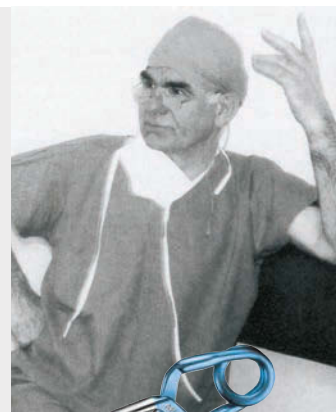
1983

- The YASARGIL Aneurysm Clip made from **Phynox** increases MRI compatibility. A patented box lock improves the characteristics with regard to the "scissoring" effect

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1995

- The YASARGIL Aneurysm Clip with 1.5 spring coil made from **Titanium** improves postoperative control and diagnosis by reducing the number of artifacts in CT and MRI scans

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From experienced hands... for experienced hands!

How can a surgeon be sure that the instruments he is using for lifesaving aneurysm surgery deserve the trust his patients place in him?

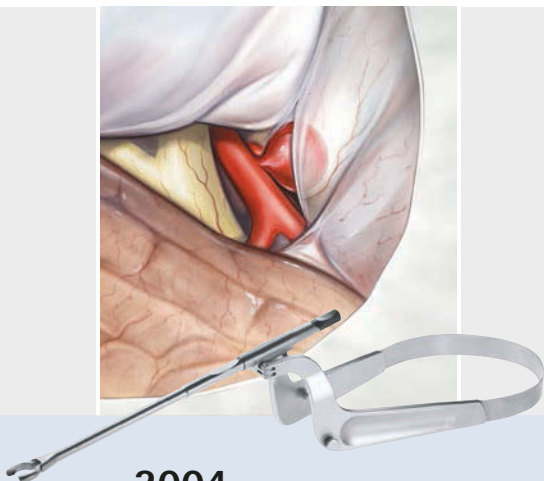
The beginning of the success is based on the development of the clip design by the famous Prof. Gazi M. YASARGIL.

Then decades of intensive cooperation with other leading neurosurgeons all over the world form the foundation for Aesculap's technical and scientific experience.

With the YASARGIL Aneurysm Clips Aesculap is the world market leader. Since their introduction, over 2.0 million clips have been implanted by thousands of neurosurgeons in more than 80 countries. A milestone in the history of neurosurgery.

The surgical treatment of cerebral aneurysms will continue to represent an important and indispensable therapy option into the future.

Through continuous further development of the YASARGIL Aneurysm Clip system, Aesculap is committed to offer implants and instruments of high quality for this treatment.

2004

- The XS Clip Applier meets the demands of keyhole surgery

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2006

- T-Bar Clips acc. to KOPITNIK made from Titanium and Phynox aids and expands treatment possibilities

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2013

- First Mini clips with fenestration allow an even preciser reconstruction of the vascular structures also in narrow spaces

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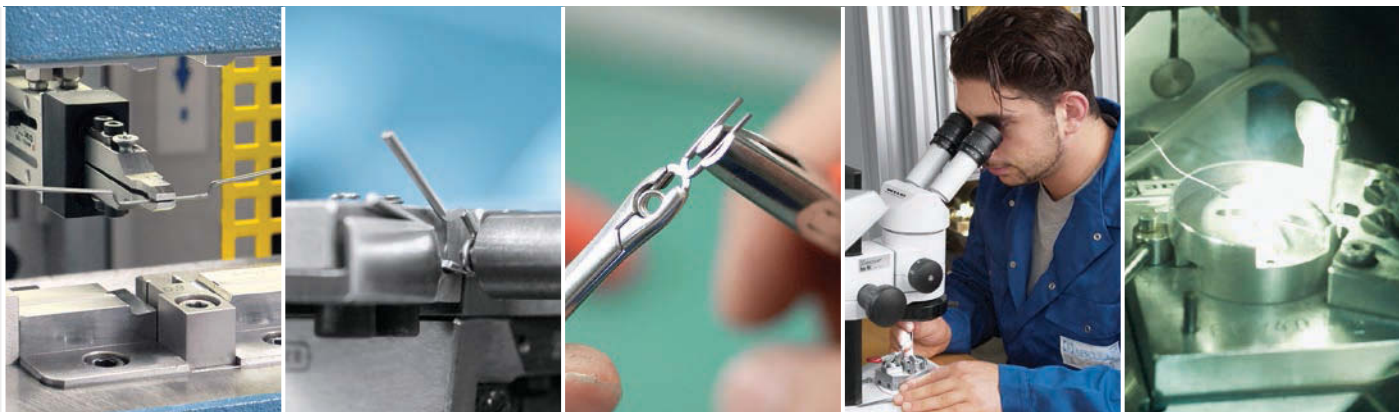
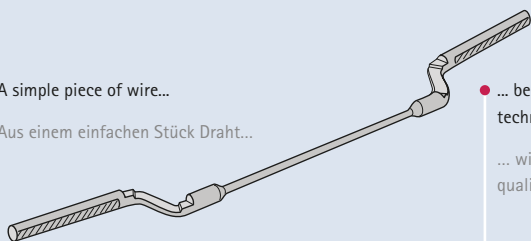
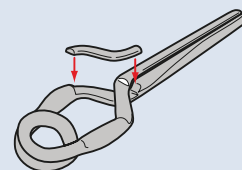
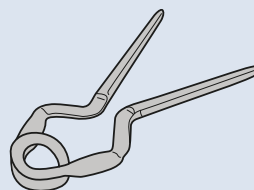
• A simple piece of wire...

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• ... becomes, thanks to advanced high technology and qualified manual work,...

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Our striving for perfection is your safeguard

Precision manual work and the most advanced high technology are intricately and accurately combined in the manufacture of YASARGIL Aneurysm Clips.

Despite the high technology and mechanization, it is only possible to achieve the required accuracy within tolerance and the necessary variability manually in the hands of highly qualified production engineers. This is the only way we can approach the goal of our own quality standards for aneurysm clips, with a smooth, burr-free and therefore atraumatic surface, exact dimensions and a reproducible closing action – a clip you can rely on.

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REM image
atraumatic tip of a YASARGIL Clip from Aesculap;
smooth surface prevents damage to tissue.

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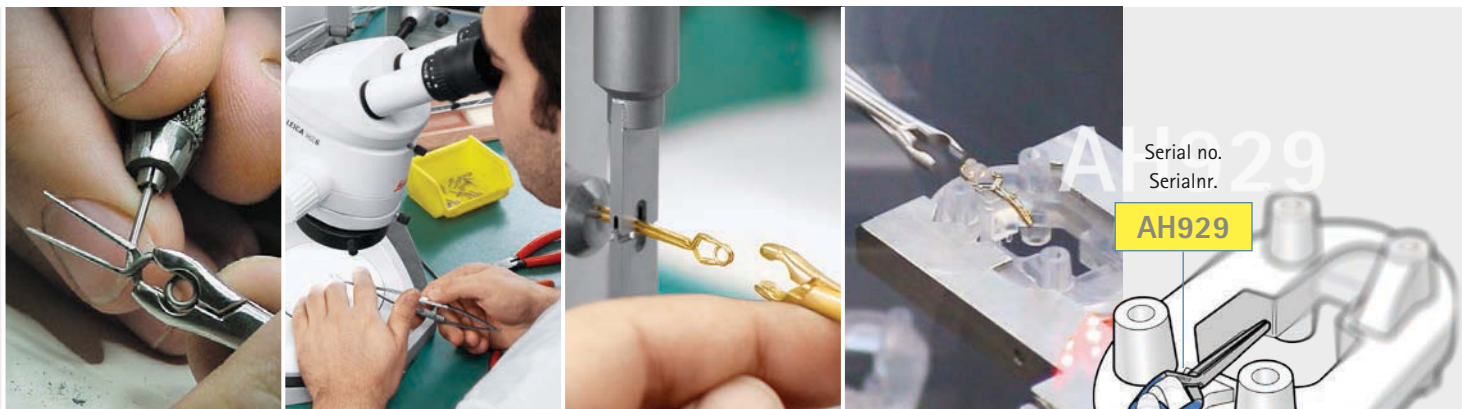
REM image
atraumatic blade profile of a YASARGIL Clip from
Aesculap; the deep, symmetrical, pyramidal stamping
of both blades doubles the contact surface area and
counteracts the slip effect.

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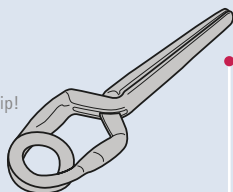
REM image
unique box lock of a YASARGIL Clip from Aesculap; the
flat design with a broad contact surface enables smooth
opening and closing as well as stability.

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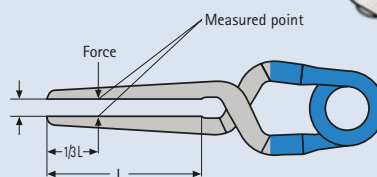
High performance in production

Did you know that...

- ... 30 production steps are required to produce one clip?
- ... 70% of the production is performed manually under the microscope?
- ... a clip is checked 10 times for dimension, surface and closing force during the production process?
- ... the profile of the blades is stamped evenly with 24 tons?
- ... the manufacturing tolerances of 0.02 mm are 10 times smaller than a human hair?
- ... the welding temperature for the metal plate reaches 2500 °C (4532 °F)?
- ... the serial no. allows to trace back every clip individually?

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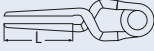
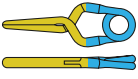
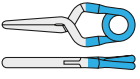
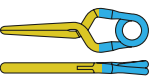
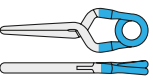
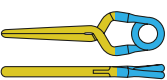
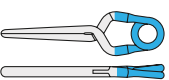
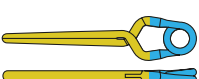
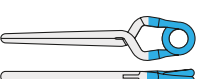
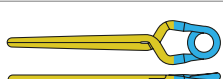
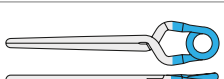
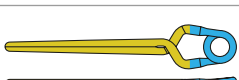
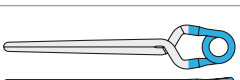
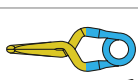
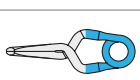
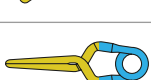
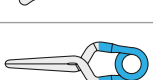
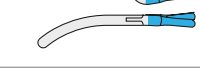
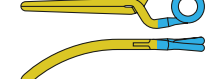
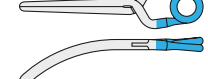
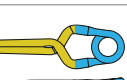
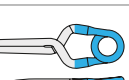
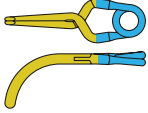
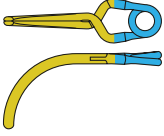
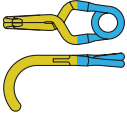
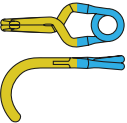
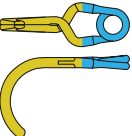
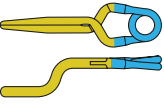
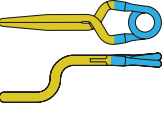
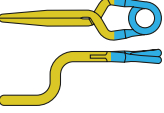
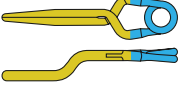
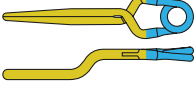
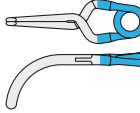
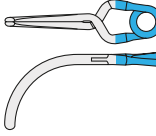
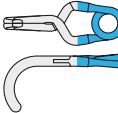
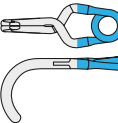
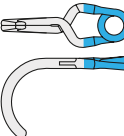
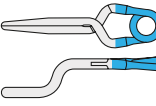
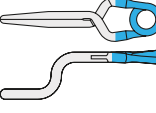
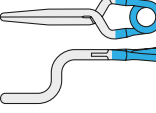
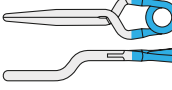
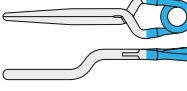
- MRI examinations using magnetic fields of 1.5 and 3.0 Tesla do not present an additional risk to implant wearers
- Minor artifacts in CT and MRI
- Color coding with blue spring (Standard clip) and golden or silver blades (temporary or permanent clip respectively)
- Sterile packaging

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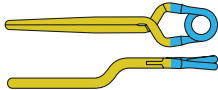
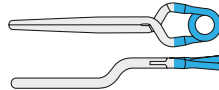
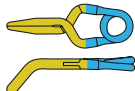
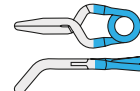
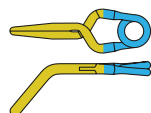
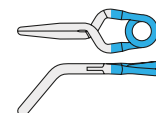
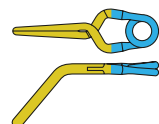
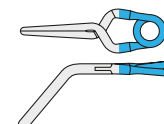
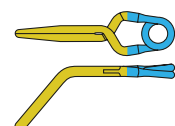
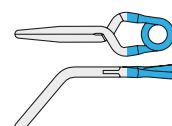
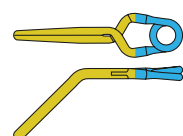
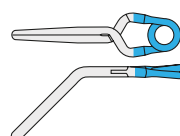
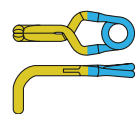
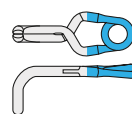
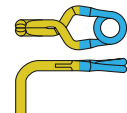
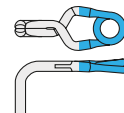
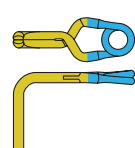
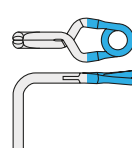
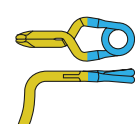
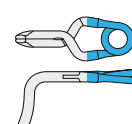
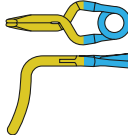
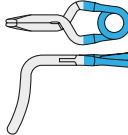
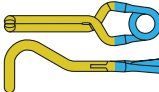
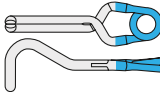
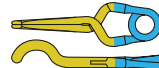
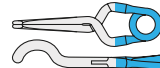
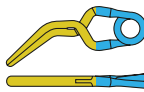
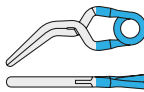
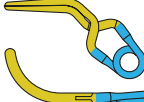
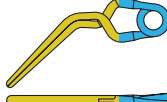
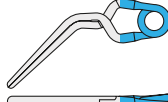
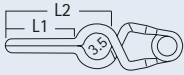
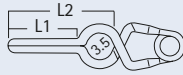
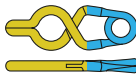
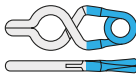
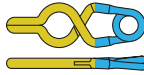
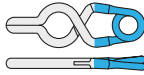
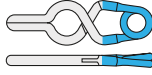
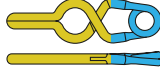
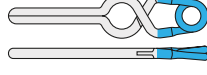
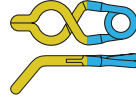
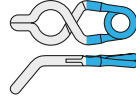
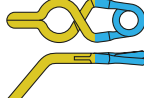
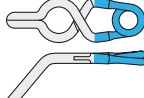
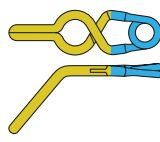
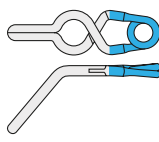
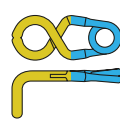
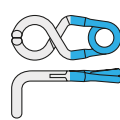
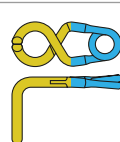
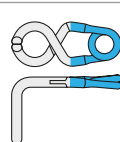
Aesculap® YASARGIL Aneurysm Clip System

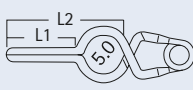
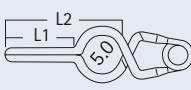
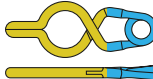
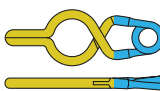
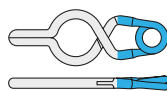
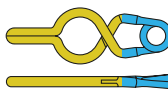
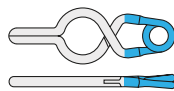
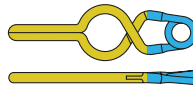
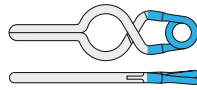
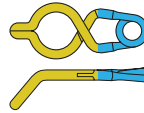
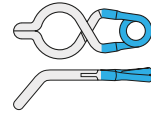
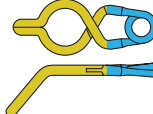
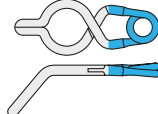
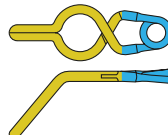
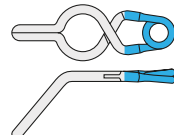
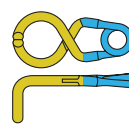
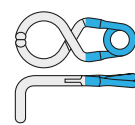
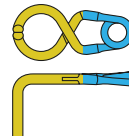
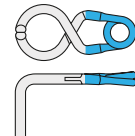
Titanium Standard Clips with Fenestration, 3.5 mm

TITANIUM STANDARD	TEMPORARY TEMPORÄR						PERMANENT					
	Art. No. Art. Nr.		Diameter of fenestration 3.5 mm Fensterdurchmesser 3,5 mm	Blade length Maullänge mm L1/L2	Maximal opening maximale Öffnung mm	Closing force Schließkraft N g	Art. No. Art. Nr.		Diameter of fenestration 3.5 mm Fensterdurchmesser 3,5 mm	Blade length Maullänge mm L1/L2	Maximal opening maximale Öffnung mm	Closing force Schließkraft N g
	FT097T			3/8.1	7.5	1.08 110	FT597T			3/8.1	7.5	1.47 150
	FT098T			4/9.1	7.4	1.08 110	FT598T			4/9.1	7.4	1.47 150
	FT099T			5/10.1	7.4	1.08 110	FT599T			5/10.1	7.4	1.47 150
	FT100T			6/11.1	7.4	1.08 110	FT600T			6/11.1	7.4	1.47 150
	FT110T			9/14.1	8.4	1.08 110	FT610T			9/14.1	8.4	1.47 150
	FT120T			12/17	9.6	1.08 110	FT620T			12/17	9.6	1.77 180
	FT102T			5/7.8	5.8	1.08 110	FT602T			5/7.8	5.8	1.47 150
	FT112T			7.5/9.4	6.5	1.08 110	FT612T			7.5/9.4	6.5	1.47 150
	FT122T			10/11	7.2	1.08 110	FT622T			10/11	7.2	1.77 180
	FT104T			5/5.1	4.9	1.08 110	FT604T			5/5.1	4.9	1.47 150
	FT114T			7.5/5.1	4.9	1.08 110	FT614T			7.5/5.1	4.9	1.47 150

Aesculap® YASARGIL Aneurysm Clip System

Titanium Standard Clips with Fenestration, 5.0 mm

Aesculap Neurosurgery

TEMPORARY TEMPORÄR						PERMANENT					
Art. No. Art. Nr.		Diameter of fenestration 5 mm Fensterdurchmesser 5 mm				Art. No. Art. Nr.		Diameter of fenestration 5 mm Fensterdurchmesser 5 mm			
		Blade length Mauillänge mm L1/L2	Maximal opening maximale Öffnung mm	Closing force Schließkraft N g				Blade length Mauillänge mm L1/L2	Maximal opening maximale Öffnung mm	Closing force Schließkraft N g	
FT137T		3/9.8	7.4	1.08	110	FT637T		3/9.8	7.4	1.47	150
FT138T		4/10.8	7.4	1.08	110	FT638T		4/10.8	7.4	1.47	150
FT139T		5/11.8	7.4	1.08	110	FT639T		5/11.8	7.4	1.47	150
FT140T		6/12.7	7.9	1.08	110	FT640T		6/12.7	7.9	1.47	150
FT150T		9/15.7	9.1	1.08	110	FT650T		9/15.7	9.1	1.47	150
FT142T		5/9	6.5	1.08	110	FT642T		5/9.4	6.5	1.47	150
FT152T		7.5/10.6	7.2	1.08	110	FT652T		7.5/10.6	7.2	1.47	150
FT162T		10/12.2	7.8	1.08	110	FT662T		10/12.2	7.8	1.77	180
FT144T		5/6.7	5.5	1.08	110	FT644T		5/6.7	5.5	1.47	150
FT154T		7.5/6.7	5.5	1.08	110	FT654T		7.5/6.7	5.5	1.47	150

TITANIUM STANDARD

Aesculap® YASARGIL Aneurysm Clip System

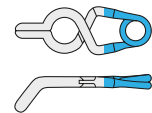
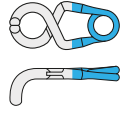
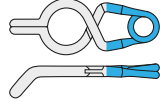
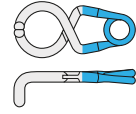
ULTRA SHORT FENESTRATED Aneurysm Clip

"Dog-ear"-Clips

Aneurysm clips with a particularly short blade length (3 mm) and fenestration serve for the treatment of minor aneurysms (e.g. "dog-ear-aneurysms").

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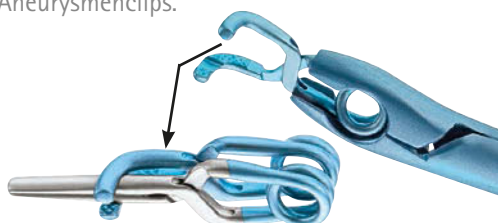
Titanium Reinforcing Clip

Titanium reinforcing clip to increase the closing force of YASARGIL Titanium Standard Aneurysm Clips.

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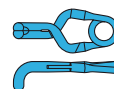
PERMANENT

First Mini clips with fenestration

- Allow using fenestrated clips in narrow spaces

Thin blades and thus a reduced

triangle-shaped gap at the proximal end

- Support the precise reconstruction of the vascular structure with multiple clipping techniques

Area of use

- Neck remnants of clipped aneurysms like "Dog-ear aneurysms"

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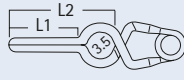
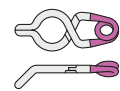
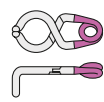
Developed in co-operation with:
Entwickelt in Zusammenarbeit mit:

Rokuya Tanikawa, M. D.

Director Stroke Center, Department of Neurosurgery
Teishinkai Hospital, Sapporo, Japan

Aesculap® YASARGIL Aneurysm Clip System

Titanium T-Bar Clips acc. to KOPITNIK, 5.0 mm

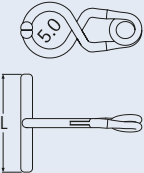
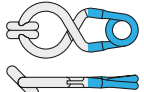
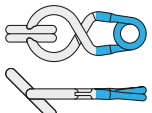
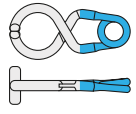
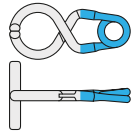
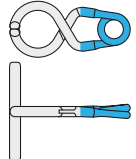
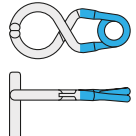
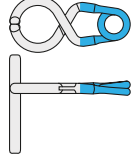
Aneurysm clip with a unique design

The blades of the T-Bar Clip are oriented in a 90°, 60° or 45° angle to the axis of the spring plane resulting in a T-like clip form. The unique shape opens up new options for the treatment of complicated cerebral aneurysms. The spring type is identical to the existing and proven standard YASARGIL Aneurysm Clip with a 5 mm diameter fenestration.

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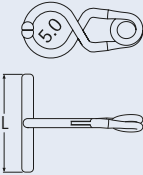
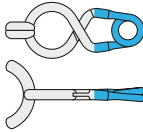
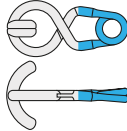
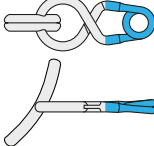
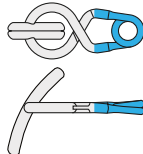
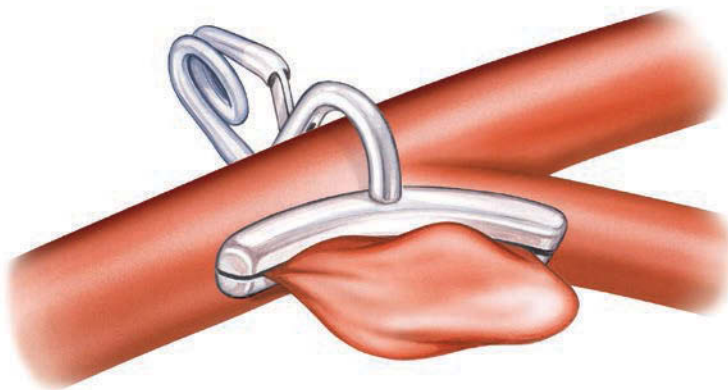
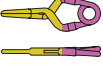
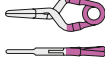
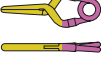
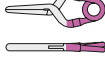
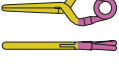
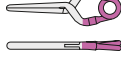
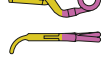
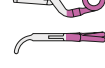
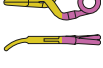
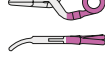
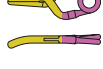
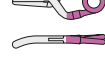
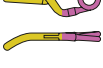
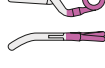
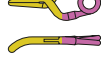
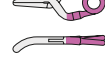
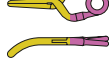
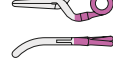
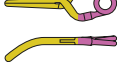
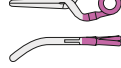
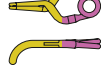
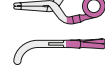
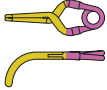
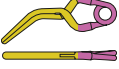
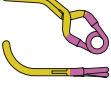
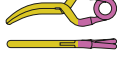
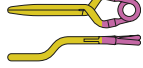
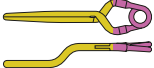
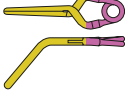
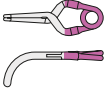
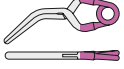
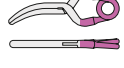
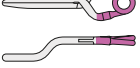
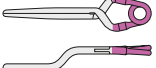
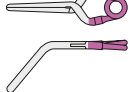
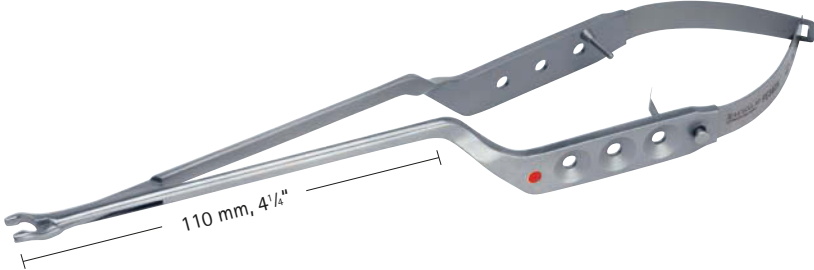
Aesculap® YASARGIL Aneurysm Clip System

Double Bayonet Clip Applier for Titanium Clips

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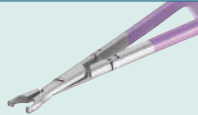
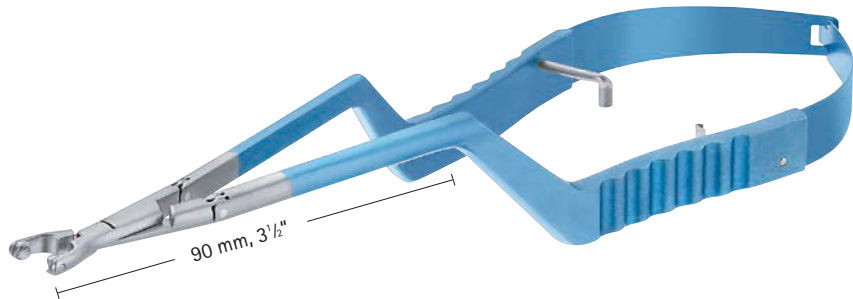
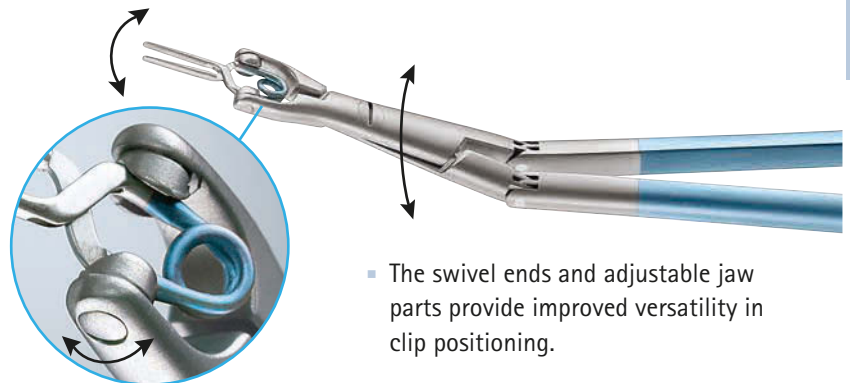
APPLIERS FOR TITANIUM CLIPS

Aesculap® YASARGIL Aneurysm Clip System XS Clip Applier for Titanium Clips

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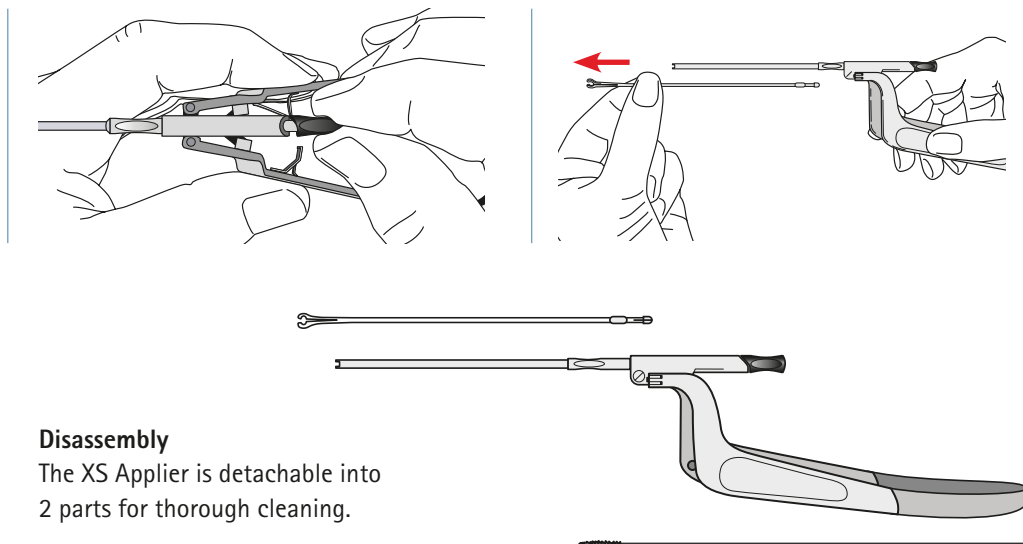
Total length: 250 mm, 9 3/4"
Gesamtlänge: 250mm, 9 3/4"

Disassembly

The XS Applier is detachable into 2 parts for thorough cleaning.

A cleaning brush is provided free of charge with every applier.

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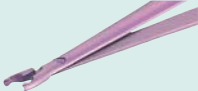
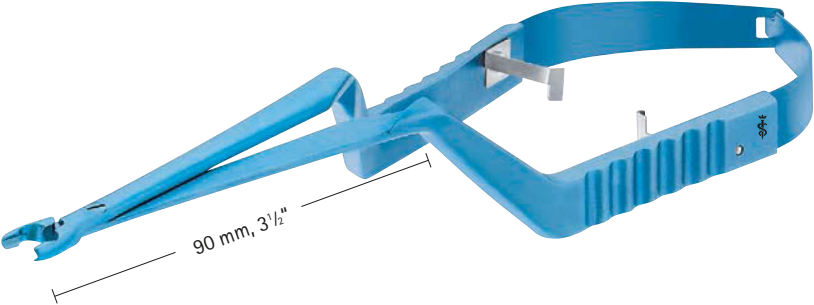
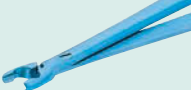
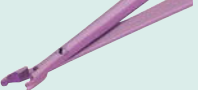
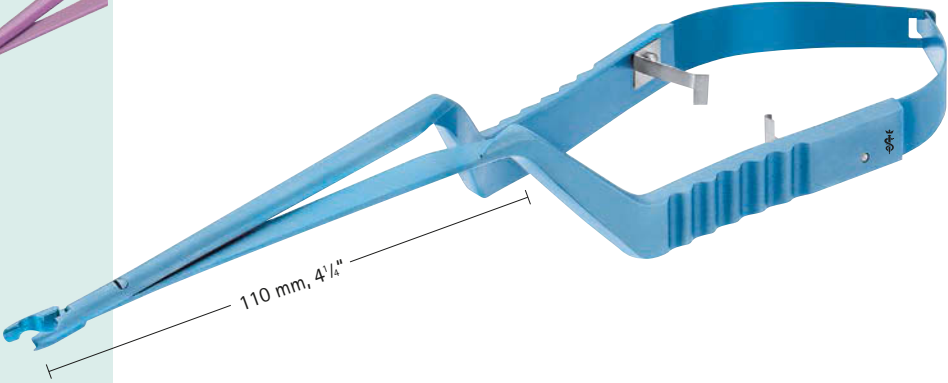
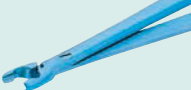
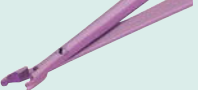
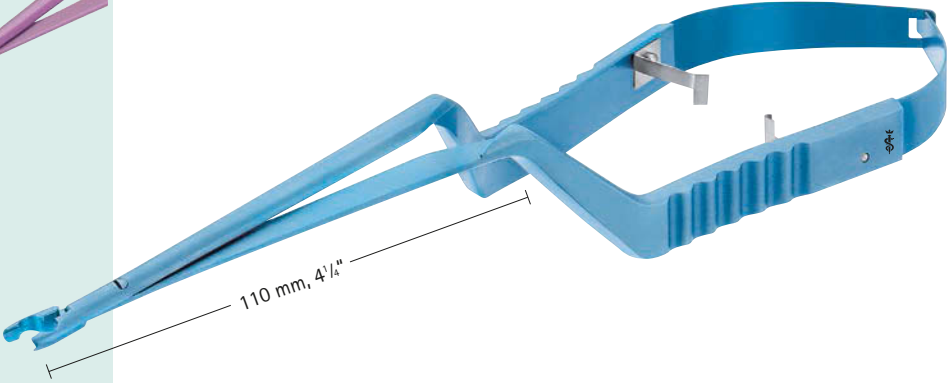
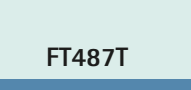
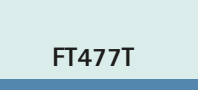
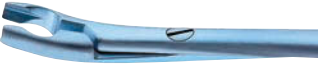
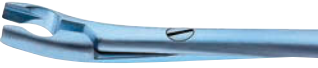
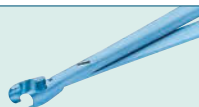
Aesculap® YASARGIL Aneurysm Clip System

Removing Forceps for Titanium Clips

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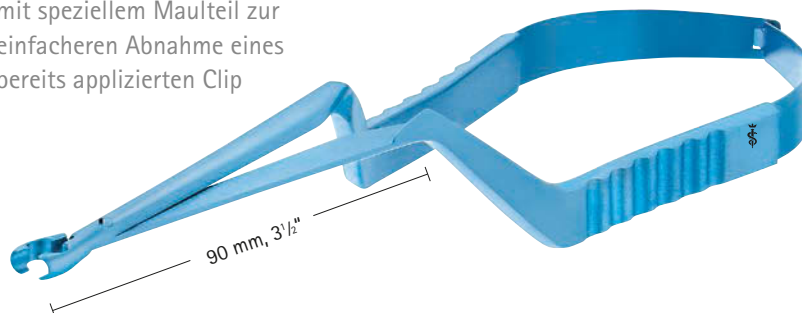
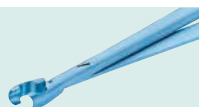
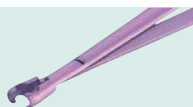
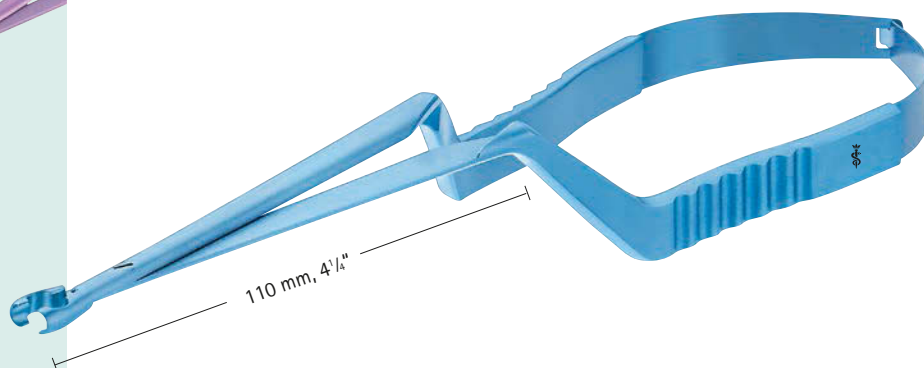
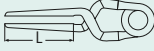
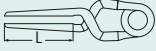
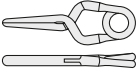
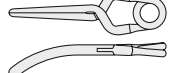
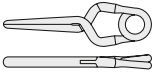
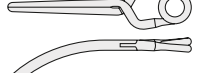
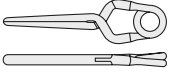
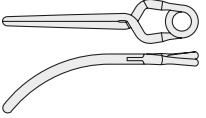
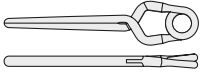
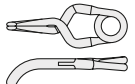
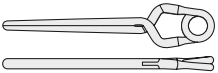
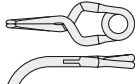
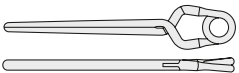
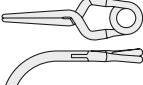
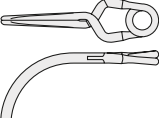
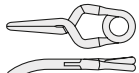
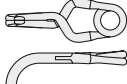
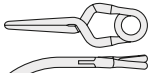
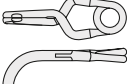
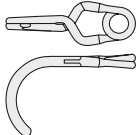
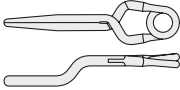
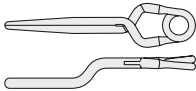
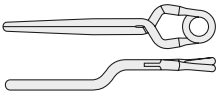
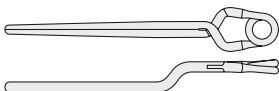
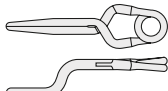
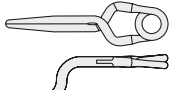
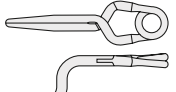
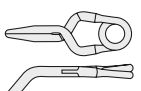
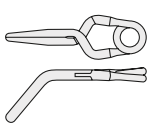
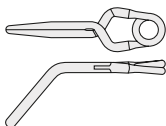
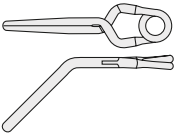
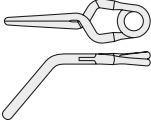
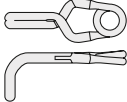
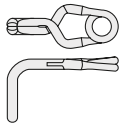
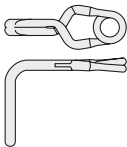
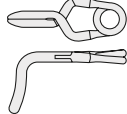
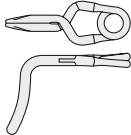
- MRI examinations using magnetic fields of 1.5 and 3.0 Tesla do not present an additional risk to implant wearers
- Color coding with silver spring and blades (permanent clip)
- Sterile packaging

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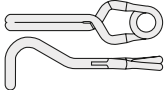
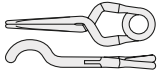
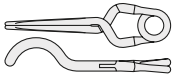
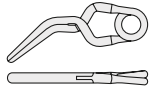
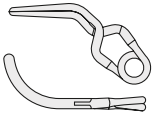
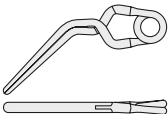
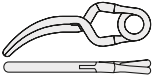
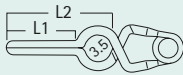
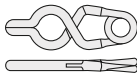
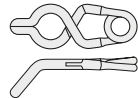
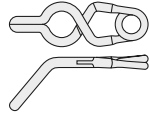
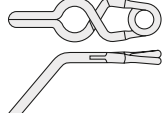
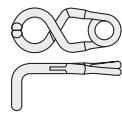
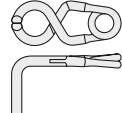
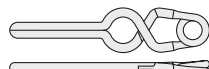
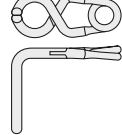
Aesculap® YASARGIL Aneurysm Clip System

Phynox Standard Clips with Fenestration, 3.5 mm

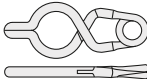
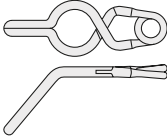
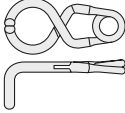
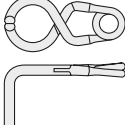
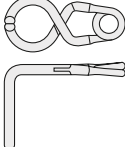
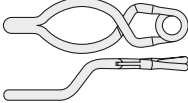
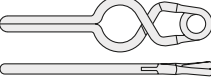
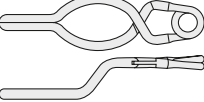
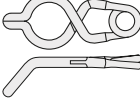
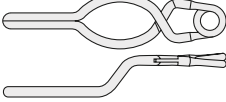
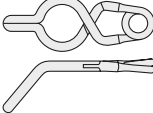
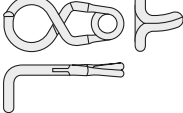
Aesculap Neurosurgery

PERMANENT						PERMANENT					
Art. No. Art. Nr.		Diameter of fenestration 3.5 mm Fensterdurchmesser 3,5 mm									
		Blade length Maullänge mm L1/L2	Maximal opening maximale Öffnung mm	Closing force Schließkraft N g							
FE597K		3/8.1	6.5	1.47	150	FE602K		5/7.3	5.8	1.47	150
FE598K		4/9.1	6.6	1.47	150	FE612K		7.5/8.9	6.5	1.47	150
FE599K		5/10.1	7.0	1.47	150	FE622K		10/10.5	7.2	1.77	180
FE600K		6/11	7.4	1.47	150	FE604K		5/4.9	4.9	1.47	150
FE610K		9/14	8.4	1.47	150	FE614K		7.5/4.9	4.9	1.47	150
FE620K		12/17	9.6	1.77	180	FE624K		10/4.9	4.9	1.77	180

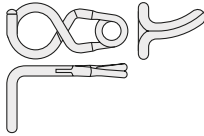
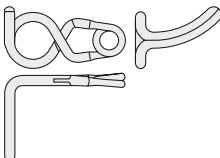
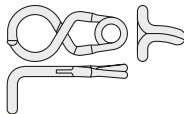
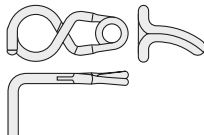
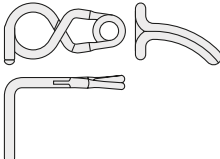
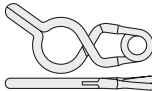
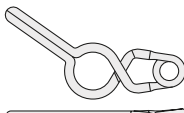
PHYNEX STANDARD

Aesculap® YASARGIL Aneurysm Clip System

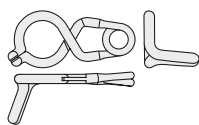
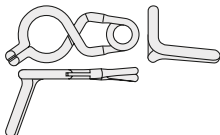
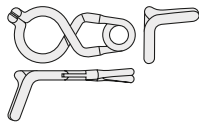
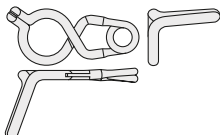
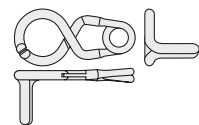
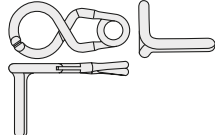
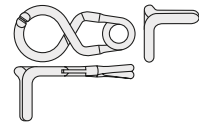
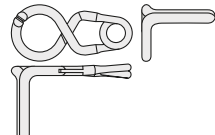
Phynox Standard Clips with Fenestration, 5.0 mm

PHYNOX STANDARD	PERMANENT						PERMANENT					
	Art. No. Art. Nr.		Diameter of fenestration 5 mm Fensterdurchmesser 5 mm	Blade length Maullänge mm L1/L2	Maximal opening maximale Öffnung mm	Closing force Schließkraft N g	Art. No. Art. Nr.		Diameter of fenestration 5 mm Fensterdurchmesser 5 mm	Blade length Maullänge mm L1/L2	Maximal opening maximale Öffnung mm	Closing force Schließkraft N g
	FE637K			3/9.8	6.7	1.47 150	FE662K			10/12.2	7.8	1.77 180
	FE638K			4/10.8	7.1	1.47 150	FE644K			5/6.4	5.5	1.47 150
	FE639K			5/11.8	7.5	1.47 150	FE654K			7.5/6.4	5.5	1.47 150
	FE640K			6/12.7	7.9	1.47 150	FE664K			10/6.4	5.5	1.77 180
	FE650K			9/15.7	9.1	1.47 150	FE648K			5/15.2	7.9	1.47 150
	FE660K			12/18.7	10.3	1.77 180	FE658K			7.5/17.7	8.8	1.47 150
	FE642K			5/9	6.5	1.47 150	FE668K			10/20.2	10.0	1.77 180
	FE652K			7.5/10.6	7.2	1.47 150	FE774K			5/6.7	5.0	1.47 150

PERMANENT

Art. No. Art. Nr.		Diameter of fenestration 5 mm Fensterdurchmesser 5 mm		Blade length Mauillänge mm L1/L2		Maximal opening maximale Öffnung mm		Closing force Schließkraft N g	
FE775K				7.5/6.7		5.0		1.47	150
FE776K				10/6.7		5.0		1.47	150
FE777K				5/6.7		5.0		1.47	150
FE778K				7.5/6.7		5.0		1.47	150
FE779K				10/6.7		5.0		1.47	150
FE881K				5/10.4		7.0		1.47	150
FE882K				7.5/12.4		7.8		1.47	150
FE883K				10/14.5		8.8		1.47	150

PERMANENT

Art. No. Art. Nr.		Diameter of fenestration 5 mm Fensterdurchmesser 5 mm		Blade length Mauillänge mm L1/L2		Maximal opening maximale Öffnung mm		Closing force Schließkraft N g	
FE886K				5/7.4		5.0		1.47	150
FE887K				7.5/8.1		5.0		1.47	150
FE888K				5/7.4		5.0		1.47	150
FE889K				7.5/8.1		5.0		1.47	150
FE891K				5/6.6		5.0		1.47	150
FE892K				7.5/6.6		5.0		1.47	150
FE893K				5/6.6		5.0		1.47	150
FE894K				7.5/6.6		5.0		1.47	150

Aesculap® YASARGIL Aneurysm Clip System

Phynox T-Bar Clips acc. to KOPITNIK, 5.0 mm

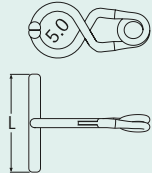
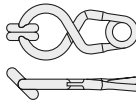
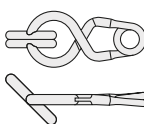
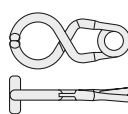
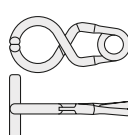
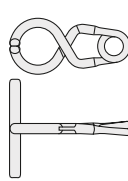
Aneurysm clip with a unique design

The blades of the T-Bar Clip are oriented in a 90° or 45° angle to the axis of the spring plane resulting in a T-like clip form. The unique shape opens up new options for the treatment of complicated cerebral aneurysms. The spring type is identical to the existing and proven standard YASARGIL Aneurysm Clip with a 5 mm diameter fenestration.

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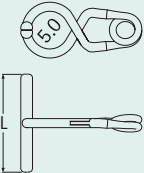
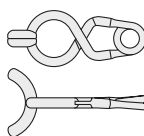
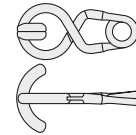
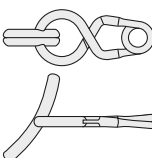
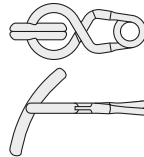
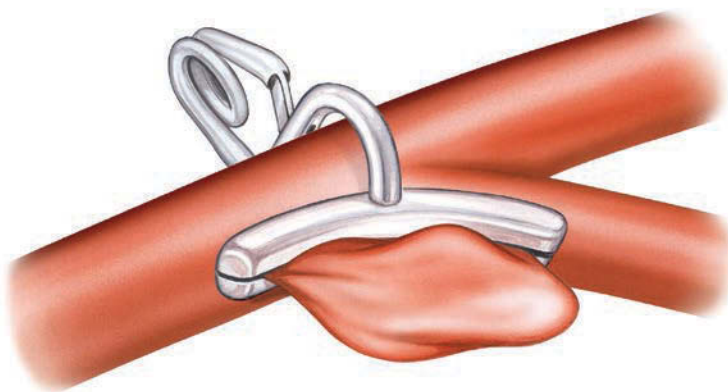
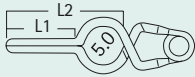
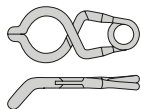
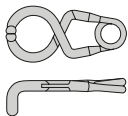
PHYNEX STANDARD

Aesculap® YASARGIL Aneurysm Clip System

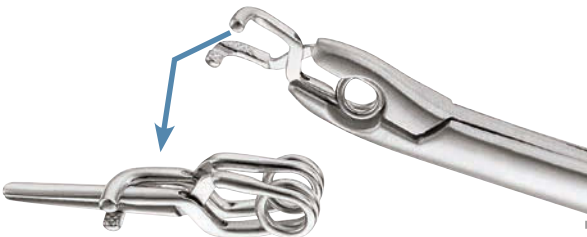
ULTRA SHORT FENESTRATED Aneurysm Clip, 5.0 mm

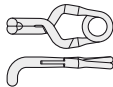
PHYNOX STANDARD	PERMANENT				
	Art. No. Art. Nr.		Diameter of fenestration 5 mm Fensterdurchmesser 5 mm		
			Blade length Maullänge mm L1/L2	Maximal opening maximale Öffnung mm	Closing force Schließkraft N
	FE942K		3.0/8.1	6.0	1.47
	FE944K		3.0/6.4	5.5	1.47

Phynox Reinforcing Clip

Phynox reinforcing clip to increase the closing force of YASARGIL Phynox Standard Aneurysm Clips.

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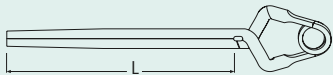
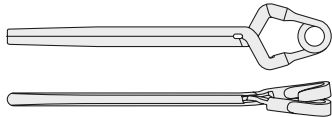
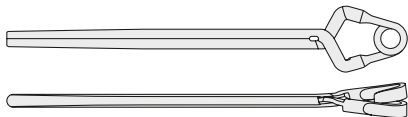
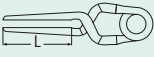
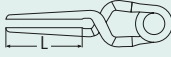
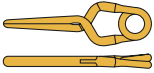
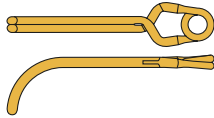
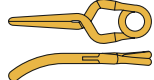
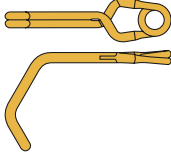
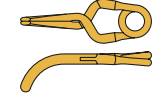
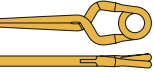
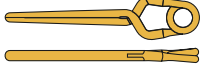
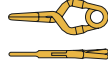
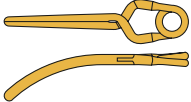
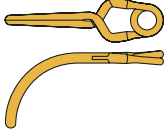
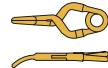
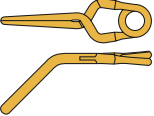
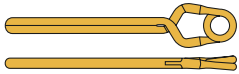
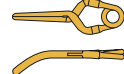
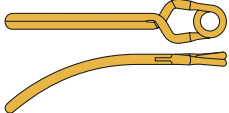
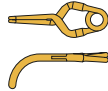
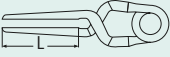
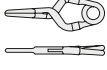
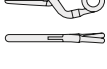
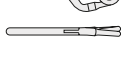
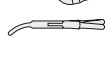
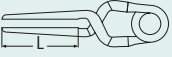
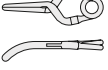
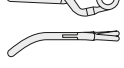
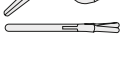
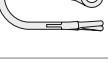
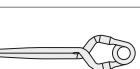
- MRI examinations using magnetic fields of 1.5 and 3.0 Tesla do not present an additional risk to implant wearers
- Color coding with silver spring and blades (permanent clip)

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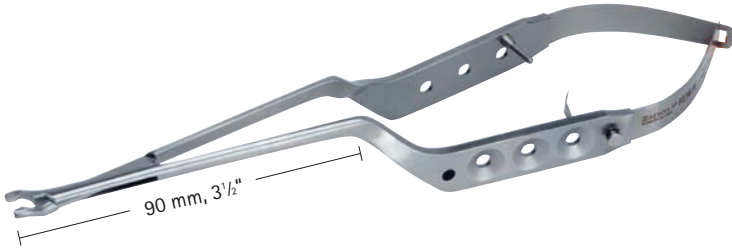
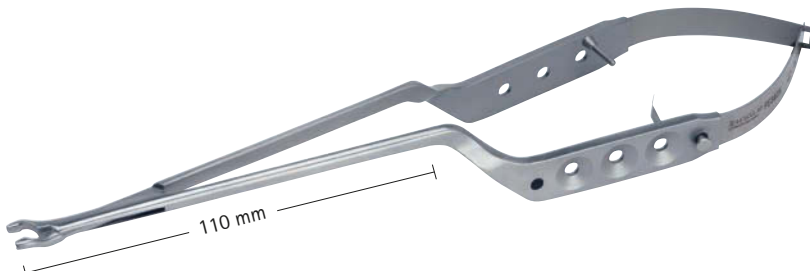
Aesculap® YASARGIL Aneurysm Clip System

Double Bayonet Clip Applier for Phynox Clips

■ Color coding is realized as a colored button in the handle area (● = Standard; ● = Mini)

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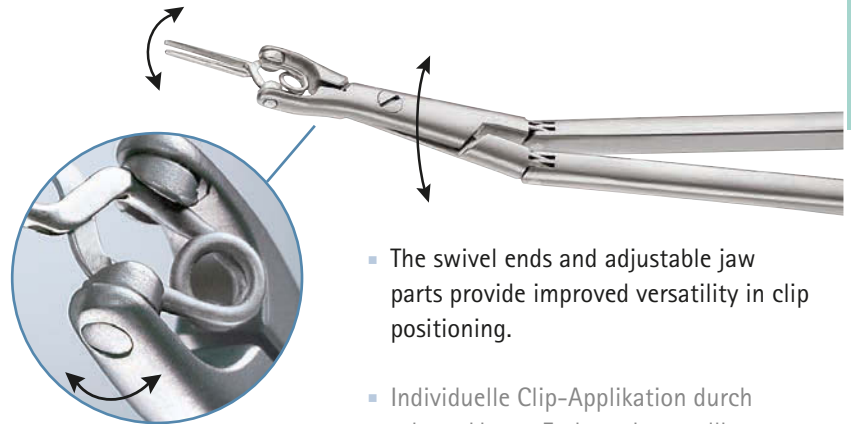
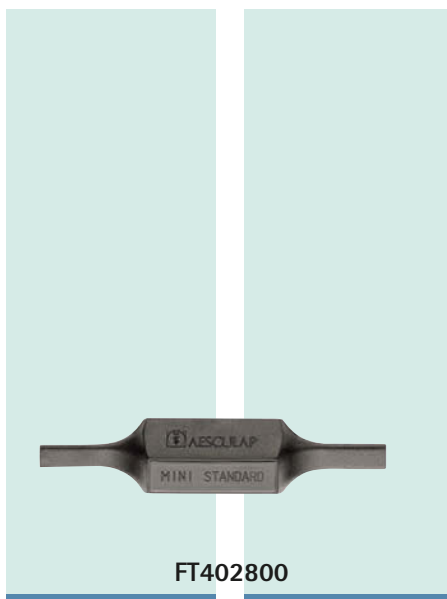
APPLIERS FOR PHYNOK CLIPS

Aesculap® YASARGIL Aneurysm Clip System XS Clip Applier for Phynox Clips

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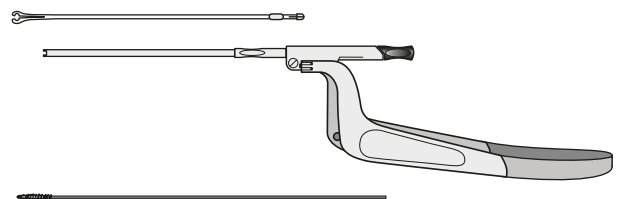
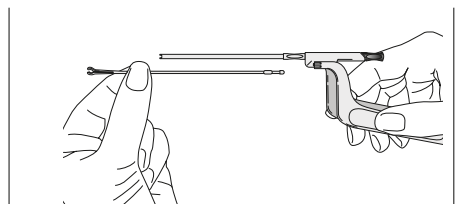
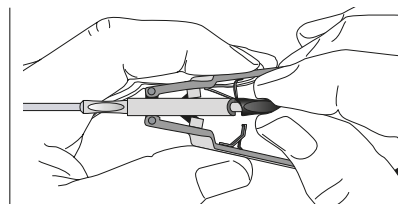
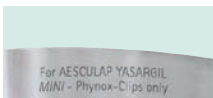
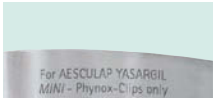
FT007P
FT006P

Clip Tray 1/1

For storage, transport and sterilization of Aesculap YASARGIL Aneurysm Clips.

- Autoclavable plastic
- Fits 1/1 standard container
- 34 storage compartments for Standard and Mini Clips
- 4 storage compartments for Long Clips
- With special identification tags for individual tray configuration

Clip Tray 1/2

For storage, transport and sterilization of Aesculap YASARGIL Aneurysm Clips.

- Autoclavable plastic
- Fits 1/2 standard container
- 15 storage compartments for Standard and Mini Clips
- 3 storage compartments for Long Clips
- With special identification tags for individual tray configuration

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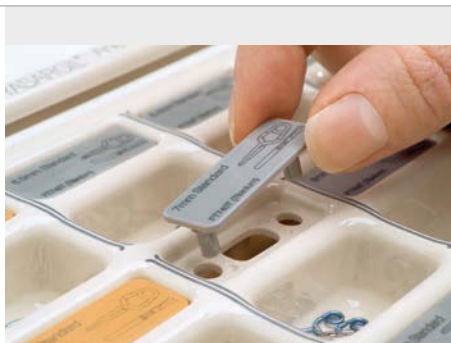
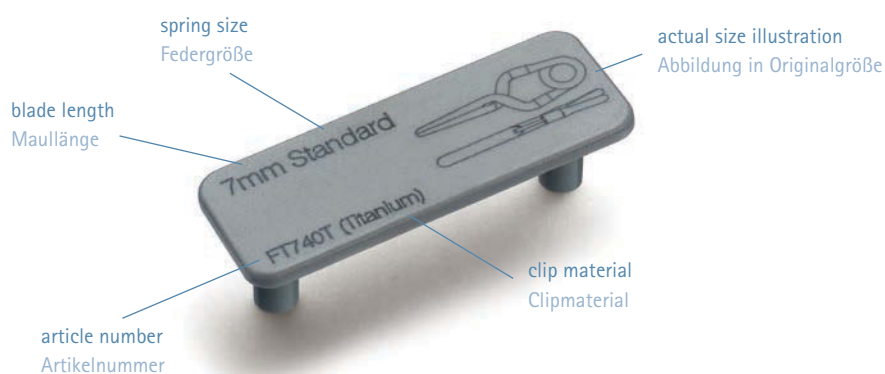
Identification tags for the Clip Trays

Allow quick and easy recording of the necessary clip data;
Helpful for classifying and reordering the clips that have been removed.

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For each clip variant there is a matching identification tag.

The article numbers of the tags are as follows:

Art. No. of clip + „-TAG“

e.g.: FT740T-TAG is the identification tag for the clip FT740T

The identification tags for temporary clips are **yellow** and for permanent clips **grey** in line with the color coding of the YASARGIL Aneurysm Clips.

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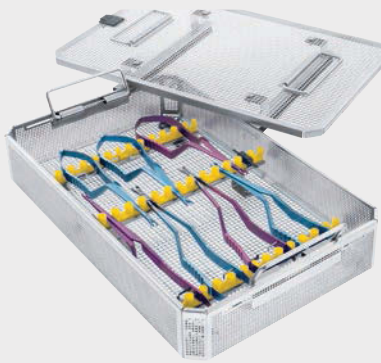
Aesculap® YASARGIL Aneurysm Clip System Storage

FM665R

Storage for clip applicators

Perforated basket with lid, for storage of clip applicators

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Address: www.aesculap-extra.net

Patient identification card for Clip implants

- Provides traceability for physician and patient for follow-ups
- Includes information about the MRI-compatibility of the clips
- Printed version available in German and English
- Download from the Extranet possible:

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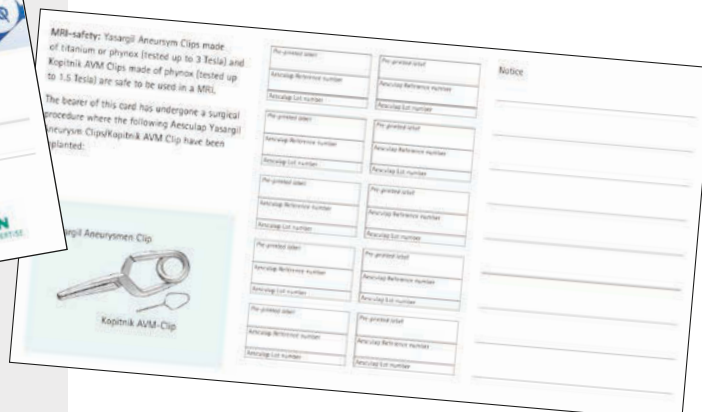
For the treatment of arteriovenous malformations.

Aesculap® KOPITNIK AVM Micro Clip System

Apart from cerebral aneurysms, arteriovenous malformations are among the most common cerebrovascular disorders. The widely renowned YASARGIL Aneurysm Clips by Aesculap form an established part of the gold standard in microsurgical aneurysm therapy. They are, also, still used in microsurgical angioma operations. However, the extremely delicate vessels often require the application of finer clips. Therefore I have developed, with Aesculap, a special AVM Micro Clip System for the microsurgical treatment of angiomas.

The KOPITNIK AVM Micro Clip System comprises straight and, for the first time, curved micro clips as well as special applying forceps. All these clips are available in the sizes (length) 2, 3, 4 and 5 mm. The applying forceps, which allows the operator an excellent view of the pathology and the clip. The slim shaft offers exceptional flexibility within the craniotomy. The rotatable shaft and the unique active lock allows the operator to apply the clips comfortably.

Another innovation incorporated in this AVM Micro Clip System is that the AVM micro clips can be applied either temporarily or permanently. Depending on the operational situation, the clips can be removed or they can be left in site as permanent implants. Phynox is a well-tried material that has proven its reliability in the production of YASARGIL Aneurysm Clips for more than 30 years. It offers excellent biocompatibility. MRI examinations using magnetic fields of 1.5 and 3.0 Tesla do not present an additional risk to implant patients.

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Thomas A. Kopitnik

The AVM Micro Clip System for the treatment of arteriovenous malformations was developed in co-operation with:

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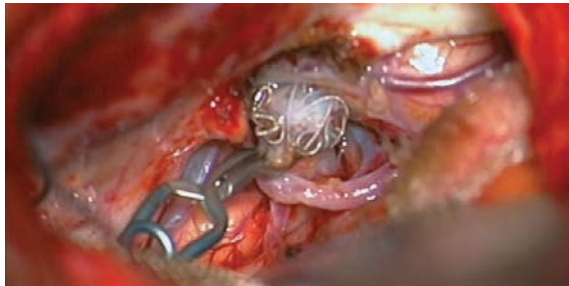
110 mm, 4 3/8"

FD063R

In special cases, coil wires have to be cut and removed. For this procedure, special coil scissors are extremely helpful.

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„With the new coil scissors it is simple and safe to cut the coils – a very helpful tool when you have to operate on an aneurysm that had been previously treated via an endovascular approach.“

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... the brochure includes all four product lines of bone flap fixation like CranioFix®2, CranioFix® absorbable, CranioFix® PEEK and CranioPlate.

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For more information see
Brochure C90602

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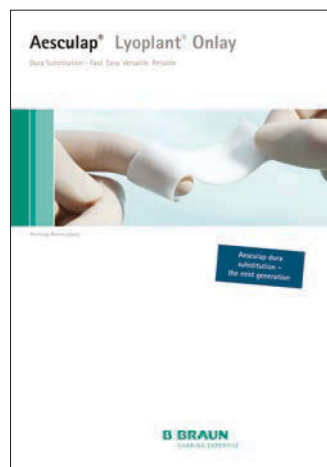
... the new biological, absorbable dura substitution designed to provide high ease of use.

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■ Non-Stick Bipolar

... advanced generator meets advanced instruments.

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■ Neuroendoscopy Neuroendoskopie

... the brochure includes systems for Intraventricular, Endoscope-Assisted, Transnasal/Transsphenoidal Endoscopy.

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■ EC/IC Bypass Set

... fine instruments particularly suits for EC/IC procedures.

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■ ScalpFix®

... for reliable haemostasis; easy handling and fast application are time and cost saving.

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