Infusion/Transfusion

Pressure Infusion Cuffs

Pressure Infusion Cuffs

The safe press can be used for liquid bags up to 500 resp. 1.000 ml filling volume. It consists of one cuff with integrated bag holder, one connecting tube with manometer and one pumpball. The cuff is used for pressure-supported infusions and transfusions.

Sterile: No

REF	Capacity	PU
43.018.00.012	500 ml	20 pcs
43.019.00.012	1.000 ml	20 pcs

Pressure Infusion Cuffs, reusable

The safe press can be used for liquid bags up to 500 resp. 1.000 ml filling volume. It consists of one cuff with integrated bag holder, one connecting tube with manometer and one pumpball with outlet valve. The cuff is used for pressure-supported infusions and transfusions. The cuff is reusable.

Allergenic potential: Latex-free

REF	Capacity	PU
43.018.00.013	500 ml	6 pcs
43.019.00.013	1.000 ml	6 pcs

Pressure Infusion Cuff

REF: 43.018.00.013

Unit: 1 pc

Packing Unit: 6 pcs.

Description

The re-usable Pressure infusion cuff "Safe Press" is suitable for up to 500 ml liquid collection bags. It contains a cuff with an integrated hanger, a connection tube with aneroid gauge and a hand bulb with integrated bottom valve.

The cuff is used for pressure-supported infusions and transfusions.

The cuff is re-usable.

The cuff does not contain latex.

Material Composition

<u>Product:</u>	bag:	Nylon
	hanger:	ABS
	tube:	PVC
	aneroid gauge:	ABS
	valve:	PVC
	bottom valve:	PVC

<u>Packing:</u>	individual packing:	PE-Bag
	outer case:	corrugated cardboard

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(GB) Directions for use
Pressure Infusion Cuff

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GB

Description

The Pressure Infusion Cuff is comprised of an inflatable cuff as well as a manometer with an inflation bulb. It is used to apply pressure to flexible IV solution bags in order to facilitate rapid fluid infusion.

Cautions

1. Before use, perform a visual inspection of all components to check for damage.
2. Avoid over-inflating the cuff. Otherwise, damage could result that could in turn be harmful to health.
3. Do not use the product if you notice a malfunction (e.g. if the manometer indicator does not move or if the pressure drops rapidly).

Use

1. Attach the black inflation bulb securely to the blue stopcock and hook the Pressure Infusion Cuff onto an IV pole.
2. Attach the IV solution bag to the hook inside the cuff. Wrap the cuff tightly around the bag and secure the Velcro strap.
3. Turn the stopcock to open the connection between the inflation bulb and the cuff.
4. Close the valve on the inflation bulb. Squeeze the inflation bulb in order to establish the required pressure. (Generally 300 mmHg is recommended, however, the doctor's instructions must always be observed.) You should also ensure that the cuff is not over-inflated. Do not allow the manometer indicator to enter the RED zone.

5. Open the valve on the inflation bulb in order to relieve the pressure in the cuff again. You can also turn the stopcock in the opposite direction so that air can be released from the cuff.

Cleaning and maintenance

1. If the Pressure Infusion Cuff comes into contact with blood, immediate cleaning is recommended. If this is not possible, cleaning should be performed as soon as possible within a matter of hours.
2. Removing the inflation ball from the cuff is not required. Clean the surface using a damp cloth. Do not soak the product or immerse it in liquid.
3. Clean the surface of the cuff, the tube and the inflation bulb using a cloth dampened with an alcohol disinfectant solution. This does not apply to the manometer. To prevent spray mist from forming, do not spray any solution. Allow to dry at room temperature. Ensure that the manometer is completely dry before packing it away or storage.
4. Regularly check whether the indicator of the manometer is fully functional by lightly squeezing the inflation bulb. Also check whether the indicator points to zero when no pressure is applied.

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P. J. Dahlhausen & Co. GmbH