All4Labels
GLOBAL PACKAGING GROUP

RAKO Etiketten GmbH & Co. KG
Möllner Landstr. 15
22969 Witzhave

Phone: +49 4104 693 0
Fax: +49 4104 693 110
Mail: info@all4labels.com
Web: www.all4labels.com

Product Name	Mat.No.	Index	Dimensions	Package	Country	EAN
thermosept ED	20002902	A	280 x 105 mm	HE-KA	DE, EN, FR, PL	-
Application Area / Category	Label No.	Version	Techn. Spec.	Date	Operator	Font Size
Instrumente/MP	305064108-00	04	015	16.04.20	JG	6 pt. / 90%

1

2

3

4

5

6

7

8

9

10

Alt: 20001276-B / 305049158_02

Zusatzetikett: 20002903-A

Exclusion of liability for the implementation of statutory provisions and color liability. This PDF is not color-coded, it only serves to check text, status and color distribution. Compliance with applicable statutory provisions with regard to font sizes, industry-sector-specific labeling requirements, trademark, competition and licensing laws must be checked and ensured by the customer.

Please use the overprint option for PDF preview.

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thermosept® ED

EN Endoscope disinfectant for chemical decontamination with heat ➤ **Application areas:** thermosept® ED is a disinfectant component, provided as a concentrate, for chemo-thermal reprocessing of flexible and rigid endoscopes including endoscopic accessories and for thermolabile medical devices (e.g. anesthesia material) in washer-disinfectors for endoscopes (AER) and washer disinfectors (WD). ➤ **Use-concentration/ contact time:** 1 % at 55 °C to 60 °C 5 minutes minimum contact time. Dosing takes place automatically via the machine's own pump ➤ **Microbiological efficacy throughout the process in accordance with DIN EN ISO 15883-4:** bactericidal, fungicidal, mycobactericidal, virucidal, bacterial endospores (C. diff.). ➤ **Notes on use:** For professional use only. Depending on the water quality, colour changes may occur on metallic parts of the endoscopes. The functioning of the endoscopes is not affected. The product is compatible with thermosept® ER / thermosept® EndoCleaner (cleaner). Do not store above room temperature. Do not use preparation after the expiry date. Any serious incidents related to the product are to be reported to the manufacturer and competent authority. ➤ **Composition:** 100 g contain the following active ingredients: 20 g Glutaraldehyde. ➤ **Hazard notes and safety advice:** **DANGER: H302+H332** Harmful if swallowed or if inhaled **H314** Causes severe skin burns and eye damage. **H317** May cause an allergic skin reaction. **H334** May cause allergy or asthma symptoms or breathing difficulties if inhaled. **H335** May cause respiratory irritation. **H412** Harmful to aquatic life with long lasting effects. **P261** Avoid breathing vapours. **P271** Use only outdoors or in a well-ventilated area. **P280** Wear protective gloves/ eye protection/ face protection. **P301+P330+P331** IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. **P303+P361+P353** IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water or shower. **P304+P340** IF INHALED: Remove person to fresh air and keep comfortable for breathing. **P305+P351+P338+P310** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor. **P501** Dispose of contents/container to an approved waste disposal plant. ➤ **Special Advice:** Keep container tightly closed. Store at room temperature in the original container. **EUH071** Corrosive to the respiratory tract.

UN1903
DISINFECTANT, LIQUID, CORROSIVE, N.O.S. (Glutaral)

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Schülke & Mayr GmbH • Robert-Koch-Str. 2
Schülke & Mayr AG • Sihlfeldstrass 58 • CH-8003 Zürich • Tel. +41 44 4665544
Schulke & Mayr UK Ltd. • Sheffield S9 1AT • UK • Tel. +44 114 2543500