



奥美医疗
ALLMED MEDICAL

GAUZE ROLL

TECHNICAL PROPERTIES


Content: 100% bleached cotton yarn;

Weft-Warp String number: 40Sx40S/30x30, $\geq 32\text{g/m}^2$;

Width: 90 cm;

Length: Standard length is 90 meters, Other length can be cut as required;

Shelf life: 5 years.

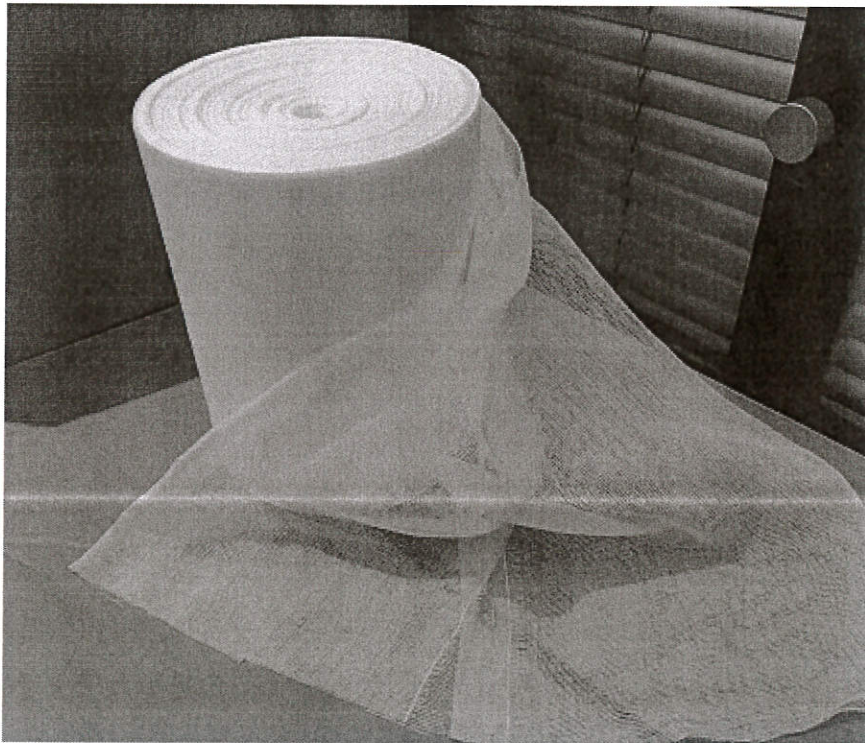

For and on behalf of
ALLMED MEDICAL PRODUCTS CO., LIMITED

Susan Liao

.....
Authorized Signature(s)

Document No.:	Revision No.:	Date of Revision:	Approved by:
WP-011100-AL-02	R-002	2022.10.12	Susan Liao



奥美医疗
ALLMED MEDICAL

Non-Sterile Hydrophilic Gauze Roll

REF Code	Specification	BOX in (Roll)	Box Size (cm)
AM-GR90-90-24A	Non-Sterile Gauze Roll, ≥32g/m ² , width: 90cm, length: 90m	4	49 x 48.5 x 24cm
AM-GR90-90-24A-1	Non-Sterile Gauze Roll, ≥32g/m ² , width: 90cm, length: 1m	195	41*35.5*24cm
AM-GR90-90-24A-2	Non-Sterile Gauze Roll, ≥32g/m ² , width: 90cm, length: 2m	120	44*36.5*24cm
AM-GR90-90-24A-5	Non-Sterile Gauze Roll, ≥32g/m ² , width: 90cm, length: 5m	56	46*40*24cm
AM-GR90-90-24A-10	Non-Sterile Gauze Roll, ≥32g/m ² , width: 90cm, length: 10m	30	48.5*40.5*24cm
AM-GR90-90-24A-20	Non-Sterile Gauze Roll, ≥32g/m ² , width: 90cm, length: 20m	12	46*34.5*24cm
AM-GR90-90-24A-50	Non-Sterile Gauze Roll, ≥32g/m ² , width: 90cm, length: 50m	6	54.5*36.5*24cm
AM-GR90-90-24A-100	Non-Sterile Gauze Roll, ≥32g/m ² , width: 90cm, length: 100m	4	52*51.5*24cm

For and on behalf of
ALLMED MEDICAL PRODUCTS CO., LIMITED

Susan Liao

Authorized Signature(s)

Document No.:	Revision No.:	Date of Revision:	Approved by:
WP-011100-AL-02	R-002	2022.10.12	Susan Liao



Intended use:

This device is intended to be widely used in medical circle and other area for preparing of dressings or tampon with their further fixation on the wound, with protection and barrier function (mechanical) of the injured skin. MD Class I according to Regulation (EU) 2017/745 of the European Parliament and Council.

Product standards: In compliance with EN 14079.

Certificates: CE, ISO13485.

Labeling: in accordance with Chapter III (23) of Annex I of Regulation 2017/745 of the European Commission.

Cautions:

- Read instructions before use.
- Before usage check the packing completeness, the LOT number and expiry date.
- Use only if package is undamaged and validity not expired;
- For single use.
- Destroy after use.

Properties:

- Non-toxic;
- Hypoallergenic;
- Non-inflammable;
- PHT and DEHP free;
- Latex free;
- Durable, hold the dressing securely in place;
- Easy to cut to any shapes and sizes;
- Super softness and patient comfort;
- Highly absorbent;
- Low linting;
- Free from loose wires;
- Non-Sterile unit packages.

For and on behalf of
ALLMED MEDICAL PRODUCTS CO., LIMITED

Susan Liao

Authorized Signature(s)

Document No.:	Revision No.:	Date of Revision:	Approved by:
WP-011100-AL-02	R-002	2022.10.12	Susan Liao



奥美医疗
ALLMED MEDICAL

Instructions for use:

1. Examine individual package of a roll for integrity of packaging prior to use. Do not use if packaging is broken.
2. Wash hands. (Wear gloves where necessary).
3. Before usage, open the roll package.
4. Cut the appropriate Gauze Material size for your procedure.
5. Apply the bandage in accordance with your facility's protocol.
6. Support the injured area while bandaging (if necessary).
7. Immediately dispose of after single use.

Packing:

- each gauze roll is individually packed
- then is packed in strong export carton

Storage:

Protect from sunlight, extreme humidity and heat sources. Not store with organic or chemical solvents. Not store with corrosive gases. Store in dry, cool, good ventilated and clean room, where temperatures between 10°C and 40°C and relative humidity between 30% and 60% are recommended. Once the package is opened, store the remaining roll isolated in clean environment in compliance with conditions mentioned above.



ALLMED MEDICAL PRODUCTS CO.,LTD.

Address: No.18 Qi Xing Road, Majiadian Town, 443200 Zhijiang City, Hubei Province, P.R.C

For and on behalf of
ALLMED MEDICAL PRODUCTS CO., LIMITED

Susan Liao

Authorized Signature(s)

Document No.:	Revision No.:	Date of Revision:	Approved by:
WP-011100-AL-02	R-002	2022.10.12	Susan Liao