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#### SAFETY CONTAINERS SQUARE SHAPE

Disposable container for sharps and cutting objects disposal. Three sizes square shape containers, solid and stable shape, with universal trolley handling system, vertical and horizontal. Lid with temporary and definitive closure, hermetic even if turned upside down. The lid, apart from having a wide open, is fitted with indents for the removal of needles. Provided with ergonomic handle. Maximum filling level indicator. All the models have temporary and definite closure and are fitted with solid handle.

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*Ecological plastic material (colored copolymer polypropylene compliant to REACH regulations and with masterbatch heavy metal free and cadmium free). Yellow body, yellow lid and red plug in order to signal the dangerous content.*

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FDA (Food and Drug Administration) Approval K071517

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Autoclavabile a +134°C per 18 minuti

*Quality certification NF302, Compliant to NF X 30-511*

*Compliant to ISO 23907:2012*

*Compliant to ADR - UN Certificate*

*FDA (Food and Drug Administration) Approval K071517*

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**ASSEMBLY:** Place the lid on the body with the "FRONT" signal in the side of the label, pressing the opposite corners and sides

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**TEMPORARY CLOSURE:** Turn the red plug and cover the hole on the lid. At this stage closure is temporary.

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**PERMANENT CLOSURE:** First, place the red plug in the temporary closure position.

Place the yellow cap in the "arrow against arrow" position and press firmly. At this stage closure is permanent

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*Turn the movable part of the lid clockwise, opening it according to the size of the object to dispose*

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*Close the container permanently and irreversibly once full*

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*Use the "V" removal device to separate needle and syringe,*

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**Hole for introduction of waste:** large orifice to allow the introduction among others, of tubes, scalpels, etc..

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*Use the product only with the provided lid*

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*Do not use the product before fitting the lid*

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*Do not reopen the closed container and sealed permanently*

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*Do not attempt to rescue objects already in the container*

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*Respect the filling level indicated*

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*Containers are tested to not being perforated by sharp objects allowing safety to the operators, as long as the container is used properly according to instructions*

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*Keep out of flame or heat sources which might damage the product*

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*Do not use after expiry date or if packing is opened*

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*Do not re-use: Disposable device*

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*Do not vary the intended purpose of the product*

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*Store in dry place, Temperature range: min -10°C max +50°C*

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*Disposal: use appropriate personal protective equipment and act according to applicable regulations*

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*Before use with particular substances check the resistance / compatibility chart on our catalogue*

## IMBALLO / PACKING

<b>CODICE CODE</b>	<b>Quantità (pz): Quantity (pcs):</b>	<b>Misura esterna scatola (cm): External box dimensions (cm):</b>	<b>Volume (m<sup>3</sup>): Volume (m<sup>3</sup>):</b>
<b>7003</b>	40	79 x 40 x 31	0,098
<b>7005</b>	40	79 x 53 x 48	0,201
<b>7010</b>	30	79 x 53 x 48	0,201