M860

Carucior si targa
– separabile


arequipment

EN 1865-1
10G

M860. Carucior si targa – separate.

- Structura din aluminiu
- 4 manere telescopice de 75 mm
- 4 roti cu diametru de 200mm (2 pivotante si 2 fixe)
- Spatar ajustabil in opt pozitii
- Suport picioare ajustabil in patru pozitii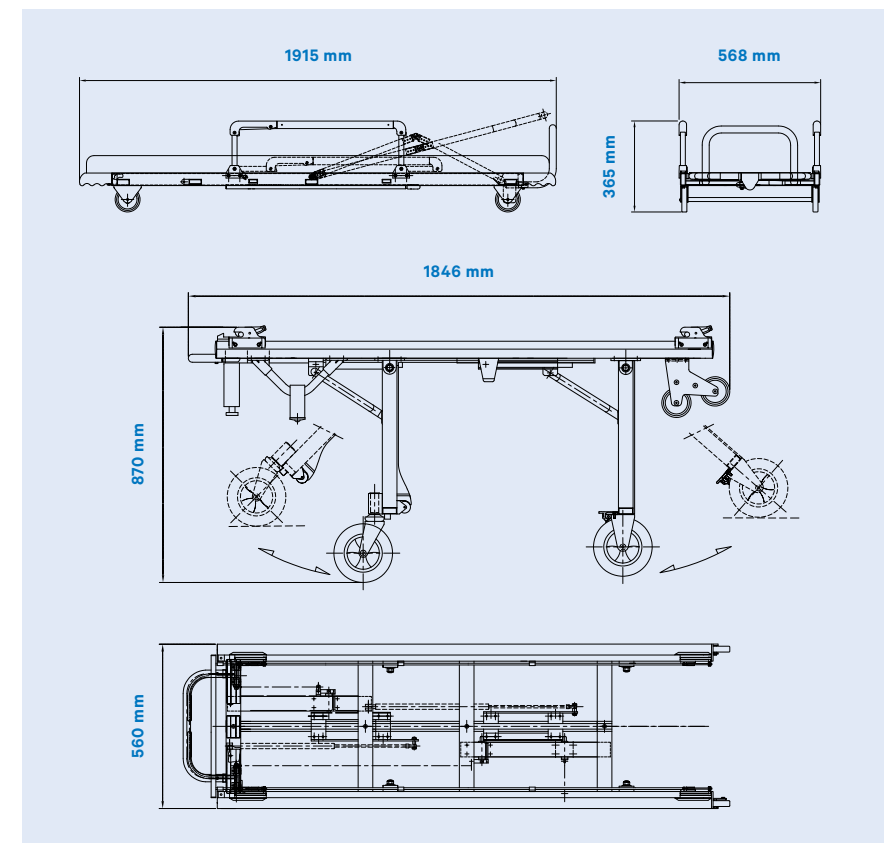

Made in Portugal

arequipment

EN 1865-1
10G

Greutate:

32,7 kg (carucior)

+ 18 kg (targa)

Capacitate maxima de incarcare:

250 kg

Dimensiuni carucior (inchis):

1846 x 560 x 455 mm

Dimensiuni carucior (deschis):

1846 x 560 x 870 mm

Dimensiuni targa:

1915 x 568 x 365 mm

Dotari:

Suport perfuzii

Manere laterale rabatabile

Centuri fixare mijloc pacient

Centuri fixare peste umar

Centuri de fixare picioare


Garantie:

2 ani

Rua de S. Caetano, 551 Apartado 526
4411-701 Canelas; Vila Nova de Gaia, Portugal
Telf: +351 227157100 , Fax: +351 2271244
autoribeiro@autoribeiro.pt
www.autoribeiro.pt

User's Manual: M860: Trolley and stretcher - Separable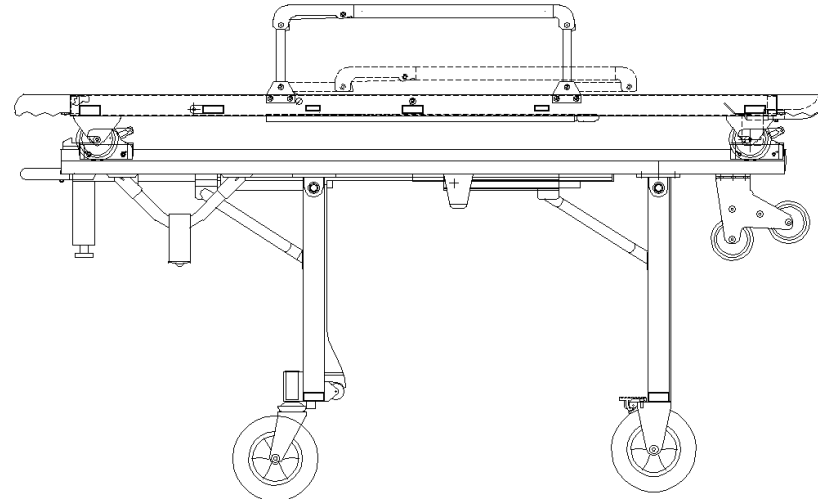

M860: Trolley / Stretcher - Separable

The trolley/stretcher, or M860, is an equipment with separable stretcher of manual drive. This equipment as for main purpose the transport of patients with mobility difficulties and post-operation condition. This product is developed and produced by Auto Ribeiro Lda.

Equipment's Overview

Equipment's Lateral view


ACESSORIES

IV pole

IV pole.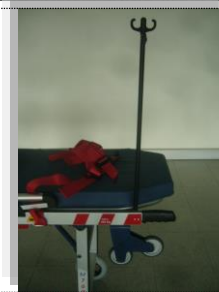

IV pole in use.

Instruments Tray

Instruments tray.

Instruments tray in use.

ACCESSORIES

Fixing Restrain Straps

Shoulder Harness restraints


Legs and feet restrains

Restraint Installation

1° - Tying restraint straps in inner bars.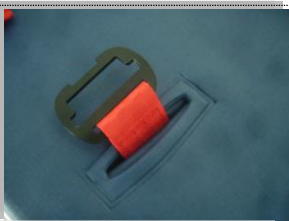

2° - Pass the restraints through the plate slots

3° - Join belt buckles..

Top view of Equipment

CHARACTERISTICS:

- Weight: 32,7 kg (trolley) + 18 kg (stretcher)
- Maximum load capacity: 250kg
- Trolley dimensions (closed): 1846 x 560 x 455 mm
- Trolley dimensions (open): 1846 x 560 x 870 mm
- Dimensions (stretcher): 1915 x 568 x 365 mm

FRONT LEG CONTROL (In mid position)

<u>Security System</u>	
	1° - Lift the lever (put in the right side of trolley).
	Final Position.

WHEELS

<u>Swivel Wheels</u>	
	2 Swivel wheels
<u>Fixed Wheels</u>	
	2 Fixed wheels
	Block of fixed wheels 1° - Engage the tabs with the foot to lock the wheels.

SAFETY WARNINGS

	This Equipment can be dangerous: Risk of entrapment ATTENTION: USE WITH CAUTION!
	 Risk of entrapment

MAINTENANCE

 - Perform visual inspection every fifty cycles of utilization. - Lubrication elements of mechanical function every six months with high viscosity lubricant - Checking of tightness of binding elements of every three months. Proceed to adjust if necessary

ADJUSTABLE FEET POSITION

Adjustable feet in three different positions:

Footrest and Knees-Flex Position

Lower Legs part Elevation (16° degrees)

Legs Elevation

BACK LEGS CONTROL

Security System

1° - Press the left lever until it moves and keep it pressed.

(Note: To put the back leg into halfway, should press the lever and release it immediately.

FRONT LEG CONTROL

Security System


1° - Press the right lever until it moves and keep it pressed.

STRETCHER BLOCK SYSTEM CONTROL

Security system

1° - Press the lever and keep it pressed.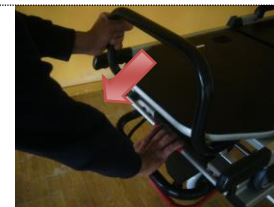

2° - Press the stretcher to release the trolley.

TELESCOPIC HANDLES

Security System

1° - Press button with finger.

2° - Push or pull to fit the desired length.

SWING DOWN SIDEARMS

Security System

1° - Press button with hand.

2° - Push or pull to achieved the desired opening position.

ADJUSTABLE BACKREST

The backrest can be adjusted in different positions:

Supine position

Semi-fowler's Position and Fowler Position

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EN 597-1:1994

Color palette



ARE Mattress Comfort on the way to recover

Compatible with all models AREquipment ,this 4 part ergonomic mattress was conceived specifically to be comfortable and to be safe.

- Recommended used in conjunction with Chest and Legs Patient Restraints
- No progressive smouldering or flaming ignition when tested in accordance with EN 597-1:1994
- The materials are not affected by disinfectants
- Easy-to-clean
- Comfortable
- Compatible with all models AREquipment

Dimensions

1880 x 510 mm

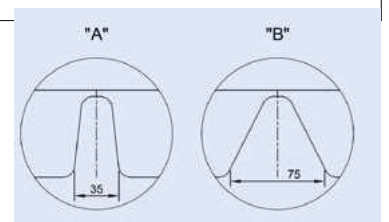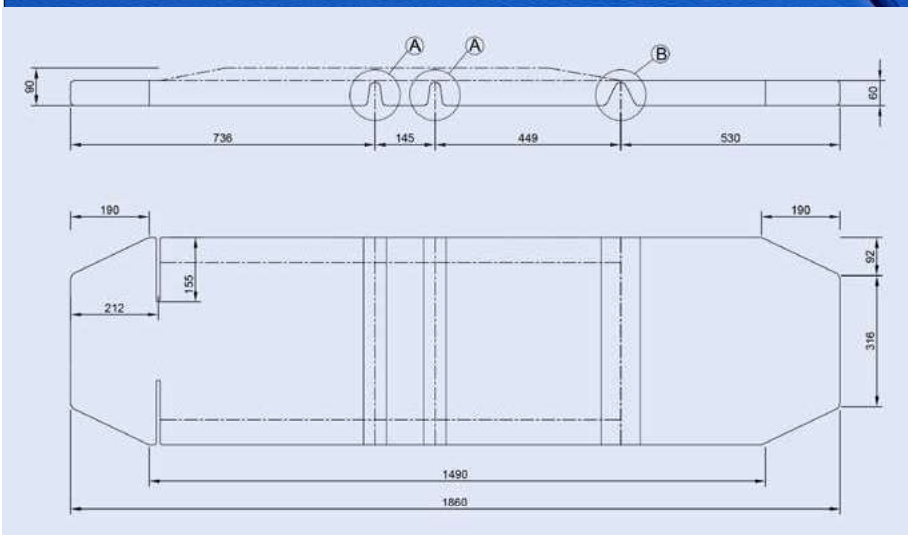EU DECLARATION OF CONFORMITY

For medical devices

according to Regulation (EU) 2017/745

Auto Ribeiro Lda., with main facilities at Rua S. Caetano, 459 and 519 – 4411-701 Canelas, Vila Nova de Gaia, Portugal, declares under its own responsibility that the following product according to REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 5 April 2017 and with the following mentioned standards.

Auto Ribeiro Lda. maintains a Quality Management System that fulfills the requirements of NP EN ISO 9001. Copies of Auto Ribeiro, Lda. ISO 9001 certificate issued by SGS are available upon request.

Product: Carrinho/Maca Amovível

Manufacturer: Auto Ribeiro Lda.

Model: M8

Risk Class of the Device: Class I

Device ID: 393141

Basic UDI-DI: 5600786656TROLLEY68
5600786656STRETCHERC5

CDM: 73573094

Based on Standards:

EN 1865-1	Patient handling equipment used in road ambulance. -Specification for general stretcher systems and patient handling equipment.
EN 1789	Medical vehicles and their equipment – Road Ambulances
EN ISO 9001	System of quality management

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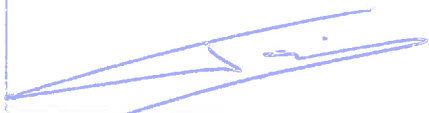
Modification added to the stretcher :

- No modification added to stretcher M760 and fixation E250.
- Product is identical to product cover with UTAC report 13/03264

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Modifications of EN 1865-1:2010 +A1:2015 standard based on EN1865-1:2010 standard does not concern paragraph 4.2.10 Fixation. The stretcher AUTORIBEIRO M760 and E250 fixation comply with paragraph 4.2.10 EN1865-1:2010 + A1:2015 standard.

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Modification added to the stretcher :

- No modification added to stretcher M860 and fixation E250.
- Product is identical to product cover with UTAC report 13/05648

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Modifications of EN 1865-1:2010 +A1:2015 standard based on EN1865-1:2010 standard does not concern paragraph 4.2.10 Fixation. The stretcher AUTORIBEIRO M860 and E250 fixation comply with paragraph 4.2.10 EN1865-1:2010 + A1:2015 standard.

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Modification added to the stretcher :

- *No modification added to stretcher M760 and fixation E250.*
- *Product is identical to product cover with UTAC report 14/07567*

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Modifications of EN 1865-1:2010 +A1:2015 standard based on EN1865-1:2010 standard does not concern paragraph 4.2.10 Fixation. The stretcher AUTORIBEIRO M764 4SW and E250 fixation comply with paragraph 4.2.10 EN1865-1:2010 +A1: 2015 standard.

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