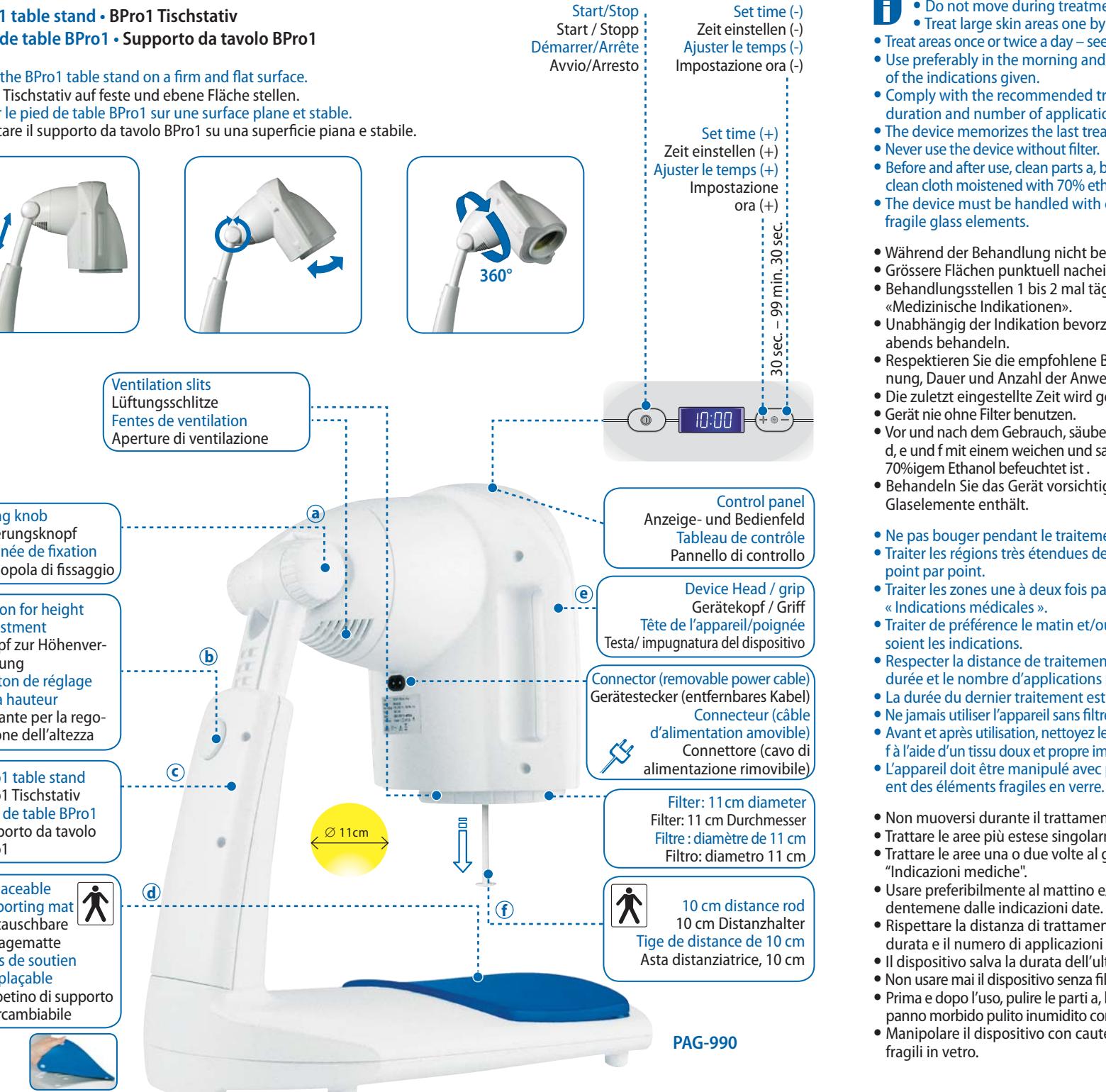
## 1 Description of Device • Gerätbeschreibung Description de l'appareil • Descrizione del dispositivo

## 2 BIOPTRON Light Therapy treatment • BIOPTRON Lichttherapie Behandlung • Traitement par photothérapie BIOPTRON • Trattamento fototerapico BIOPTRON

**Application flow of a treatment • Ablauf einer Behandlung**  
**Exécution du traitement • Svolgimento di un trattamento**

- Carefully clean the area to be treated.** Behandlungsbereich vorsichtig reinigen. Nettoyez soigneusement la zone à traiter. Pulire attentamente l'area che si intende trattare.
- Hold the device at 90°, for distance (scale below) and duration refer to "3| Medical Indications".** Im rechten Winkel (90°) behandeln, für Distanz (Massstab unten) und Dauer beziehen Sie sich auf "3| Medizinische Indikationen". Tenez l'appareil à 90°. Pour connaître la distance (échelle ci-dessous) et la durée, reportez-vous à la section "3| Indications médicales". Tenere il dispositivo inclinato di 90°, per la distanza (righe in basso) e la durata, fare riferimento a "3| Indicazioni mediche".
- Set the treatment time.** Behandlungszeit einstellen. Réglez la durée du traitement. Imposta la durata del trattamento.
- Press start.** Start drücken. Appuyez sur Démarrer. Premere start.

**Error messages • Fehlernachrichten**  
**Messages d'erreur • Messaggi d'errore**

**i** When the device cools down, the «Hot» sign will disappear. Leave unplugged until the sign disappears. If «Err» message appears, unplug and replug after 1 min. If the error message persists contact your local distributor.

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**BIOPTRON improves microcirculation • Harmonizes metabolic processes • Reinforces the body's defence system • Stimulates regenerative and reparative processes in the entire organism - these basic mechanisms are responsible for the therapeutic healing effects as complementary treatment of the following medical conditions:**

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**IN PEDIATRICS: APPLICATIONS ON CHILDREN UNDER 6 YEARS OLD MUST BE PERFORMED UNDER SUPERVISION OF A PHYSICIAN.** Recommended treatment time 4-10 min, distance 10 cm.  
• Pediatric dermal affections • Endogenous eczema • Upper respiratory tract infections • Allergic respiratory diseases • Pediatric musculoskeletal and neurological disorders and deficits

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**DOULEUR : APPLICATIONS MUST BE PERFORMED UNDER SUPERVISION OF A PHYSICIAN:** Recommended treatment time 4-10 min, distance 10 cm.  
• Dermal affections as phlebitis, decubitus, intertrigo.

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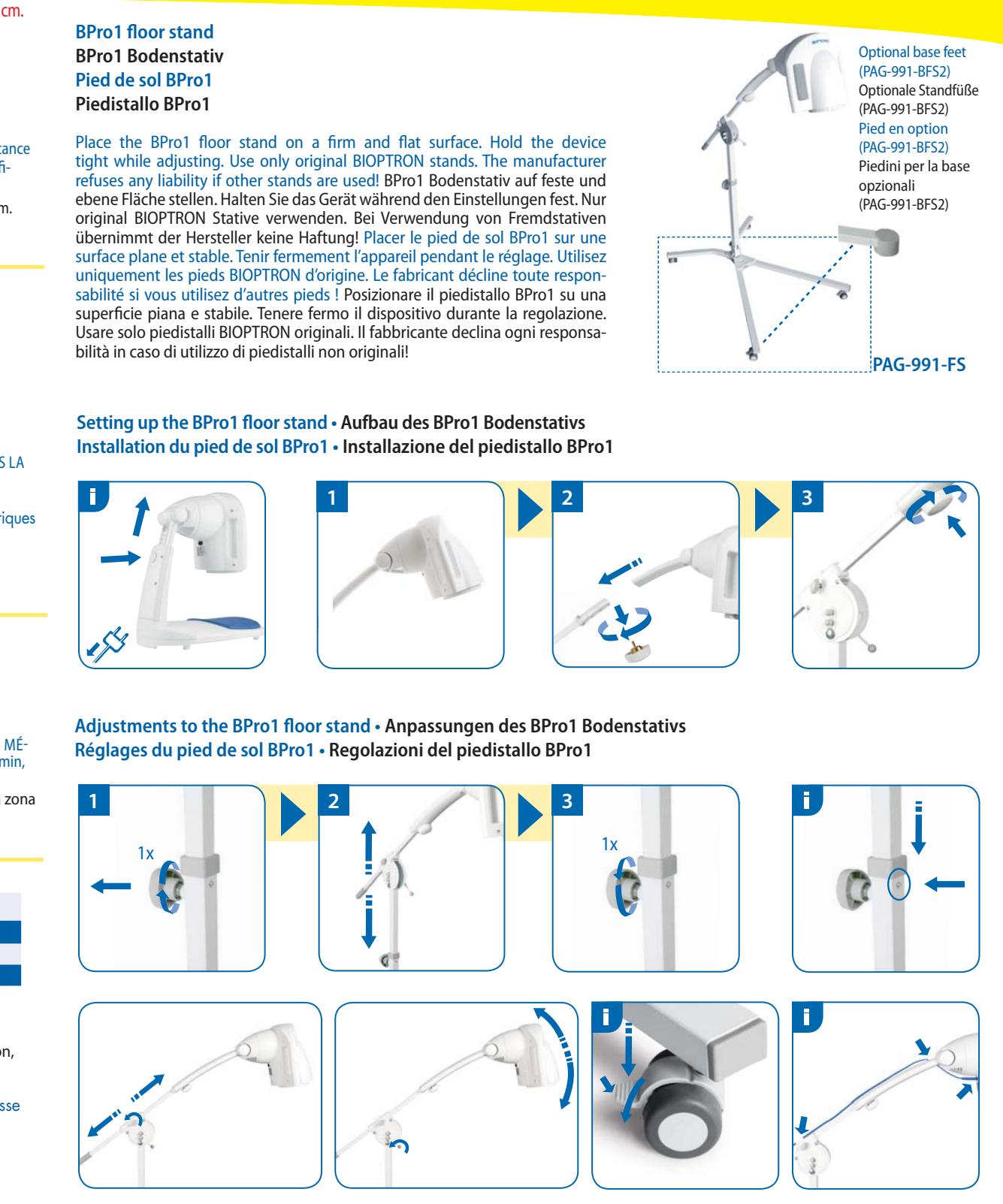
This device may become a safety hazard if these instructions/safety precautions are ignored.

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**Danger of electric shock! Keep dry! NEVER immerse in water! Do not open!**

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**Never block ventilation slits on the device.**

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**Never operate a damaged device (cables, plugs, housing or optical component).**

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**Risk of electric shock. Unplug the device if any liquids or particles enter the device. Contact your service center.**

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**Before use consult your doctor to see if this therapy is suitable for the intended treatment purpose.**

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**For the treatment of newborns use blindfold (PAG-500-BF).**

Augenbinde für die Behandlung von Neugeborenen verwenden (PAG-500-BF). Pour le traitement des nouveau-nés, utiliser un bandeau (PAG-500-BF).

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**Repairs and replacements of the halogen bulb should only be conducted by BIOPTRON-authorised service centres!**

Do not carry out repairs yourself! No modification of this equipment is allowed! Danger if non-genuine substitute parts are used. The expected operational lifetime for the BPro1's device is 5 years and 10 years for the floor stand accessory. After that time, a maintenance service should be carried out.

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Do not look directly into light source. If not respected eye damage may occur. For the treatment of the eye area remove contact lenses and keep eyes closed.

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The power cord poses a strangulation hazard to children.

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The power cord is used to disconnect the device from the mains power supply.

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Applications on children and people with cognitive impairment always need to be performed under adult supervision.

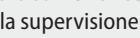
Applications on children under six years old and on newborns must be performed under supervision of a physician.

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BIOPTRON devices can be used by clients with heart implants and pacemakers.

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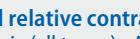
For the treatment of newborns use blindfold (PAG-500-BF).

Augenbinde für die Behandlung von Neugeborenen verwenden (PAG-500-BF). Pour le traitement des nouveau-nés, utiliser un bandeau (PAG-500-BF).

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Repairs and replacements of the halogen bulb should only be conducted by BIOPTRON-authorised service centres!

Do not carry out repairs yourself! No modification of this equipment is allowed! Danger if non-genuine substitute parts are used. The expected operational lifetime for the BPro1's device is 5 years and 10 years for the floor stand accessory. After that time, a maintenance service should be carried out.

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
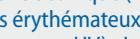
Find further information in the Technical Description.

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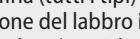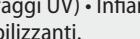
Unplug the device when cleaning. Do not use any liquid. Keep away from dust. Do not vibrate or shake.

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**The recommended minimum distance of the device BPro1 to portable and mobile RF communication equipments depends on their transmission power and operation frequency. The distance can be taken from the below table.**

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