

## ORGENTEC Diagnostika GmbH


Carl-Zeiss-Straße 49-51

55129 Mainz - Germany

Phone: +49 (0) 61 31 / 92 58-0

Fax: +49 (0) 61 31 / 92 58-58

Internet: [www.orgentec.com](http://www.orgentec.com)

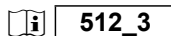

## ORG 512 Anti-Sci-70

### INTENDED PURPOSE

Anti-Sci-70 is an ELISA test system for the quantitative measurement of IgG class autoantibodies against Sci-70 in human serum or plasma. This product is intended for professional in vitro diagnostic use only.

Antibodies against Sci-70 (DNA topoisomerase I) are an accepted marker for progressive systemic sclerosis. They contribute to the differential diagnosis of sclerosis. Evaluation of a test result should always take into account all clinical and laboratory diagnostic findings.

### SYMBOLS USED ON LABELS



In vitro diagnostic medical device



Manufacturer



Catalogue number



Sufficient for 96 determinations



Batch code



Use by



Temperature limitation



Keep away from sunlight



Do not reuse



Date of manufacture



CE marked according to 98/79/EC




Consult instructions for use



Electronic Instruction For Use: version




Microplate

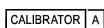

Calibrator




Calibrator

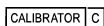


Calibrator



Calibrator

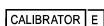

Calibrator

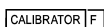

Calibrator



Control positive



Control negative



Sample Buffer P



Enzyme Conjugate



TMB Substrate



Stop solution



Wash Buffer



Ready to use

### PRINCIPLE OF THE TEST

Highly purified Sci-70 is bound to microwells.

The determination is based on an indirect enzyme linked immune reaction with the following steps:

Specific antibodies in the patient sample bind to the antigen coated on the surface of the reaction wells. After incubation, a washing step removes unbound and unspecifically bound serum or plasma components. Subsequently added enzyme conjugate binds to the immobilized antibody-antigen-complexes. After incubation, a second washing step removes unbound enzyme conjugate. After addition of substrate solution the bound enzyme conjugate hydrolyses the substrate forming a blue coloured product. Addition of an acid stops the reaction generating a yellow end-product. The intensity of the yellow color correlates with the concentration of the antibody-antigen-complex and can be measured photometrically at 450 nm.



### WARNINGS AND PRECAUTIONS

- All reagents of this kit are intended for professional in vitro diagnostic use only.
- Components containing human serum were tested and found negative for HBsAg, HCV, HIV1 and HIV2 by FDA approved methods. No test can guarantee the absence of HBsAg, HCV, HIV1 or HIV2, and so all human serum based reagents in this kit must be handled as though capable of transmitting infection.
- Bovine serum albumin (BSA) used in components has been tested for BSE and found negative.
- Avoid contact with the substrate TMB (3,3',5,5'-Tetramethyl-benzidine).
- Stop solution contains acid, classification is non-hazardous. Avoid contact with skin.
- Control, sample buffer and wash buffer contain sodium azide 0.09% as preservative. This concentration is classified as non-hazardous.
- Enzyme conjugate contains ProClin 300 0.05% as preservative. This concentration is classified as non-hazardous.

During handling of all reagents, controls and serum samples observe the existing regulations for laboratory safety regulations and good laboratory practice:

- First aid measures: In case of skin contact, immediately wash thoroughly with water and soap. Remove contaminated clothing and shoes and wash before reuse. If system fluid comes into contact with skin, wash thoroughly with water. After contact with the eyes carefully rinse the opened eye with running water for at least 10 minutes. Get medical attention if necessary.
  - Personal precautions, protective equipment and emergency procedures:  
Observe laboratory safety regulations. Avoid contact with skin and eyes. Do not swallow. Do not pipette by mouth. Do not eat, drink, smoke or apply makeup in areas where specimens or kit reagents are handled. When spilled, absorb with an inert material and put the spilled material in an appropriate waste disposal.
  - Exposure controls / personal protection: Wear protective gloves of nitril rubber or natural latex. Wear protective glasses. Used according to intended use no dangerous reactions known.
  - Conditions to avoid: Since substrate solution is light-sensitive. Store in the dark.
  - For disposal of laboratory waste the national or regional legislation has to be observed.
- Observe the guidelines for performing quality control in medical laboratories by assaying control sera.

## CONTENTS OF THE KIT

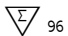ORG 512		96	Sufficient for 96 determinations
<b>MICROPLATE</b>		1	One divisible microplate consisting of 12 modules of 8 wells each. Ready to use. Color code on module
<b>CALIBRATOR A</b>	1x 1.5 ml		Calibrator A 0 U/ml, containing serum/buffer matrix (PBS, BSA, detergent, NaN3 0.09%), yellow. Ready to use.
<b>CALIBRATOR B</b>	1x 1.5 ml		Calibrator B 12.5 U/ml, containing Scl-70 antibodies in a serum/buffer matrix (PBS, BSA, detergent, NaN3 0.09%), yellow. Ready to use.
<b>CALIBRATOR C</b>	1x 1.5 ml		Calibrator C 25 U/ml, containing Scl-70 antibodies in a serum/buffer matrix (PBS, BSA, detergent, NaN3 0.09%), yellow. Ready to use.
<b>CALIBRATOR D</b>	1x 1.5 ml		Calibrator D 50 U/ml, containing Scl-70 antibodies in a serum/buffer matrix (PBS, BSA, detergent, NaN3 0.09%), yellow. Ready to use.
<b>CALIBRATOR E</b>	1x 1.5 ml		Calibrator E 100 U/ml, containing Scl-70 antibodies in a serum/buffer matrix (PBS, BSA, NaN3 0.09%), yellow. Ready to use.
<b>CALIBRATOR F</b>	1x 1.5 ml		Calibrator F 200 U/ml, containing Scl-70 antibodies in a serum/buffer matrix (PBS, BSA, detergent, NaN3 0.09%), yellow. Ready to use.
<b>CONTROL +</b>	1x 1.5 ml		Control positive, containing Scl-70 antibodies in a serum/buffer matrix (PBS, BSA, detergent, NaN3 0.09%), yellow. Ready to use. The concentration is specified on the certificate of analysis.
<b>CONTROL -</b>	1x 1.5 ml		Control negative, containing Scl-70 antibodies in a serum/buffer matrix (PBS, BSA, detergent, NaN3 0.09%), yellow. Ready to use. The concentration is specified on the certificate of analysis.
<b>DILUENT</b>	20 ml		Sample Buffer P, containing PBS, BSA, detergent, preservative sodium azide 0.09%, yellow, concentrate (5 x).
<b>CONJUGATE</b>	15 ml		Enzyme Conjugate containing anti-human IgG antibodies, HRP labelled; PBS, BSA, detergent, preservative PROCLIN 0.05%, light red. Ready to use.
<b>TMB</b>	15 ml		TMB Substrate; containing 3,3', 5,5'- Tetramethylbenzidin, colorless. Ready to use.
<b>STOP</b>	15 ml		Stop solution; contains acid. Ready to use.
<b>WASH</b>	20 ml		Wash Buffer, containing Tris, detergent, preservative sodium azide 0.09%; 50 x conc.
	1		Certificate of Analysis

## MATERIALS REQUIRED

- Microplate reader capable of endpoint measurements at 450 nm; optional: reference filter at 620 nm
- Data reduction software
- Multi-channel dispenser or repeatable pipette for 100 µl
- Vortex mixer
- Pipettes for 10 µl, 100 µl and 1000 µl
- Laboratory timing device
- Distilled or deionised water
- Measuring cylinder for 1000 ml and 100 ml
- Plastic container for storage of the wash solution

This ELISA assay is suitable for use on open automated ELISA processors. Each assay has to be validated on the respective automated system. Detailed information is provided upon request.

## SPECIMEN COLLECTION, STORAGE AND HANDLING

- Collect whole blood specimens using acceptable medical techniques to avoid hemolysis.
- Allow blood to clot and separate the serum or plasma by centrifugation.
- Test serum should be clear and non-hemolyzed. Contamination by hemolysis or lipemia should be avoided, but does not interfere with this assay.
- Specimens may be refrigerated at 2-8°C for up to five days or stored at -20°C up to six months.
- Avoid repetitive freezing and thawing of serum or plasma samples. This may result in variable loss of antibody activity.
- Testing of heat-inactivated sera is not recommended.

## STORAGE AND STABILITY

- Store test kit at 2-8°C in the dark.
- Do not expose reagents to heat, sun, or strong light during storage and usage.
- Store microplate sealed and dessicated in the clip bag provided.
- Shelf life of the unopened test kit is 18 months from day of production.  
Unopened reagents are stable until expiration of the kit. See labels for individual batch.
- Diluted Wash Buffer and Sample Buffer are stable for at least 30 days when stored at 2-8°C.  
We recommend consumption on the same day.

## PROCEDURAL NOTES

- Do not use kit components beyond their expiration dates.
- Do not interchange kit components from different lots and products.
- All materials must be at room temperature (20-28°C) prior to use.
- Prepare all reagents and samples. Once started, perform the test without interruption.
- Double determinations may be done. By this means pipetting errors may become obvious.
- Perform the assay steps only in the order indicated.
- Always use fresh sample dilutions.
- Pipette all reagents and samples into the bottom of the wells.
- To avoid carryover or contamination, change the pipette tip between samples and different kit controls.
- Wash microwells thoroughly and remove the last droplets of wash buffer.
- All incubation steps must be accurately timed.
- Do not re-use microplate wells.

## PREPARATION OF REAGENTS

### WASH

Dilute the contents of one vial of the buffered wash solution concentrate (50x) with distilled or deionised water to a final volume of 1000 ml prior to use.

### DILUENT

Sample Buffer P: Prior to use dilute the contents (20 ml) of one vial of sample buffer 5x concentrate with distilled or deionised water to a final volume of 100 ml.

### Preparation of samples

Dilute patient samples 1:100 before the assay: Put 990 µl of prediluted sample buffer in a polystyrene tube and add 10 µl of sample. Mix well. Note: Calibrators / Controls are ready to use and need not be diluted.

## TEST PROCEDURE

Prepare enough microplate modules for all calibrators / controls and patient samples.

- Pipette **100 µl** of calibrators, controls and prediluted patient samples into the wells.  
Incubate for **30 minutes** at room temperature (20-28 °C).  
Discard the contents of the microwells and **wash 3 times** with **300 µl** of wash solution.
- Dispense **100 µl** of enzyme conjugate into each well.  
Incubate for **15 minutes** at room temperature.  
Discard the contents of the microwells and **wash 3 times** with **300 µl** of wash solution.
- Dispense **100 µl** of TMB substrate solution into each well.  
Incubate for **15 minutes** at room temperature
- Add 100 µl** of stop solution to each well of the modules  
Incubate for **5 minutes** at room temperature.  
Read the optical density at 450 nm (reference 600-690nm) and calculate the results.  
The developed colour is stable for at least 30 minutes. Read during this time.

Example for a pipetting scheme:

	1	2	3	4	5	6	7	8	9	10	11	12
A	A	P1										
B	B	P2										
C	C	P3										
D	D											
E	E											
F	F											
G	C+											
H	C-											

P1, ... patient sample    A-F calibrators    C+, C- controls

## VALIDATION

Test results are valid if the optical densities at 450 nm for calibrators / controls and the results for controls comply with the reference ranges indicated on the Certificate of Analysis enclosed in each test kit.  
If these quality control criteria are not met the assay run is invalid and should be repeated.

## CALCULATION OF RESULTS

For quantitative results plot the optical density of each calibrator versus the calibrator concentration to create a calibration curve. The concentration of patient samples may then be estimated from the calibration curve by interpolation.  
Using data reduction software a 4-Parameter-Fit with lin-log coordinates for optical density and concentration is the data reduction method of choice.

## PERFORMANCE CHARACTERISTICS

### Calibration

The assay system is calibrated against the internationally recognized reference sera from CDC, Atlanta USA.

### Measuring range

The calculation range of this ELISA assay is        0 - 200 U/ml

### Expected values

In a normal range study with samples from healthy blood donors the following ranges have been established with this ELISA assay: Cut-off                                25 U/ml

### Interpretation of results

Negative:                < 15 U/ml  
Borderline:            15 - 25 U/ml  
Positive:                > 25 U/ml

## Linearity

Patient samples containing high levels of specific antibody were serially diluted in sample buffer to demonstrate the dynamic range of the assay and the upper / lower end of linearity. Activity for each dilution was calculated from the calibration curve using a 4-Parameter-Fit with lin-log coordinates.

Sample	Dilution	Observed U/ml	Expected U/ml	O/E [%]
1	1:100	146.9	146.9	100
	1:200	76.3	73.5	104
	1:400	38.1	36.7	104
	1:800	18.8	18.4	102
	1:100	122.3	122.3	100
	1:200	60.4	61.2	99
	1:400	29.6	30.6	97
	1:800	14.8	15.3	97

## Limit of detection

Functional sensitivity was determined to be:                1 U/ml

## Reproducibility

Intra-assay precision: Coefficient of variation (CV) was calculated for each of three samples from the results of 24 determinations in a single run. Results for precision-within-assay are shown in the table below.  
Inter-assay precision: Coefficient of variation (CV) was calculated for each of three samples from the results of 6 determinations in 5 different runs. Results for run-to-run precision are shown in the table below.

Intra-Assay		
Sample	Mean U/ml	CV %
1	45.7	4.0
2	90.4	3.2
3	184.1	3.4

Inter-Assay		
Sample	Mean U/ml	CV %
1	41.1	2.8
2	89.9	2.8
3	157.4	2.3

## Interfering substances

No interference has been observed with haemolytic (up to 1000 mg/dl) or lipemic (up to 3 g/dl triglycerides) sera or plasma, or bilirubin (up to 40 mg/dl) containing sera or plasma. Nor have any interfering effects been observed with the use of anticoagulants (Citrate, EDTA, Heparine). However for practical reasons it is recommended that grossly hemolyzed or lipemic samples should be avoided.

## Study results

<u>Study population</u>	<u>n</u>	<u>n Pos</u>	<u>%</u>
Scleroderma	25	19	76.0
Rheumatoid arthritis	20	0	0.0
Normal human sera	80	1	1.3

		Clinical Diagnosis		
		POS	NEG	
ORG 512	POS	19	1	25      100      125
	NEG	6	99	
Sensitivity:		76.0	%	
Specificity:		99.0	%	
Overall agreement:		94.4	%	

## LIMITATIONS OF THE PROCEDURE

This assay is a diagnostic aid. A definite clinical diagnosis should not be based on the results of a single test, but

should be made by the physician after all clinical and laboratory findings have been evaluated concerning the entire clinical picture of the patient. Also every decision for therapy should be taken individually.

The above pathological and normal reference ranges for antibodies in patient samples should be regarded as recommendations only. Each laboratory should establish its own ranges according to ISO 15189 or other applicable laboratory guidelines.

## REFERENCES

1. Alba P, Bento L, Cuadrado MJ, Karim Y, Tungekar MF, Abbs I et al. Anti-dsDNA, anti-Sm antibodies, and the lupus anticoagulant: significant factors associated with lupus nephritis. *Ann Rheum Dis* 2003; 62(6):556-560.
2. Antico A, Platzgummer S, Bassetti D, Bizzaro N, Tozzoli R, Villalta D. Diagnosing systemic lupus erythematosus: new-generation immunoassays for measurement of anti-dsDNA antibodies are an effective alternative to the Farr technique and the Crithidia luciliae immunofluorescence test. *Lupus* 2010; 19(8):906-912.
3. Brouwer R, Hengstman GJ, Vree EW, Ehrfeld H, Bozic B, Ghirardello A et al. Autoantibody profiles in the sera of European patients with myositis. *Ann Rheum Dis* 2001; 60(2):116-123.
4. Castro C, Gourley M. Diagnostic testing and interpretation of tests for autoimmunity. *J Allergy Clin Immunol* 2010; 125(2 Suppl 2):S238-S247.
5. Defendenti C, Atzeni F, Spina MF, Grosso S, Cereda A, Guercilena G et al. Clinical and laboratory aspects of Ro/SSA-52 autoantibodies. *Autoimmun Rev* 2011; 10(3):150-154.
6. Eriksson C, Kokkonen H, Johansson M, Hallmans G, Wadell G, Rantapaa-Dahlqvist S. Autoantibodies predate the onset of Systemic Lupus Erythematosus in northern Sweden. *Arthritis Research & Therapy* 2011; 13(1):R30.
7. Haugbro K, Nossent JC, Winkler T, Figenschau Y, Rekvig OP. Anti-dsDNA antibodies and disease classification in antinuclear antibody positive patients: the role of analytical diversity. *Ann Rheum Dis JID* - 0372355 2004; 63(4):386-394.
8. Ippolito A, Wallace DJ, Gladman D, Fortin PR, Urowitz M, Werth V et al. Autoantibodies in systemic lupus erythematosus: comparison of historical and current assessment of seropositivity. *Lupus* 2011; 20(3):250-255.
9. Isenberg DA, Manson JJ, Ehrenstein MR, Rahman A. Fifty years of anti-ds DNA antibodies: are we approaching journey's end? *Rheumatology (Oxford)* 2007; 46(7):1052-1056.
10. Kattah NH, Kattah MG, Utz PJ. The U1-snRNP complex: structural properties relating to autoimmune pathogenesis in rheumatic diseases. *Immunol Rev* 2010; 233(1):126-145.
11. Kumar Y, Bhatia A, Minz RW. Antinuclear antibodies and their detection methods in diagnosis of connective tissue diseases: a journey revisited. *Diagn Pathol* 2009; 4:1.
12. Meroni PL, Schur PH. ANA screening: an old test with new recommendations. *Ann Rheum Dis* 2010; 69:1420-1422.
13. Petri M, Magder L. Classification criteria for systemic lupus erythematosus: a review. *Lupus* 2004; 13(11):829-837.
14. Poole BD, Schneider RI, Guthridge JM, Vette CA, Reichlin M, Harley JB et al. Early targets of nuclear RNP humoral autoimmunity in human systemic lupus erythematosus. *Arthritis Rheum* 2009; 60(3):848-859.
15. Putova I, Dostal C, Becvar R. Prevalence of antinucleosome antibodies by enzyme-linked immunosorbent assays in patients with systemic lupus erythematosus and other autoimmune systemic diseases. *Ann N Y Acad Sci* 2007; 1109:275-286.
16. Reveille JD. Predictive value of autoantibodies for activity of systemic lupus erythematosus. *Lupus JID* - 9204265 2004; 13(5):290-297.
17. Simon JA, Cabiedes J, Ortiz E, Alcocer-Varela J, Sanchez-Guerrero J. Anti-nucleosome antibodies in patients with systemic lupus erythematosus of recent onset. Potential utility as a diagnostic tool and disease activity marker. *Rheumatology (Oxford)* 2004; 43(2):220-224.
18. Sinclair D, Saas M, Williams D, Hart M, Goswami R. Can an ELISA replace immunofluorescence for the detection of anti-nuclear antibodies?--The routine use of anti-nuclear antibody screening ELISAs. *Clin Lab* 2007; 53(3-4):183-191.
19. Tozzoli R, Bizzaro N, Tonutti E, Villalta D, Bassetti D, Manoni F et al. Guidelines for the laboratory use of autoantibody tests in the diagnosis and monitoring of autoimmune rheumatic diseases. *Am J Clin Pathol* 2002; 117(2):316-324.
20. Maidhof W., Hilius O. Lupus: an overview of the disease and management options. *P T* 2012; 37(4):240-9.
21. Hahn BH, McMahon MA, Wilkinson A, Wallace WD, Daikh DI, Fitzgerald JD et al. American College of Rheumatology guidelines for screening, treatment, and management of lupus nephritis. *Arthritis Care Res (Hoboken)* 2012; 64(6):797-808.

Notice to the user (European Union):

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the

competent authority of the EU Member State in which the user and/or the patient is established .

Change Control

Former version: *ORG 512\_IFU\_EN\_QM113139\_2013-12-16\_1.2*

Reason for revision: *Introduction electronic IFU on homepage*

- ① Pipet **100 µl** calibrator, control or patient sample
  - Incubate for **30 minutes** at room temperature
  - Discard the contents of the wells and wash 3 times with **300 µl** wash solution
- ② Pipet **100 µl** enzyme conjugate
  - Incubate for **15 minutes** at room temperature
  - Discard the contents of the wells and wash 3 times with **300 µl** wash solution
- ③ Pipet **100 µl** substrate solution
  - Incubate for **15 minutes** at room temperature
- ④ Add **100 µl** stop solution
  - Leave untouched for **5 minutes**
  - Read at **450 nm**