




**SILVAN
MEDICAL**

CE Certified (Class III)

Clothoid Vascular Closure System

Clothoid Vascular Closure System is a suture mediated device to close femoral artery puncture site with a single monofilament polypropylene following diagnostic or interventional catheterization procedures.

- ✓ One-person operation
- ✓ Easy access to puncture site
- ✓ Pre-knotting

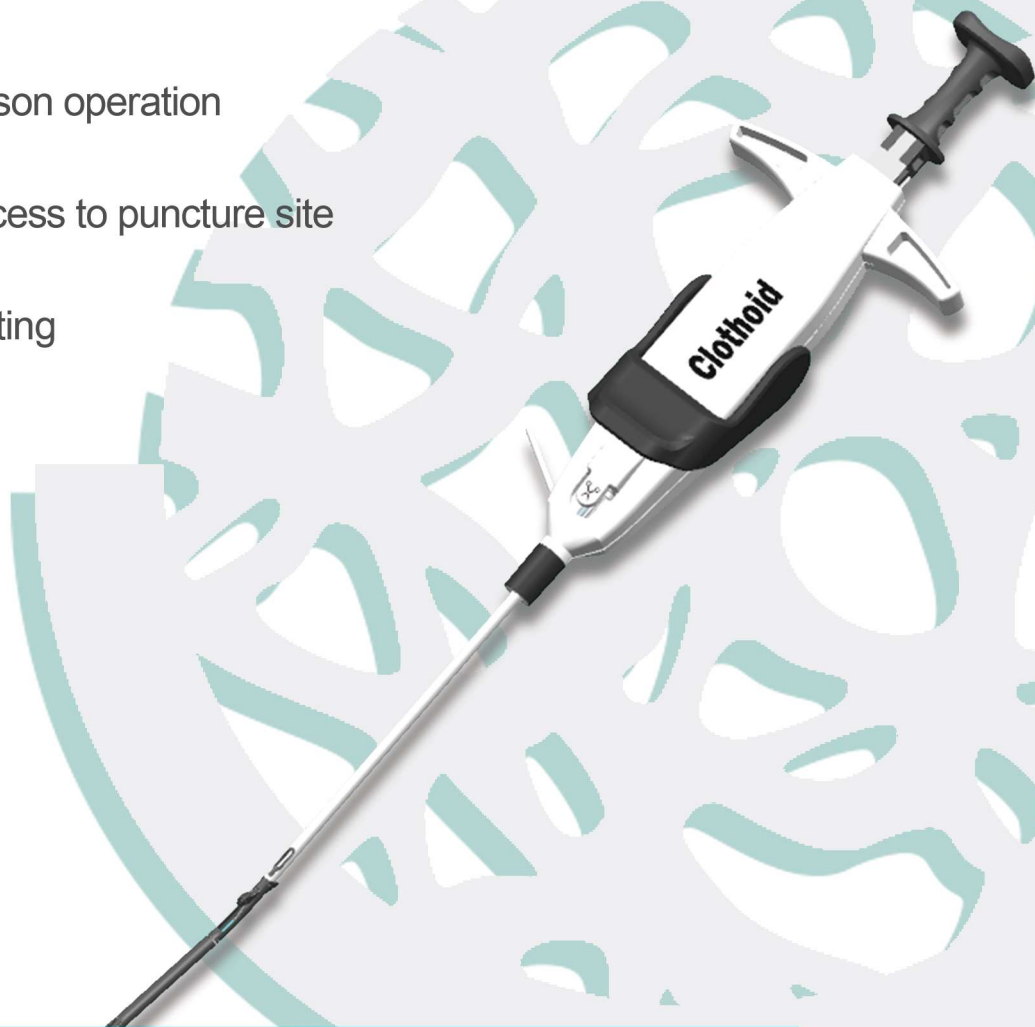

SUZHOU SILVAN MEDICAL CO., LTD

 Suzhou Industrial Park, China




 www.silvanmedical.com







The Clothoid Vascular Closure System is designed for patients who have undergone diagnostic or interventional catheterization procedures using 5F to 21F sheaths. The pre-knot polypropylene suture is delivered percutaneously to close the common femoral artery access site. At least two devices are needed to close arteriotomy sites using an 8-21F procedural sheath.



1. Open the foot

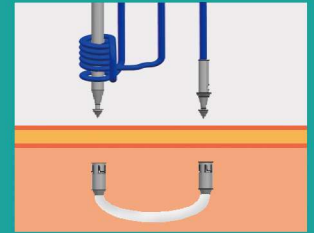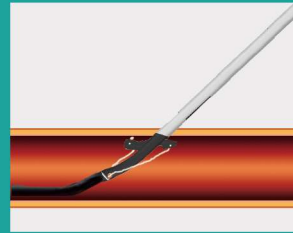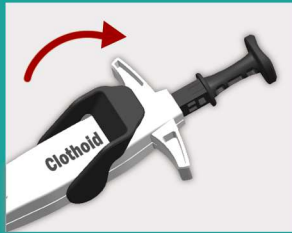



2. Push the needle plunger

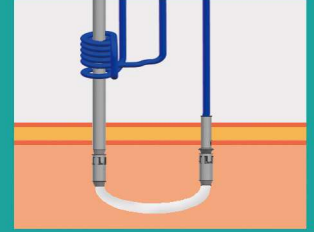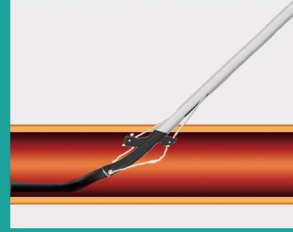

3. Pull out the needles

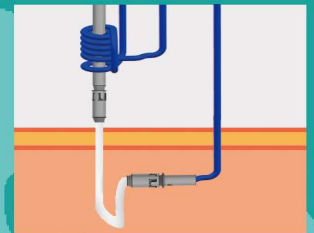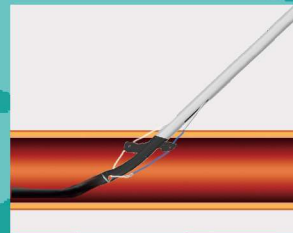

4. Push the knot and cut the suture

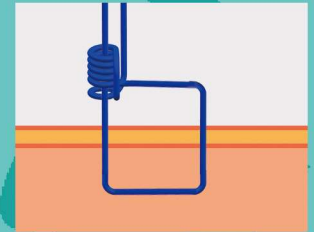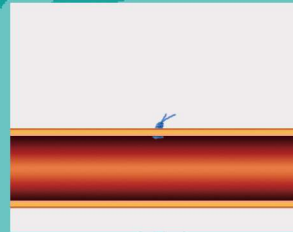

Order Information:

Item	Reference No.	Product Name
1	FS910001	Clothoid Vascular Closure System
2	FN00018	Suture Pusher and Cutter

Different Hemostasis Methods

By Compression

- 30 mins of compression;
- Pressure dressing after operation, patients stay in bed for 24 hours;
- Complications of hematoma, pseudoaneurysm and vagal nerve reflex.

v.s.

By Clothoid Vascular Closure System

- Confirmed during operation;
- Patients can walk immediately after the operation;
- Reduce the time on painkillers and inserting catheters.

EASY
SAFE
CONTROLLED