

DATA SHEET



Art. 2150 - Paediatric limb holder

Features

The limb holder is designed to limit and control the movements of confused patients. The Velcro closure allows for easy application and quick release of the device.

Indications

The device is intended for the application on the upper (wrists) or lower (ankles) limbs only. Not to be used to limit other body parts.

Target

Operating rooms, hospital wards, ICU, emergency rooms.

Storage

Store in a dry environment at room temperature. Protect from direct sun rays.

Contraindications

The device is not indicated for agitated, psychiatric, combative and suicidal patients. Do not use on limbs affected by fractures, dislocations or open wounds.

Do not use the device if I.V. lines can be compromised.

Warning

Do not apply the device too tightly or too loose: leave enough space to insert one finger. Patients conditions must be surveyed frequently. Check blood circulation and monitor skin color often.

Technical datas	
Material	TNT, Velcro, velour, cotton/polyester, foam, thread
Finishing	Welding and machine stitching
Laundering instructions	Wash at max. 40°C with water and mild soap
Strap dimensions	Length cm 152
	Width cm 2
Cuff dimensions	Length cm 15
	Width cm 3
Foam dimensions	Length cm 15
	Width cm 3

Sales unit: box of 60 pcs.

Instruction for use



Place the upper (wrist) or lower (ankle) limb in the middle of the protective pad. Ensure that the strap is toward the skin.



Wrap the wristlet end without the Velcro around the limb first.



Wrap the wristlet end with Velcro around the limb.



Fix the wristlet with the Velcro closure and secure the straps to a rigid bed or wheelchair frame.

Class I Medical Device

Coremec S.r.l. 2002 Copyright - All rights reserved

COREMEC S.r.I

LATEX FREE

Administration and Logistic Corso Vittime del Lavoro, n. 9 - 10042 Nichelino (TO) Tel. 011-35.89.481 - Fax 011-35.89.483 www.coremec.it



COREMEC S.r.I.

Export Department

e-mail: export@coremec.it