


5.4 Changing the Radiography Conditions

Press buttons or keys on the touch panel to change the radiography conditions.

- **Technique**.....⑧
- **Focus**.....②
- **kV**.....①
- **Switch mA/mAs**④
- **mA/mAs**.....③
- **sec**⑤
- **AEC (optional)**
 - AEC ON/OFF⑥
 - Density⑦
 - Photo pickup field.....⑨
 - Speed⑪

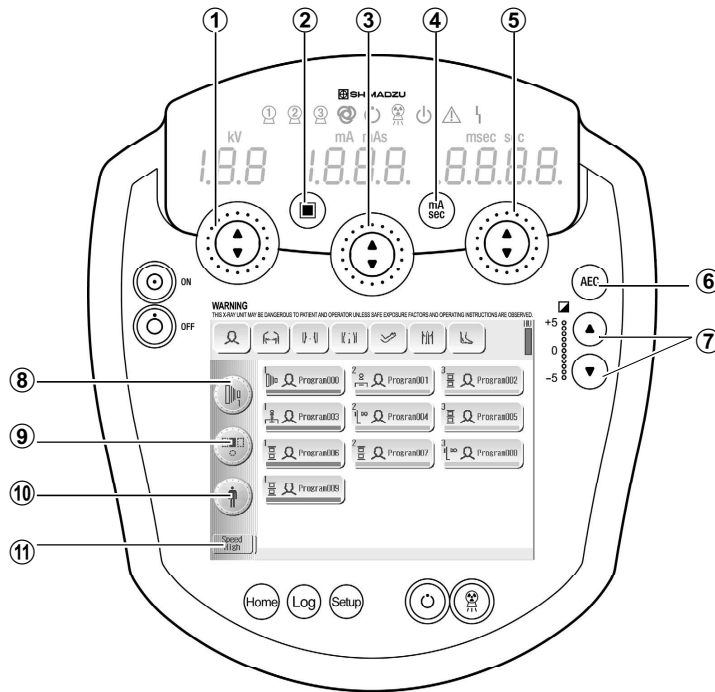

Reverts changed radiography conditions to their original settings.

Changed radiography conditions are displayed underlined.