

Medical Laminated Protective Overall TYPE 5B/6B

Product description:

Disposable medical laminated protective overall made of laminated spunbond, density 56 gr./m². It has hood with elastic and sewn chin flap, fastened with adhesive tape; zipper with protective flap and adhesive tape; booties - integral part of the overall; with elastic at hood, sleeves, ankles and waist. Provides an effective barrier against fluids (waterproof).

Instructions for use:


The equipment is worn over the work coat. The zipper opens, and the coverall is dressed starting with the feet, then the hands are inserted into the sleeves of the coverall, the elastic ends are arranged correctly, the hood is positioned on the head and, at the end, the zipper is closed and the protective flap is fixed with adhesive tape.

Undressing instructions:


The coverall will be undressed in the designated area, away from pollution and dangers; the help of another person is recommended, avoiding contamination of skin and clothing. Carefully remove the self-adhesive flap for the zipper, unzip the coverall and remove the hood making sure the gloves do not touch the inside of the coverall; then the coverall is totally removed. The gloves should be removed according to the manufacturer's instructions and with safety procedures. Dispose of the coverall in a suitable waste container.

Storage instructions:

Storage is done in the original packaging. Store at a temperature between 0 °C and + 20 °C; in a dry and clean place, away from moisture; away from direct sunlight, any flame, water droplets and solvent vapors.

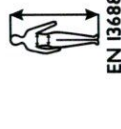

ISO 13982-1

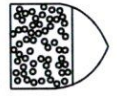

TYPE 5 B

EN 13034




TYPE 6 B

EN 14126



Model
YFS-03

2XL

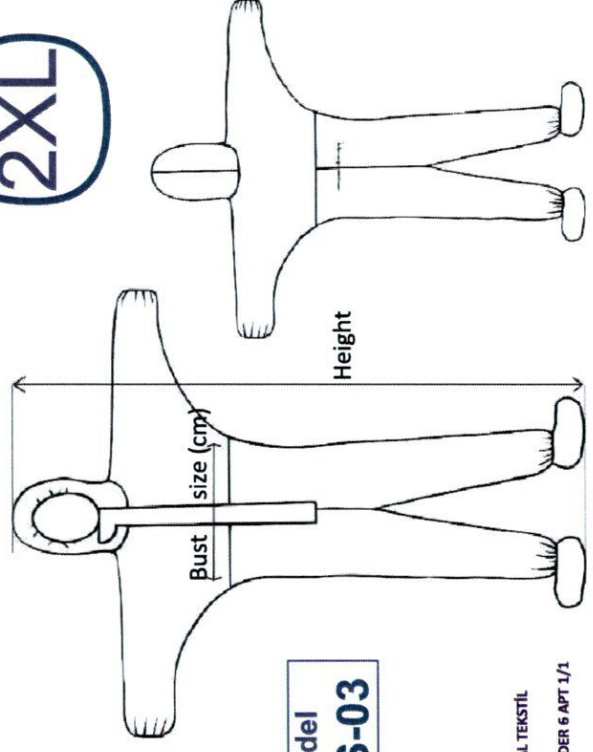

Registration number:

Size	Height (cm)	Bust size (cm)
L	170-176	132-136
XL	176-182	136-140
2XL	182-188	140-144
3XL	188-194	144-148



Do not iron



Do not bleach



Do not wash



Do not dry clean



Do not tumble dry



09.2021



09.2024

Manufacturer:



YFS GIDA TEMİZLİK AMBALAJ MEDİKAL TEKSTİL
SANAYİ TİCARET LİMİTED ŞİRKETİ
Adres: BAĞCILAR MAH.1084.SOK.ENDER 6 APT 1/1
BAĞLAR/DIYARBAKIR
Tel: +90 412 251 8526

Y.F.S. GIDA TEM. AMBALAJ
MEDİKAL TEKSTİL SAN. TIC. LTD. ŞTİ.
Tel: +90 412 251 8526 - Faks: +90 412 251 8526
Bağlar Mah. 1084. Sok. Ender 6 Apt. 1/1
Göşap Yolu 25100/172 Bağlar-DIYARBAKIR

Medical Laminated Protective Overall TYPE 5B/6B

Product description:

Disposable medical laminated protective overall made of laminated spunbond, density 56 gr./m². It has hood with elastic and sewn chin flap, fastened with adhesive tape; zipper with protective flap and adhesive tape; booties - integral part of the overall; with elastic at hood, sleeves, ankles and waist. Provides an effective barrier against fluids (waterproof).

Instructions for use:


The equipment is worn over the work coat . The zipper opens, and the overall is dressed starting with the feet, then the hands are inserted into the sleeves of the coverall, the elastic ends are arranged correctly, the hood is positioned on the head and, at the end, the zipper is closed and the protective flap is fixed with adhesive tape.

Undressing instructions:

The coverall will be undressed in the designated area, away from pollution and dangers; the help of another person is recommended, avoiding contamination of skin and clothing. Carefully remove the self-adhesive flap for the zipper, unzip the coverall and remove the hood making sure the gloves do not touch the inside of the coverall; then the coverall is totally removed . The gloves should be removed according to the manufacturer's instructions and with safety procedures. Dispose of the coverall in a suitable waste container.

Storage instructions:


Storage is done in the original packaging . Store at a temperature between 0 ° C and + 20 ° C; in a dry and clean place, away from direct sunlight, any flame, water droplets and solvent vapors.

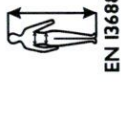

ISO 13982-1

EN 13034

EN 14126

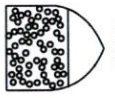


TYPE 5 B

TYPE 6 B



2841

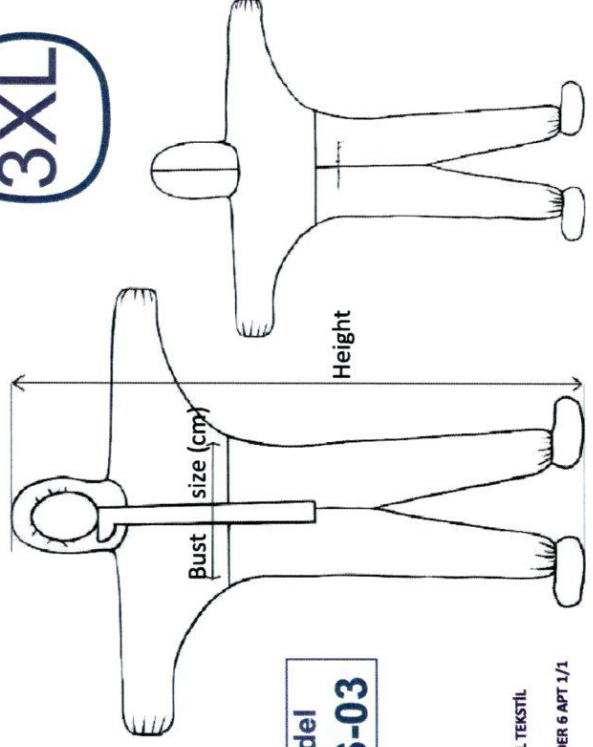

Registration number:

Size	Height (cm)	Bust size (cm)
L	170-176	132-136
XL	176-182	136-140
2XL	182-188	140-144
3XL	188-194	144-148



Do not iron



Do not bleach



Do not wash



Do not dry clean




Do not Tumble dry




Manufacturer:

YFS GIDA TEMİZLİK AMBALAJ MEDİKAL TEKSTİL
SANAYİ TİCARET LİMİTED ŞİRKETİ
Address: BAĞCILAR MAH.1084.SOK ENDER 6 APT 1/1
BAĞLAR/DIYARBAKIR
Tel: +90 412 251 8526

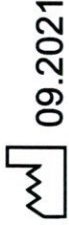

09.2021

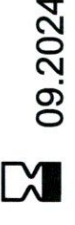

09.2024



Medical Laminated Protective Overall TYPE 5B/6B

Product description:

Disposable medical laminated protective overall made of laminated spunbond, density 56 gr./m2. It has hood with elastic and sewn chin flap, fastened with adhesive tape; zipper with protective flap and adhesive tape; booties - integral part of the overall; with elastic at hood, sleeves, ankles and waist. Provides an effective barrier against fluids (waterproof).

Instructions for use:

The equipment is worn over the work coat. The zipper opens, and the coverall is dressed starting with the feet, then the hands are inserted into the sleeves of the coverall, the elastic ends are arranged correctly, the hood is positioned on the head and, at the end, the zipper is closed and the protective flap is fixed with adhesive tape.

Undressing instructions:

The coverall will be undressed in the designated area, away from pollution and dangers; the help of another person is recommended, avoiding contamination of skin and clothing. Carefully remove the self-adhesive flap for the zipper, unzip the coverall and remove the hood making sure the gloves do not touch the inside of the coverall; then the coverall is totally removed. The gloves should be removed according to the manufacturer's instructions and with safety procedures. Dispose of the coverall in a suitable waste container.


Storage instructions:

Storage is done in the original packaging. Store at a temperature between 0 ° C and + 20 ° C; in a dry and clean place, away from moisture; away from direct sunlight, any flame, water droplets and solvent vapors.



EN 13688

ISO 13982-1

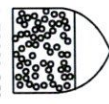

TYPE 5 B

EN 13034



TYPE 6B


EN 14126



2841

Model
YFS-03

L

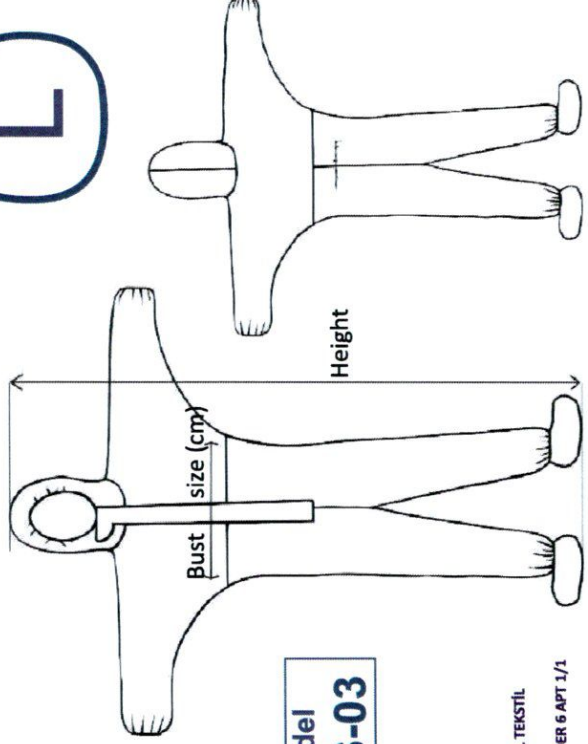

Registration number:

Size	Height (cm)	Bust size (cm)
L	170-176	132-136
XL	176-182	136-140
2XL	182-188	140-144
3XL	188-194	144-148



Do not iron



Do not bleach



Do not wash



Do not dry clean



Do not Tumble dry



09.2021



09.2024

Manufacturer:



YFS GIDA TEMİZLİK AMBALAJ MEDİKAL TEKSTİL
SANAYİ TİCARET LİMİTED ŞİRKETİ
Address: BAĞCILAR MAH.1094.SOK ENDER 6 APT 1/1
BAĞLAR/DIYARBAKIR
Tel: +90 412 251 8526

Y.F.S. GIDA TEMİZLİK AMBALAJ
MEDİKAL TEKSTİL SAN. TİC. LİM. ŞİT.
Tel: +90 412 251 8526
B. 1094 Sok. Ender 6 Apt. 1/1
Bağlar Mah. Diyarbakır

Medical Laminated Protective Overall TYPE 5B/6B

Product description:

Disposable medical laminated protective overall made of laminated spunbond, density 56 gr./m2. It has hood with elastic and sewn chin flap, fastened with adhesive tape; zipper with protective flap and adhesive tape; booties - integral part of the overall; with elastic at hood, sleeves, ankles and waist. Provides an effective barrier against fluids (waterproof).

Instructions for use:

The equipment is worn over the work coat. The zipper opens, and the coverall is dressed starting with the feet, then the hands are inserted into the sleeves of the coverall, the elastic ends are arranged correctly, the hood is positioned on the head and, at the end, the zipper is closed and the protective flap is fixed with adhesive tape.

Undressing instructions:

The coverall will be undressed in the designated area, away from pollution and dangers; the help of another person is recommended, avoiding contamination of skin and clothing. Carefully remove the self-adhesive flap for the zipper, unzip the coverall and remove the hood making sure the gloves do not touch the inside of the coverall; then the coverall is totally removed. The gloves should be removed according to the manufacturer's instructions and with safety procedures. Dispose of the coverall in a suitable waste container.

Storage instructions:

Storage is done in the original packaging. Store at a temperature between 0 ° C and + 20 ° C; in a dry and clean place, away from direct sunlight, any flame, water droplets and solvent vapors.



EN 13688

ISO 13982-1



TYPE 5 B

EN 13034




TYPE 6B

EN 14126



2841

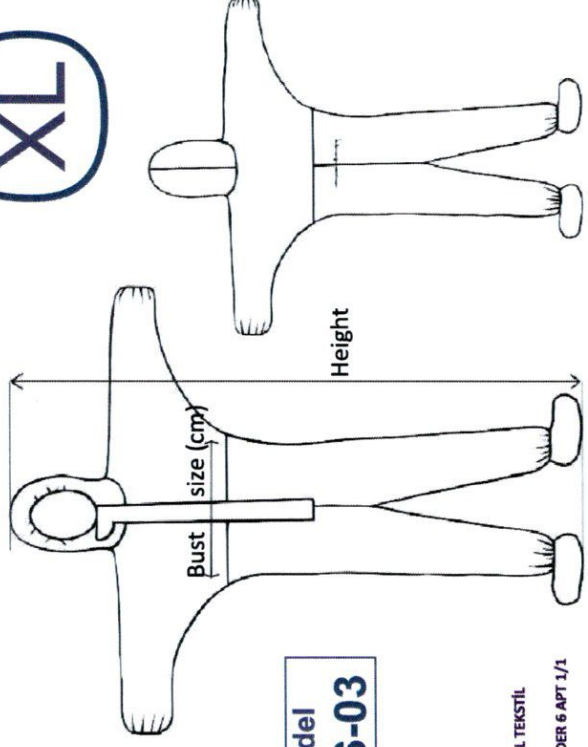

Registration number:

Size	Height (cm)	Bust size (cm)
L	170-176	132-136
XL	176-182	136-140
2XL	182-188	140-144
3XL	188-194	144-148



Do not iron



Do not bleach



Do not wash



Do not dry clean



Do not Tumble dry



09.2021



09.2024

Manufacturer:



YFS GIDA TEMİZLİK AMBALAJ MEDİKAL TEKSTİL
SANAYİ TİCARET LİMİTED ŞİRKETİ
Adress: BAĞCILAR MAH.1084.SOK.ENDER 6 APT 1/1
BAĞLAR/DIYARBAKIR
Tel: +90 412 251 9526

Y.F.S. GIDA TEMİZLİK AMBALAJ MEDİKAL TEKSTİL SANAYİ TİCARET LİMİTED ŞİRKETİ
Tel: +90 412 251 9526
Bağlar Mah. 1084 Sok. Ender 6 Apt 1/1
Gazipaşa Yolu No: 1084 Bağlar/Diyarbakır