

2.3 Touch Panel

■ Positioned Screen (Typical Screen)

CH-200 Tracking APR Mode Screen

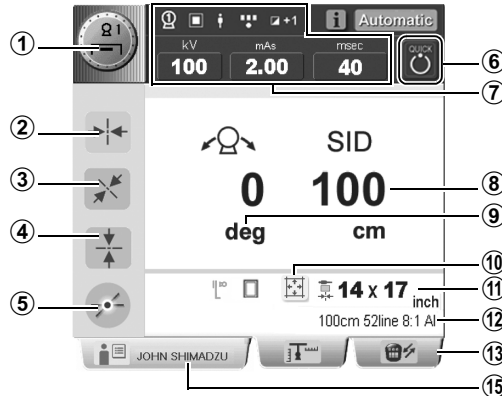


Fig. 2.6 CH-200 Tracking APR Mode

CH-200 Non-tracking Mode Screen

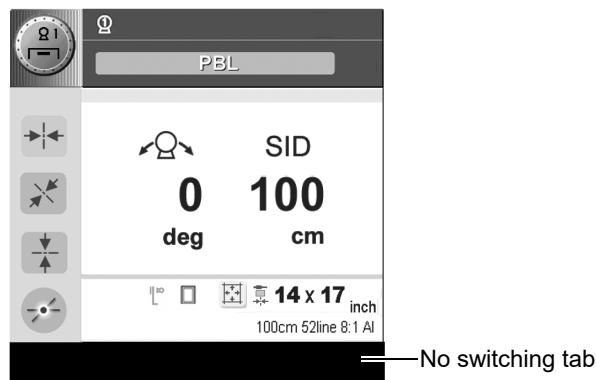


Fig. 2.7 CH-200 Non-tracking Mode

① Technique key

Shows the technique and No. of X-ray tube currently selected. They can be changed with this key.

② Longitudinal detent position indicator

Lights up when the X-ray tube support has reached the longitudinal setting position.

③ Transversal detent position indicator

Lights up when the X-ray tube support has reached the transversal setting position.

④ Vertical position indicator

Lights up when the X-ray tube support has reached the vertical setting position.