



Thermo Sealer User's Guide

Thank you for purchasing thermo sealer, please read this User's Guide carefully before using and preserve it for referring. Thank you !

Purpose

For packaging before disinfection.

Technology Parameters

Input Power :AC220V 50Hz Power 500W

Specialties

Constant temperature, short sealing time, fast seal edge, sharp paper knife, firm structure and it is really a durable appliance.

Operation process

1. Open the carton and check the parts of the appliance:
2. Plug on the power supply , install the pole and fit on the paper for disinfection.
3. Open the power supply and the indicator lights, adjust sealing temperature controller at the back of the machine(Usually at 150°C) the preheating lights on, and when the preheating lights off means the warm up finished, the sealing can be done.
4. Put the paper through the sealing area, pull the lever to the operator's chest and the heating indicator lights. The heating indicator turnoff after several seconds, then sealing is over. Press the paper knife and cut break the paper, then push the lever to the front ,and you can do another operation after several seconds.
5. When the temperature is lower than the set temperature in the continuous packaging process, the heating indicator lights up again. At this time, the operator can continue the packaging

Dimensions :

(Out side) 458x376x230(mm) G.W.:10.5kg

Storage Environment:

- Temperature: -40°C~55°C
- Humidity : ≤90% (R.H)
- Air pressure: 50kpa~110kpa

Working Environment:

- Temperature: 0°C~55°C
- Humidity : ≤80% (R.H)
- Air pressure: 80kpa~110kpa

Notice

1. Power and the shell should be connect with the earth reliable
2. Do not touch any part of machine under the cover when the machine working to avoid the scald

Packing List:

- | | |
|--------------------|---|
| 1. Sealing Machine | 1 |
| 2. Power cable | 1 |
| 3. User Manual | 1 |