

- Press the [ID] key on the alphanumeric keyboard or the custom switch.
  - The ID screen closes and the examination starts using the information displayed in the input area.
- NOTE: If you start an examination using a patient ID of an already examined patient, the following message appears: "This order is already performed. Do you want to perform an additional examination?" Select Yes to perform a new examination using the same study ID.
- NOTE: Selecting [ID] during an examination displays the ID screen, and the information can be edited. However, the patient ID and study ID cannot be edited.

## 4.3 Switching probes and applications

Use the steps below to switch probes and applications to be used for an examination.

For details about how to connect probes, see "Connecting a probe" in this manual.

For details about how to register applications to probes, see the separate manual "Basic Operations".

NOTE: Operation procedures vary depending on the [Probe << Preset Link] setting on the touch panel. Check the setting first.

NOTE: To switch to an application that is not displayed on the touch panel, select [All Preset]. All registered applications are displayed.

### Procedure

- By selecting an application, you can switch both an application and probe at the same time.
  - a. Press the [Probe/Preset] key.
  - b. Set [Probe << Preset Link] to On.
  - c. Select an application from the application preset switches (c) on the touch panel.
    - The probe will also switch at the same time.



(c) Application preset switch

- By selecting a probe, you can switch an application.
  - a. Press the [Probe/Preset] key.
  - b. Set [Probe << Preset Link] to Off.
  - c. Select a probe from the probe switches (c) on the touch panel.
  - d. If necessary, select an application from the application preset switches (d).