



MDKmed MS-31 Syringe Pump

Technical Data

Function spectrum

- 2.8" TFT Color and Touch Screen
- Horizontal flat and compact design with anti-corrosion material
- Online titration function: change flow rate during infusion with increment 0.01ml/h
- Occlusion Pressure: 9 levels range adjustable
- Automatic calculation of the delivery rate based on different modes
- Anti-bolus function for the safety of patients
- Infusion pause/standby: preset parameters remain stored
- Recognize syringe brands and size from 2ml to 60ml automatically
- Both audible and visible alarm, and 9 level alarm adjustable
- Smart occlusion removal: rotate reversely to release the infusion line pressure
- WiFi function

Physical specification

- W x D x H: 270 x 140 x 80 (mm)
- Net Weight: 1.5 kg
- Classification: Type CF, Class II, IPX2

Display

- Screen: 2.8" TFT touch color LCD
- Brightness: 1-9 levels adjustable
- Information: System time, infusion mode, IV set brand, battery capacity, infusion status, remaining time, VTBI, total volume, pressure limit, alarms, TVI, drugs
- Mode: Daytime mode and night time mode interchange automatically

- CPU type** Dual-CPU (ARM Cortex)

Syringe selection

- 2ml, 5ml/6ml, 10ml, 20ml, 30ml, 50/60ml
- Automatic recognition of syringe size and fixation
- Customize and Calibration more than 4 brands of syringe by user
- Can be installed and calibration new brand by Manufacturer

Basic parameters

- Mode: 8 modes: Rate mode, V-T, R-T, R-V, Drug Library mode, Body Weight mode, Relay mode, Gradient mode
- Flow rate: 2ml: 0.01-100ml/h, 5ml: 0.01-150ml/h, 10ml: 0.01-400ml/h, 20ml: 0.01-600ml/h, 30ml: 0.01-1000ml/h, 50/60ml: 0.01-2100ml/h
- Increment: 0.01ml(0.1-99.99ml/h), 0.1ml(100-999.9ml/h), 1ml(1000-2100ml/h)
- Preset time: 00:01-99:59 (hh:mm)adjustable
- KVO(keep vein open): 0.01-3.0ml/h
- Purge: 0.01-2100ml/h (depending on syringe size)

Accuracy

- ±2% (Recommended Syringe)

Bolus

- Bolus rate: Automatic/Manual Bolus: 0.01-2100ml/h (depending on syringe size)
- Preset bolus volume: Automatic bolus: 0.10-9999.99ml

Automatic Bolus:	0.01-3.0ml/h
Language	2 languages: English, Chinese (Support increase Multiple Languages)
Safety system	
Occlusion pressure:	Pressure unit: KPa
	9 level occlusion pressure, 26kpa(±20kPa)~130kpa(±30kPa)
Alarm	
Alarm type:	Visual and acoustic alarms
	Syringe size error, Plunger error, Occlusion alarm, VTBI complete alarm,
	Out of battery alarm, Battery/mains power double disconnect alarm, Malfunction alarm,
	Pause over time alarm, Internal battery low voltage alarm, Injection near to end alarm.
Alarm volume:	1-5 levels selectable
Battery:	
	Charging Time: ≤4 hours. for 100% of the capacity
	Working Time: ≥7 hours @5ml/h
Power supply	AC power: 100-240V~, 50/60Hz; DC power: 10-15V, 2A
Environmental requirement	
Temperature:	5°C ~ 40°C for Operating; -30°C ~ 55°C for Storage
Humidity:	20% ~ 90% for Operating; ≤75% for Storage
Air pressure:	80.0kPa ~ 106.0kPa
Water-proof:	IPX2
Lifetime	> 5 years under continuous duty conditions
Drug Library	
Programmable:	It can program drug list according to customer request
	Drug Library should program by customer according local regulation and clinic demands.
	Drug Library: 200 drugs / Event logs: 200 cases <i>(Can extend to 2,000 drugs/cases)</i>

5F, No. 28 Cangling Road, Huzhen Town, Jinyun
County, Lishui City, Zhejiang, China
Tel: +86 0578-3555507 Fax: +86 0578-3555507
Contact: Max Pan Whatsapp: +86-13678907329
E-mail: info.mdkmed@gmail.com
www.mdkmed.com



Zhejiang MDKingdom Technology Co., Ltd. (MDK) provides the most comprehensive infusion therapy products. We are dedicated to medical pumps and infusion therapy solution. We do our best to support your health!