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Qualmedi Technology Co.,Ltd.

Address:

A302 Room, 23 HangBu Road, High-tech Zone, Hefei City, Anhui Province, China.

EC-Representative:

Kingsmead Service B.V.

Address:

Zonnehof, 36 2632 BE Nootdorp

We declare under our sole responsibility that

Products: Vein Finder **Model:** QV-500, QV-600
UMDNS: 14346
Class: I (According to Annex VIII Rule 13 of Regulation (EU) 2017/745)

Intended Use: Vein finder can help medical professionals to locate certain superficial veins. This equipment is intended to be used as a supplement to appropriate medical training and experience.

BASIC UDI-DI code: QV-500(697401159QV-500DK / GS1)

QV-600(697401159QV-600DQ / GS1)

Manufacturer SRN code: CN-MF-000028309

EC Representative SRN code: NL-AR-000002066

Conformity assessment procedure: Conformity assessment procedure According to Art. 52 section 1 and 7 Regulation EU 2017/745

meet the provisions of the Regulation (EU) 2017/745 in national laws which apply to it, and of RoHS Directive 2011/65/EU Annex II amending Annex (EU) 2015/863 and amending Annex (EU) 2017/2102

Applied standards:

EN ISO 15223-1:2021	EN ISO 14971:2019/A11:2021	ISO 13485:2016
EN ISO 20417:2021	IEC 60601-1-2:2014+A1:2020	IEC 60601-1:2005+AMD1:2012
MEDDEV 2.7.1: REV4.	EN ISO 10993-1:2020	

The CE Mark:

For and on behalf of
Qualmedi Technology Co., Limited
安徽康沐醫療器械科技有限公司

The above-mentioned declaration of conformity is exclusively under the responsibility of...
Jason Zhang
Authorized Signature(s)

Company: Qualmedi Technology Co.,Ltd.

Address: A302 Room, 23 HangBu Road, High-tech Zone, Hefei City, Anhui Province, China.

Hefei, 2023-01-01

Place, Date

General Manager, JASON ZHANG

Duly authorized to sign this Authorization on behalf of:
Qualmedi Technology Co., Ltd.

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SNQA 查询网站：www.snqa.com.cn

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0015

This is to certify that the Quality Management System of

Qualmedi Technology Co., Ltd

Unified Social Credit Code: 91340102MA2UGH175J

Operation Address: A302 Room, Anhui Zhongke Zicheng Incubator, 23 Hangbu Road, High-tech Zone, Hefei City, Anhui Province, China

Registered Address: A302 Room, Anhui Zhongke Zicheng Incubator, 23 Hangbu Road, High-tech Zone, Hefei City, Anhui Province, China

applicable to

Production and sales of vein finder (within the scope of licensed qualifications)

has been assessed and registered by NQA against the provisions of

ISO 13485:2016

This registration is subject to the company maintaining a quality management system, to the above standard, which will be monitored by NQA.

Certified Clients shall accept regular surveillance assessments, the validity of certificates shall be maintained for the positive result of audit.

The information of this certificate can be checked on CNCA's website (www.cnca.gov.cn)

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USER MANUAL

Vein Finder V3.1

**PLEASE READ THIS OPERATION MANUAL CAREFULLY
BEFORE OPERATING THE VEIN FINDER**

Intended Use

Vein finder can help medical professionals to locate certain superficial veins. This equipment is intended to be used as a supplement to appropriate medical training and experience. It should not be used as the sole method for locating veins. And should be used only by a qualified medical professional, who should do so either prior to palpation to help identify the location of a vein, or afterwards to confirm or refute the perceived location of a vein. When using the device, medical practitioners should always follow the appropriate medical protocols and practices as required by their medical facility, as well as exercise sound medical judgment.

When used properly, it enables users to locate certain superficial veins in connection with medical procedures, such as venipuncture.

This vein finder can be used whenever the determination of vein location is appropriate such as hospitals and clinics.

Product Description

Vein finder operates by using infrared light to detect veins beneath the skin, then projecting the position of the veins on the skin surface directly above the veins. Qualified medical personnel can observe the vasculature as displayed to assist them in finding a vein of the right size and position for venipuncture and other medical procedures requiring the location of superficial veins. No training is required to operate this device.

The device only shows superficial vasculature. The maximum depth that veins are displayed varies by patient. In addition, some patients' veins or a portion of their veins might not be displayed well or at all. Causes for less than optimal or lack of vein display include, but are not limited to, vein depth, skin conditions (e.g., eczema, tattoos), hair, scarring or other highly contoured skin surface, and adipose (fatty) tissue.

When held directly overhead, the device accurately locates the center of a vein. Increasing the displacement from directly overhead results in an offset in the displayed vein position. Width of displayed vein may differ from the actual width depending on patient to patient differences and vein width. The center line of the vein is accurate when the device is being used correctly and should always be used as the target when performing venipuncture or other medical procedures.

Vein finder is portable machine. And the optional is table stand and mobile stand.

Use and Operation


Hold the device from 15 to 25 cm over the surface of the skin. Scan the area of interest to view. Once a vein is selected, make sure the vein display light is centered directly above the vein's center line. Tilting the device to either side of the vein will offset the projected vein from its true location beneath the skin. You can often enhance display quality by slightly adjusting the height and angle to the skin. In particular, moving the device closer or further from the skin can help bring additional veins into view, depending on the patient's vasculature, room lighting, and depth of the veins.

While the vein display light is on, shine the device over the patient's skin. You can do this before palpation, use sport mode scanning quickly over the skin to help narrow down possible locations. After confirming suitable vein location, you can change to enhance mode to find the deep veins.

After assessing the patient's vasculature, confirm the site for your procedure by verifying the location and suitability of the vein using normal medical techniques and good medical judgment, such as vein visualization, palpation, and other medical techniques

Operating Instructions

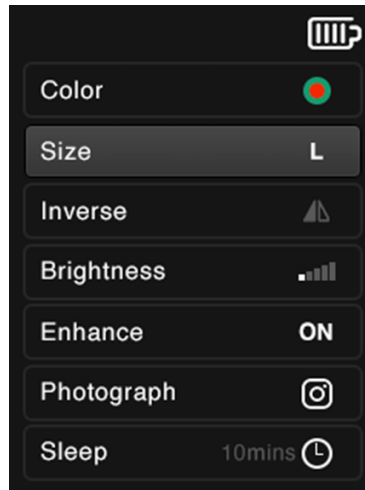LCD User Interface

Key

Power, Sleep, OK, and Four Arrow Keys

Key Power		Short press the button to startup or shutdown. Automatic power off when machine idle for 35 minutes. Light indicator: Blue: Working; Green: Fully charged; Red: Charging
Key Sleep		Machine enter sleep mode automatically when idle for 10mins(default) or set idle time by LCD interface. Short press make machine enter low energy consumption mode immediately.
LCD Color		Using the up and down arrow keys, select row "Color" in LCD and press the "OK" key or the left and right arrow keys to choose appropriate color.
LCD Size		Using the up and down arrow keys, select row "Size" in LCD and press the "OK" key or the left and right arrow keys to choose appropriate size.
LCD Inverse		Using the up and down arrow keys, select row "Inverse" in LCD and press the "OK" key or the left and right arrow keys to turn on/off inverse mode.
LCD Brightness		Using the up and down arrow keys, select row "Brightness" in LCD and press the "OK" key or the left and right arrow keys to choose appropriate brightness.
LCD Enhance		Using the up and down arrow keys, select row "Inverse" in LCD and press the "OK" key or the left and right arrow keys to turn on/off enhance mode.
LCD Photograph		Using the up and down arrow keys, select row "Photograph" in LCD and press the "OK" key or the left and right arrow keys to save real-time vein images. Captured veins can be accessed by computer through Type-C cable.
LCD Sleep		Using the up and down arrow keys, select row "Sleep" in LCD and press the "OK" key or the left and right arrow keys to set sleep idle time.

Technical parameters

Infrared light detection without harm to human body.

Infrared Wave: 760-940nm

Infrared detection depth: 0-10mm

Best detection distance: 18-30cm

Accuracy of blood vessel position: $\pm 0.3\text{mm}$

Accuracy of blood vessel resolution: $\pm 0.3\text{mm}$

Low working noise: $\leq 40\text{DB}$

The battery capacity can be displayed.

Low Battery capacity prompt.

Samsung 5000mAh rechargeable lithium battery, 3.5-4 hours duration.

TI DLP projection technology.

Power supply of charging: 5V 5A, 100V-240V 50Hz-60Hz

Size: 24(L)*6(W)*6.5(H)cm

Symbols

	<p>Refer to instruction manual</p>
	<p>European Conformity. This symbol means that the device fully complies with European Directive 93/42/EEC</p>
	<p>Product Serial Number</p>
	<p>Reference to Catalog Model Number</p>
	<p>European Representative</p>
	<p>Manufacturer</p>
	<p>European Union Waste Electrical and Electronic Equipment Directive Logo. Return is allowed for proper disposal.</p>
	<p>Keep dry</p>
	<p>Fragile, handle with care</p>
	<p>Manufacturing date</p>

Packing List

No	Name	Quantity
1	Main Machine	1
2	Charging cable	1
3	Charging adapter	1
4	Aluminum Carrying Case	1
5	User's manual	1
6	Calibration card	1
7	Product certification	1
8	Warranty card	1

Maintenance

- Maintain and clean the device once a month.
- Please unplug power cable and make sure device is off before cleaning.
- Please use the dust-free cloth to dip concentration of 70% alcohol to wipe the device.
- Should be returned to factory for servicing if the vein light window is scratched.
- If you do not use the device for more than 3 months, please remove the battery from Device. This device is with Independent battery compartment. It is easy to take out the battery.
- Please do not open the device to clean the inner of the machine.
- After using, please put the device to aluminum carrying case.
- Please store in a dry and cool place.

Warnings and Cautions

- This vein finder should be used as a supplement for qualified professionals to determine the location of the veins.
- This vein finder is dependent on a variety of patient factors and may not display veins on patients with deep veins, skin conditions, hair, scarring or other highly contoured skin surface, and adipose (fatty) tissue.
- This vein finder displays only superficial veins and does so only to limited depths dependent on a variety of patient factors. The device does not indicate vein depth.
- This vein finder should not be used to locate veins in the eyes.
- Do not shine vein display light to the eyes for a long time.
- The device projection may not display veins if operated under bright light such as bright sunlight.
- This vein finder is not intended to be used as a diagnostic device or for treatment of any kind.
- Keep the instrument dry and clean to prevent any fluid from entering the machine.
- This vein finder must be repaired by authorized and qualified staff.
- Do not attempt to open, disassemble, or service the battery.
- Do not attempt to sterilize the device with heat or pressure sterilization.
- Electromagnetic Interference can affect the performance of the device.
- The Machine is fragile, do not falling.
- Using unofficial accessories can reduce safety, so use only official accessories.
- Do not throw this vein finder and battery to fire.

**Guidance and Manufacturer's Declaration
Electromagnetic Emissions
IEC 60601-1-2**

<p>The Vein Finder is suitable for use in the specified electromagnetic environment. The customer and/or the user of the Vein Finder should assure that it is used in an electromagnetic environment as described below:</p>		
Emissions Test	Compliance	Electromagnetic Environment Guidance
RF emissions CISPR 11	Group 1	The Vein Finder uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The Vein Finder is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Guidance and Manufacturer's Declaration

Electromagnetic Immunity

IEC 60601-1-2

<p>The Vein Finder is suitable for use in the specified electromagnetic environment. The customer and/or the user of the Vein Finder should assure that it is used in an electromagnetic environment as described below:</p>			
Immunity Test	IEC 60601-1-2 Test Level	Compliance Level	Electromagnetic Environment Guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete, or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	± 2 kV for power supply lines ± 1 kV for input/output lines	Mains power quality should be that of a typical commercial and/or hospital environment
Surge IEC 61000-4-5	± 1 kV differential mode ± 2 kV common mode	± 1 kV differential mode ± 2 kV common mode	Mains power quality should be that of a typical commercial and/or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	$< 5\% U_T$ ($> 95\%$ dip in U_T) for 0,5 cycle $40\% U_T$ (60% dip in U_T) for 5 cycles $70\% U_T$ (30% dip in U_T) for 25 cycles $< 5\% U_T$ ($> 95\%$ dip in U_T) for 5 sec	$< 5\% U_T$ ($> 95\%$ dip in U_T) for 0.5 cycle $40\% U_T$ (60% dip in U_T) for 5 cycles $70\% U_T$ (30% dip in U_T) for 25 cycles $< 5\% U_T$ ($> 95\%$ dip in U_T) for 5 sec	Mains power quality should be that of a typical commercial and/or hospital environment. If the user of the Vein Finder requires continued operation during power mains interruptions, it is recommended that the Vein Finder be powered from an uninterruptible power supply or a battery.

Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	0.3 A/m	If image distortion occurs, it may be necessary to position the Vein Finder further from sources of power frequency magnetic fields or to install magnetic shielding. The power frequency magnetic field should be measured in the intended installation location to assure that it is sufficiently low.
NOTE U_T is the a.c. mains voltage prior to application of the test level.			

Guidance and Manufacturer's Declaration
Electromagnetic Immunity
IEC 60601-1-2

The Vein Finder is suitable for use in the specified electromagnetic environment. The customer and/or the user of the Vein Finder should assure that it is used in an electromagnetic environment as described below:

Immunity Test	IEC 60601-1-2 Test Level	Compliance Level	Electromagnetic Environment Guidance
<p>Conducted RF IEC 61000-4-6</p> <p>Radiated RF IEC 61000-4-3</p>	<p>3 Vrms 150 kHz to 80 MHz</p> <p>3 V/m 80 MHz to 2,5 GHz</p>	<p>3 Vrms</p> <p>3 V/m</p>	<p>Portable and mobile RF communications equipment should be used no closer to any part of the Vein Finder, including cables, than the recommended separation distance calculated from the equation appropriate for the frequency of the transmitter.</p> <p style="text-align: center;">Recommended Separation Distance</p> $d = 1,2\sqrt{p}$ $d = 1,2\sqrt{p} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = 2,3\sqrt{p} \quad 800 \text{ MHz to } 2,5 \text{ GHz}$ <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^b</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> <div style="text-align: center;">  </div>

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast, and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Vein Finder is used exceeds the applicable RF compliance level above, the Vein Finder should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Vein Finder.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

NOTE These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

Recommended Separation Distances
between Portable and Mobile RF Communications Equipment
and the Vein Finder
IEC 60601-1-2

Frequency of Transmitter Equation Rated Maximum Output Power of Transmitter watts	150 kHz to 80 MHz $d=1,2 \sqrt{P}$ Separation Distance metres	150 kHz to 800 MHz $d=1,2 \sqrt{P}$ Separation Distance metres	800 MHz to 2,5 GHz $d= 2,3 \sqrt{P}$ Separation Distance metres
0.01	0,12	0,12	0,23
0.1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23
<p>For transmitters rated at a maximum output power not listed above, the separation distance can be estimated using the equation in the corresponding column, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.</p> <p>NOTE These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.</p>			