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#### **VacuAptaca® Vacuum collection disposable holder.**

Compatible with all VacuAptaca® devices. The Holder standard is compatible with the majority of multiple needles and needles butterfly today in commerce. It is a disposable product to be used in accordance with the best practices in place at health care facilities.

Manufactured in transparent, no-toxic polypropylene (PP), latex free.

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*Intended purpose is "MEDICAL DEVICE" Class I Not Sterile (EU Regulation 2017/745 - MDR) - Holder disposable, no sterile, intended to be used in combination with a needle for vacuum tube for blood sampling.*

**For professional use only.**

**National classification of medical devices (CND) > A01018001 (Holder for sampling needles)**

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*After collecting the last tube, remove the needle from the vein and dispose all devices (Holder, Luer adapter, multi-sample needle, butterfly needle, etc.) in special containers in accordance with the present regulations. No re-hooding needles, there is the risk of injury by puncturing and relative danger of infection. All sampling devices must be considered potentially infected after use.*

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- > *To be used exclusively for the sampling and analysis of serum, plasma or whole blood in clinical laboratories;*
- > *Do not change the intended use;*
- > *To be used only with compatible accessories (needles and collection set, butterfly needles, luer adapters, shirts, etc.);*
- > *It is not recommended to transfer the blood sample from a syringe to the vacuum tube;*
- > *Do not use the product expired or damaged;*
- > *Use the device with aseptic procedures and using appropriate DPE (gloves, eye protectors, suitable clothing, etc.);*
- > *Do not reuse, disposable device;*
- > *Do not bring the device to flame or heat sources;*
- > *Keep in a cool and dry place, min temperature +4 °C (39.2 °F)/max +30 °C (86 °F). Do not expose to sunlight;*
- > *Every device, after use, may contain infectious microorganisms, use the appropriate PPE and dispose specimen and sampling accessories (needles and collection set, levy set butterfly, luer adapters, shirts, etc.) according to the regulations on force.*

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